



<b>Product name</b>	<b>Host computer XW8000 XW8200</b>
Identification code(s)	4522 132 3313x 4522 132 2720x 4522 132 4469x 4522 132 3489x 4522 132 1967x  4522 132 4481x 4522 132 4482x 4522 132 5456x 4522 132 6371x
Total weight [kg]	24
<b>Manufacturer</b>	<b>Hewlett Packard</b>
Address	Orteliuslaan 1000
Zip code	3528 BD Utrecht
Country	The Netherlands
Electronic info	

## Recycling information


<b>Batteries</b>	<b>Items, type</b>	<b>Location</b>
	Lithium battery type CR2032 (3V)	Figure 1

<b>Hazardous substances</b>	<b>Items</b>	<b>Location</b>
	Printed Circuit Boards (PWB) one in the power supply	Figure 2
	Printed Circuit Boards (PWB) one systemboard (motherboard)	Figure 3
	Printed Circuit Boards (PWB) one add-in card	Figure 4
	Electrolytic capacitors/ condensers greater then 2.5cm in diameter or height: Three attached to power supply boards.	Figure 2

Name:	<b>Recycling Passport Host Computer XW8xxx</b>	12NC:	<b>4522 132 44690</b>	Version: <b>01</b>
Author:		Pag.: <b>1 of 4</b>	Group No: <b>5790</b>	Doc.Date: <b>2007-07-31</b>
			Section: <b>-</b>	Status: <b>Authorized</b>

**Locations as mentioned in the Passport ( Pictures information)**

**Figure 1**

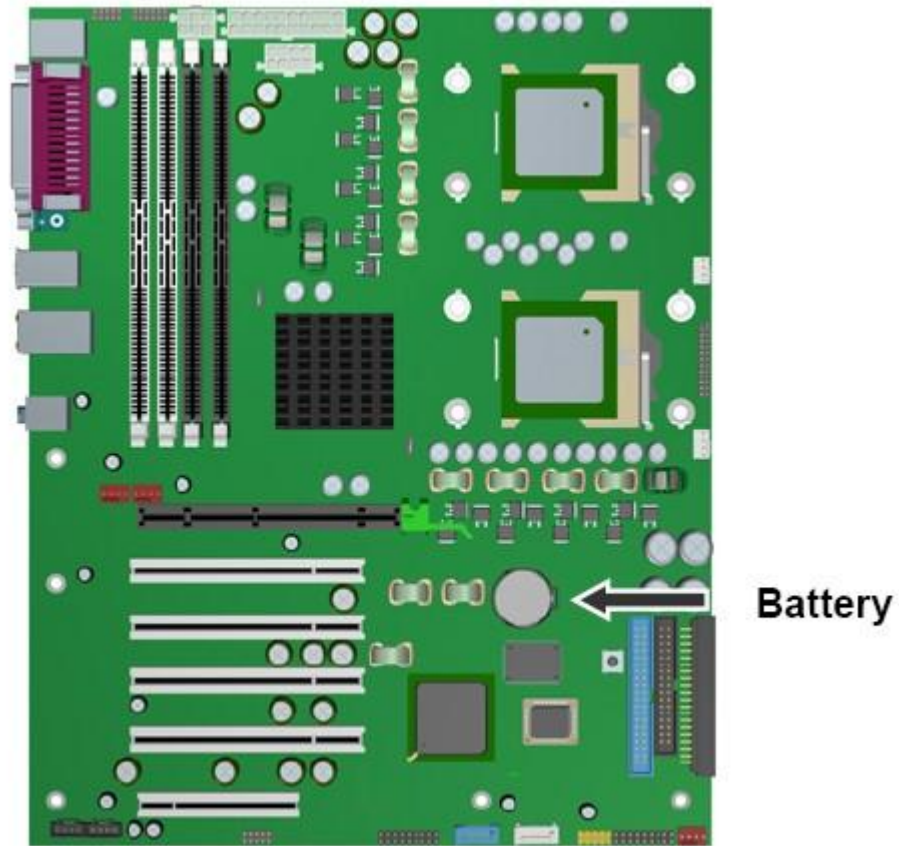

Name: <b>Recycling Passport Host Computer XW8xxx</b>		12NC: <b>4522 132 44690</b>		Version: <b>01</b> Doc.Date: <b>2007-07-31</b>	
Author:		Pag.: <b>2 of 4</b>	Group No: <b>5790</b>	Section: <b>-</b>	Status: <b>Authorized</b>

© PHILIPS MEDICAL SYSTEMS NEDERLAND BV, BEST 2007 Template version 2.

**Figure 2**




Name:	<b>Recycling Passport Host Computer XW8xxx</b>	12NC:	<b>4522 132 44690</b>	Version: <b>01</b>
Author:		Pag.: <b>3 of 4</b>	Group No: <b>5790</b>	Doc.Date: <b>2007-07-31</b>
			Section: <b>-</b>	Status: <b>Authorized</b>

© PHILIPS MEDICAL SYSTEMS NEDERLAND BV, BEST 2007 Template version 2.

**Figure 3**

Remove the system board. Slide the board forward ❶ to disengage the mounting standoffs from the chassis. Lift the board out of the chassis ❷.

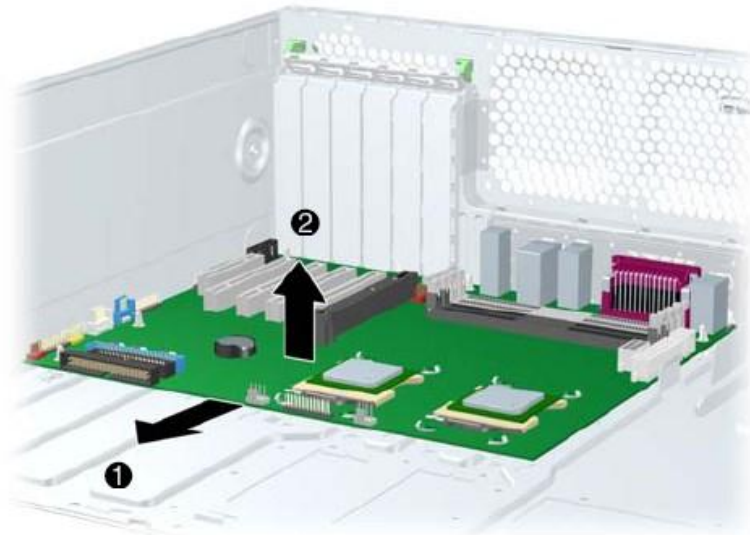


**Figure 4**

Remove the PCI retainer by pressing down on it, then out ❶.

Lift the PCI card out of the chassis ❷.

For PCI Express cards, first remove the power supply cable (not shown) if required. Then move the release lever ❸ and lift the card out of the chassis.

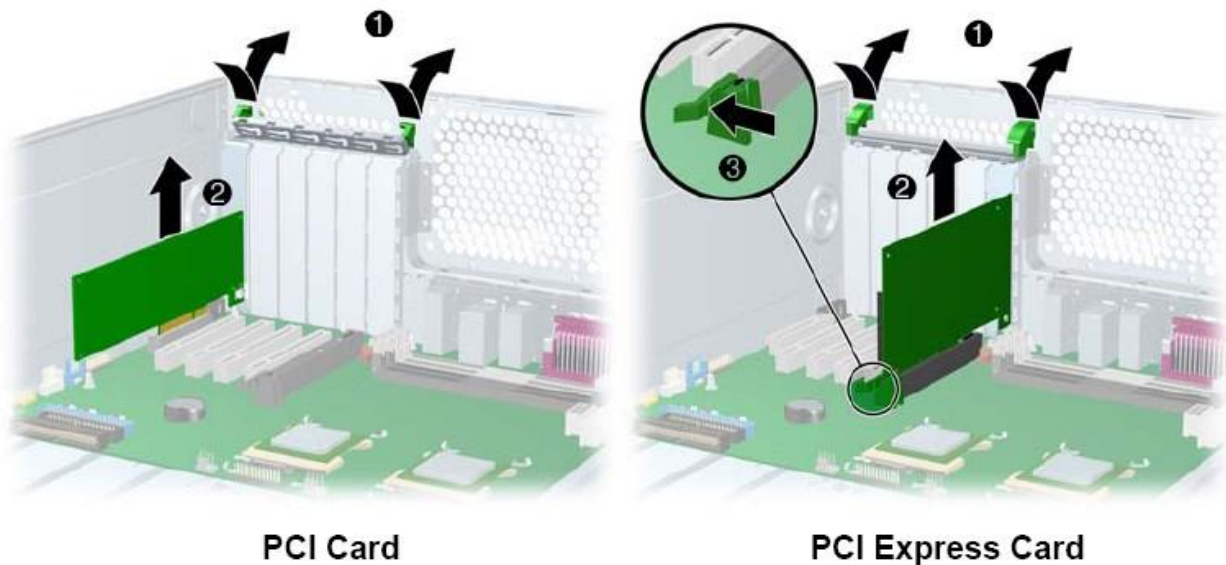

Name:	<b>Recycling Passport Host Computer XW8xxx</b>	12NC:	<b>4522 132 44690</b>	Version: <b>01</b>
Author:		Pag.: <b>4 of 4</b>	Group No: <b>5790</b>	Doc.Date: <b>2007-07-31</b>
			Section: <b>-</b>	Status: <b>Authorized</b>