



TrueBlue – your complete package.

TrueBlue gel nasal mask with Auto Seal technology.

PHILIPS
RESPIRONICS
sense and simplicity

TrueBlue — the first blue gel nasal mask with Auto Seal technology.

Designed to turn heads.

Meet TrueBlue, bringing our best technologies together in one mask. Designed to push the limits of performance and fit, this mask sets the gold standard for a good night's sleep.

The TrueBlue gel nasal mask with Auto Seal is designed to deliver a higher degree of comfort, stability, and freedom of movement with minimal adjustments.

TrueBlue is available in five comfortable sizes: petite, small, medium, medium wide, and large.


This intuitive gel mask is designed for quicker set-ups and fittings, fewer callbacks, and easier lab titrations. With TrueBlue, you can deliver a more complete sleep solution. Why settle for anything less?

Designed to the highest standards — yours.

Headgear has inlaid adjustable straps that converge into a single back panel with a crown strap for added stability.

Premium blue gel works together with the thin, form-fitting outer silicone membrane to create an effective, self-adjusting seal. Not only does our blue gel provide extra stability, it is thinner and lighter than ever before.


Freeform spring gives patients the ability to move in any direction. The soft silicone spring helps patients to get a better night's sleep by limiting disturbances to the seal.

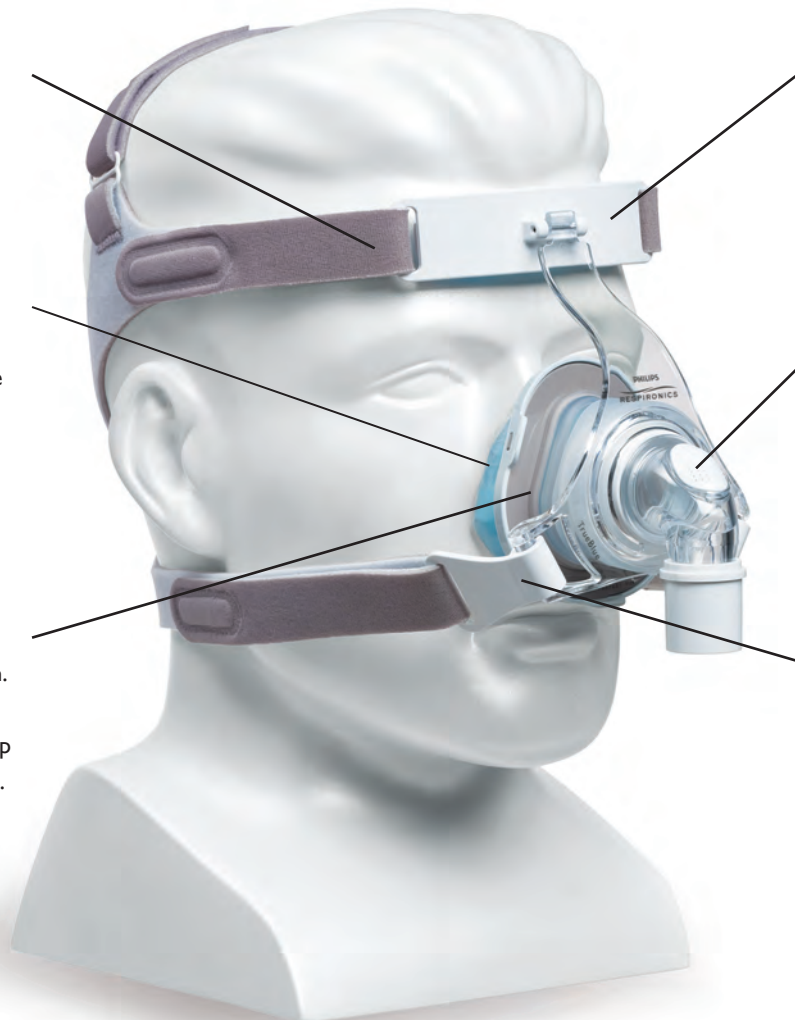

Intuitive forehead pad is designed expressly for TrueBlue and enhanced with a remarkably soft premium blue gel.

Angled exhalation micro ports make operation quieter and redirect air away from a bed partner.

Talon clips are included, making it easy for patients to remove the mask and lock in the same fit, night after night.

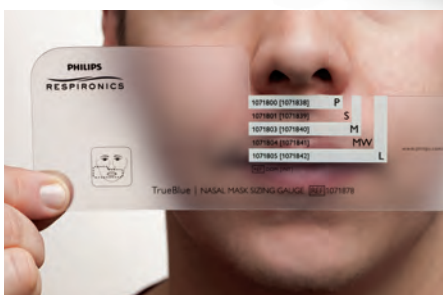

Sizing tip

Your TrueBlue sizing gauge is designed to easily and quickly determine the right size mask for every patient.



Scan with your smart phone to see a video on the Auto Seal technology that makes TrueBlue the complete package.

Fitting instructions



Disconnect one talon clip.
Connect the flexible tubing from your therapy device to the swivel on the mask. Slide the headgear on over your head. Turn on the airflow before adjustments are made to activate the Auto Seal technology. Reattach the clip.



Evenly adjust the side and lower straps. **Do not overtighten:** the cushions should rest lightly on the face.



Adjust the crown strap so the upper straps rest above the ears. The upper and lower straps should be parallel to one another.



Make final adjustments while lying down.

Note: Talon clips do not need to be used. If preferred, they can be removed and the straps can be threaded directly through the mask frame where the clips attach.

Disassembly and assembly instructions



Pull outward on the three attachment tabs on each side of the mask cushion until it separates from the mask frame. Remove the gel cushion and flap to allow for cleaning.



To replace the gel cushion, position it inside the flap and press into place.



Place the flap and gel cushion over the spring and press down along the sides and bottom of the flap until it clicks into place.



To remove the gel forehead pad, simply grasp the pad and peel it away from the forehead support bracket. To replace, line up the tabs on the bracket so they are over the white base of the gel pad and press into place.

Ordering information

TrueBlue mask	Petite	Small	Medium	Medium wide	Large	U.S. HCPCS
Mask with headgear	1071800	1071801	1071803	1071804	1071805	A7034, A7035
Mask only	1071806	1071807	1071808	1071809	1071810	A7034
DuoPack: includes headgear and two cushions with flaps of the same size	1071818	1071819	1071820	1071821	1071822	A7034, A7035, A7032
FitPack: includes headgear and two cushions with flaps in two sizes		1071816 small and medium		1071817 medium wide and large		

Replacement parts	Part number	U.S. HCPCS
Petite cushion and flap	1071861	A7032
Small cushion and flap	1071862	A7032
Medium cushion and flap	1071863	A7032
Medium wide cushion and flap	1071864	A7032
Large cushion and flap	1071865	A7032

Replacement parts	Part number	U.S. HCPCS
TrueBlue headgear	1071875	A7035
TrueBlue headgear, reduced size	1071874	A7035
TrueBlue gel forehead pad	1071888	N/A
TrueBlue headgear clips (pack of 2)	1071877	N/A

**Philips Healthcare is part of
Royal Philips Electronics**

How to reach us

www.philips.com/healthcare
healthcare@philips.com

Asia
+49 7031 463 2254

Europe, Middle East, Africa
+49 7031 463 2254

Latin America
+55 11 2125 0744

North America
+1 425 487 7000
800 285 5585 (toll free, US only)

Philips Respironics
1010 Murry Ridge Lane
Murrysville, PA 15668

Customer Service
+1 724 387 4000
800 345 6443 (toll free, US only)

Philips Respironics International
Headquarters
+33 1 47 28 30 82

Philips Respironics Asia Pacific
+65 6882 5282

Philips Respironics Australia
+61 (2) 9666 4444

Philips Respironics China
+86 021 24127311

Philips Respironics Deutschland
+49 8152 93 06 0

Philips Respironics France
+33 2 51 89 36 00

Philips Respironics Italy
+39 039 203 1

Philips Respironics Sweden
+46 8 120 45 900

Philips Respironics Switzerland
+41 6 27 45 17 50

Philips Respironics United Kingdom
+44 800 1300 845

www.philips.com/respironics

Respironics, DuoPack, FitPack, and TrueBlue are trademarks of Respironics, Inc. and its affiliates. All rights reserved.

Please visit www.philips.com/trueblue



© 2011 Koninklijke Philips Electronics N.V.
All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.
Geyer JJ 6/15/11 MCI 4104304 PN 1084459