

**Philips Healthcare is part of  
Royal Philips Electronics**

**How to reach us**

www.philips.com/healthcare  
healthcare@philips.com  
fax: +31 40 27 64 887

Asia  
+852 2821 5888

Europe, Middle East, Africa  
+49 7031 463 2254

Latin America  
+55 11 2125 0744

North America  
+1 425 487 7000  
800 285 5585 (toll free, US only)

Philips Respiration  
1010 Murry Ridge Lane  
Murrysville, PA 15668

Customer Service  
+1 724 387 4000  
800 345 6443 (toll free, US only)

Respiration Asia Pacific  
+65 6298 1088

Respiration Australia  
+61 (2) 9666 4444

Respiration Deutschland  
+49 8152 93 06 0

Respiration Europe, Middle East, Africa  
+33 1 47 52 30 00

Respiration France  
+33 2 51 89 36 00

Respiration Italy  
+39 03 62 63 43 1

Respiration Sweden  
+46 8 120 45 900

Respiration Switzerland  
+41 6 27 45 17 50

Respiration United Kingdom  
+44 800 1300 845

[www.philips.com/respiration](http://www.philips.com/respiration)



# Advanced Thinking

## BiPAP autoSV Advanced Conversion Guide

Respiration, autoSV, BiPAP and Bi-Flex are trademarks of Respiration, Inc. and its affiliates.

©2009 Koninklijke Philips Electronics N.V.  
All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.  
CAUTION: US federal law restricts this device to sale by or on the order of a physician.

Hoech KW 06/11/09 MCI 4102402 PN 1063372



BiPAP autoSV Advanced combines proven technologies to help make treatment and management of complicated sleep apnea cases easier and more effective.

- The combination of auto EPAP technology and an enhanced back-up rate make titration simpler.
- Enhanced algorithms and sophisticated servo ventilation continually monitor and adjust to changing patient therapy needs.
- Proven Bi-Flex technology helps to assure patient comfort for long-term compliance.

It all adds up to advanced thinking that helps you simplify the complicated.



**Advanced thinking. Simpler solutions.**  
**BiPAP autoSV Advanced.**

To convert a current BiPAP autoSV prescription to the BiPAP autoSV Advanced settings, the following table can be used:

	BiPAP autoSV (current)	BiPAP autoSV Advanced
<b>Pressure settings</b>		
	EPAP Example: EPAP = 8 cm	EPAPmin = EPAPmax Example: EPAPmin = 8 cm EPAP max = 8 cm
	IPAPmin Example: IPAPmin = 10 cm	PSmin Example: PSmin = 2 cm (Calculation: IPAPmin – EPAP)
	IPAPmax: Example: 25 cm	PSmax: Example: PSmax = 17 cm (Calculation: IPAPmax – EPAP)
		Max pressure: Example: 25 cm (Can be set anywhere between EPAP + PSmax to 30 cm)
<b>Back-up rate</b>		
	Off, Auto, 4 – 30	No difference in autoSV Advanced Off, Auto, 4 – 30
<b>Bi-Flex</b>		
	N/A	Flex = Off