

# Trilogy200 ventilator

## Caregiver competency checklist

Patient name: \_\_\_\_\_ Caregiver name: \_\_\_\_\_

Goals	Date goals met	Trainer
<b>The caregiver will be able to:</b>		
1. Understand basic indications for ventilator usage.		
2. Explain the buttons, indicators, and/or connectors on the front, rear, and side panels of Trilogy.  <b>Front panel</b> <ul style="list-style-type: none"> <li><input type="checkbox"/> Start/stop button</li> <li><input type="checkbox"/> Up, down button</li> <li><input type="checkbox"/> A/C power LED</li> <li><input type="checkbox"/> Red alarm LED</li> <li><input type="checkbox"/> Audio pause button</li> <li><input type="checkbox"/> Left, right button</li> <li><input type="checkbox"/> Backlight LED</li> <li><input type="checkbox"/> Yellow alarm LED</li> </ul> <b>Rear and side panel</b> <ul style="list-style-type: none"> <li><input type="checkbox"/> A/C power inlet</li> <li><input type="checkbox"/> Exhalation porting block</li> <li><input type="checkbox"/> Serial connector</li> <li><input type="checkbox"/> Ethernet connector</li> <li><input type="checkbox"/> O<sub>2</sub> connector</li> <li><input type="checkbox"/> Detachable battery slot</li> <li><input type="checkbox"/> Breathing circuit connection</li> <li><input type="checkbox"/> SD data card slot</li> <li><input type="checkbox"/> Remote alarm</li> <li><input type="checkbox"/> External battery connector</li> <li><input type="checkbox"/> Air inlet and filter</li> <li><input type="checkbox"/> Cord retainer</li> </ul>		
3. Connect to A/C power.		
4. Attach O <sub>2</sub> source and discuss O <sub>2</sub> safety.		
5. Assemble and connect: <ul style="list-style-type: none"> <li><input type="checkbox"/> Patient circuit</li> <li><input type="checkbox"/> Filter</li> <li><input type="checkbox"/> Exhalation device</li> <li><input type="checkbox"/> Tubing</li> <li><input type="checkbox"/> Patient connector (swirl or flex)</li> <li><input type="checkbox"/> Humidifier (if ordered)</li> <li><input type="checkbox"/> Flow sensor (if applicable)</li> </ul>		
6. Turn ventilator on and off		

Goals	Date goals met	Trainer
The caregiver will be able to:		
<p>7. Access monitoring screens.</p> <ul style="list-style-type: none"> <li><input type="checkbox"/> Describe selected viewing options <ul style="list-style-type: none"> <li><input type="checkbox"/> Detailed view off <ul style="list-style-type: none"> <li><input type="checkbox"/> Top monitor panel</li> <li><input type="checkbox"/> Middle date/time panel</li> <li><input type="checkbox"/> Bottom status panel</li> </ul> </li> <li><input type="checkbox"/> Detailed view <ul style="list-style-type: none"> <li><input type="checkbox"/> Top monitor panel</li> <li><input type="checkbox"/> Middle measured settings panel</li> <li><input type="checkbox"/> Bottom status panel</li> </ul> </li> </ul> </li> </ul>		
<p>8. Navigate the Menu screens.</p> <ul style="list-style-type: none"> <li><input type="checkbox"/> Switch to secondary settings</li> <li><input type="checkbox"/> Safely remove the SD card</li> <li><input type="checkbox"/> My Settings</li> <li><input type="checkbox"/> Options</li> <li><input type="checkbox"/> Alarm log</li> <li><input type="checkbox"/> Information</li> </ul>		
<p>9. Locate and explain alarm messages and alarm indicators (audio and visual).</p> <ul style="list-style-type: none"> <li><input type="checkbox"/> Explain the Audio Pause feature</li> <li><input type="checkbox"/> Understands what to do when an alarm occurs</li> </ul>		
<p>10. Connect to D/C power and explain power source indicators.</p> <ul style="list-style-type: none"> <li><input type="checkbox"/> External battery</li> <li><input type="checkbox"/> Detachable battery pack</li> <li><input type="checkbox"/> Internal battery</li> </ul>		
<p>11. Review care, cleaning, and maintenance of ventilator and accessories.</p>		
<p>12. Review emergency procedures</p>		

Caregiver signature \_\_\_\_\_ Clinician signature \_\_\_\_\_

Respironics and Trilogy are trademarks of Respironics, Inc. and its affiliates.



© 2010 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: U.S. Federal law restricts these devices to sale by or on the order of a physician.

Philips Healthcare is part of Royal Philips Electronics

Europe, Africa, Middle East:  
+33 1 47 52 30 00  
Asia Pacific:  
+852 3194 2280

[www.philips.com/respironics](http://www.philips.com/respironics)

Philips Respironics  
1010 Murry Ridge Lane  
Murrysville, PA 15668  
+1 724 387 4000  
+1 800 345 6443