

## Configuring the HeartStart MRx for AHA/ERC 2005 Guidelines

This document describes how to configure your HeartStart MRx to support the recommendations of the American Heart Association's and European Resuscitation Council's 2005 resuscitation guidelines. With Product Version B.05, you can configure the HeartStart MRx to the recommended protocol of 1 shock then 2 minutes of CPR. Additionally, Product Version B.06 modifies the AED CPR prompt and supports the recommendation of immediate CPR after a "No Shock Advised" decision.

Modify the following AED Configuration settings (Refer to the *HeartStart MRx Instructions for Use* for information on modifying configuration settings):

Parameter	Set to:
Shock Series	1
CPR Timer	120
No Shock Advised Action	120
CPR Prompt	Short

