


Basic Defibrillator Cabinet

(Philips P/N 989803136531)

BATTERY INSTALLATION INSTRUCTIONS

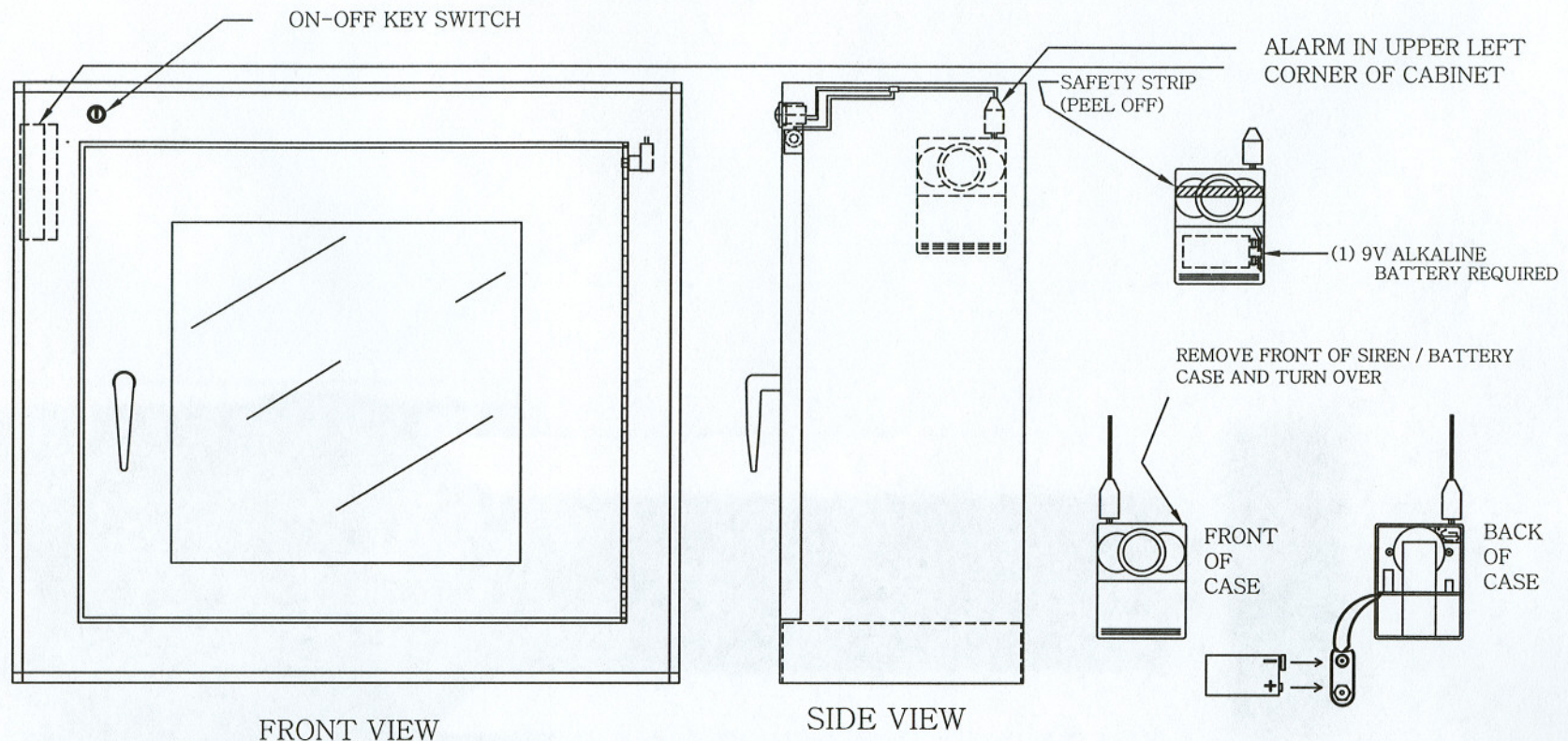

NOTE: Remove shipping tape strip from front of siren / battery case and discard.

1. Insert key provided into on-off switch and turn counter-clockwise to 12 o'clock position to ensure alarm is disarmed.
2. Open cabinet and locate siren / battery case.
3. Grip sides of siren / battery case, and pull front portion away from case backing holder. (Backing holder will remain fixed to inside of cabinet.)
4. Turn case over and note battery cavity and connectors for (1) 9V alkaline battery. (If replacing a spent battery, disconnect it from connectors and discard appropriately.)
5. Connect a fresh 9V alkaline battery to connectors, place battery into case cavity, and snap all back into case backing holder.
6. Close door and arm cabinet alarm using keys provided. (Turn key to 3 o'clock position to set alarm to the on position.)