

# System One sleep therapy platform modem information

## Modems and EncoreAnywhere

- The transmission status of wired and wireless modems can be Active or Inactive.
- All modems are shipped as Inactive. An Inactive modem will not be listed in the EncoreAnywhere modem list.
- A modem must be in Active status in order for it to communicate with EncoreAnywhere.
- Transmission fees only occur when a modem is in Active status.

The following instructions identify how to activate and deactivate a wired or wireless modem.

## Activating a modem


1. All modems are shipped as Inactive. An Inactive modem will not be listed in the EncoreAnywhere modem list.
2. A modem is automatically activated when the Encore ID is entered into a patient prescription record with the System One device serial number.
3. Once Active, the status and information about a modem is available to the Modem Manager. If the Modem Manager accesses the “Modem Administration” tab and clicks on a modem, the “Modem List” screen will display the following information (refer to illustration A):
  - Modem state (either Active or a **Deactivate** link)
  - Summary of modem (Assigned to, Clinician, Next call, etc.)
  - History of modem usage

*Note: A user must be assigned the role of Modem Manager in order to view the “Modem Administration” tab.*

## Modem part numbers and transmission plan information

Modem options		Call frequency	Transmission fees
100200W	System One wired flow modem	Daily calls Weekly waveforms	\$3.50 / month
100200C	System One wireless flow modem	Daily calls Weekly waveforms	\$7.50 / month
100200D	System One wired data modem	Month 1: Daily calls Month 2 and 3: Weekly calls Month 4-12: Bi-weekly calls	N/A


## Illustration A

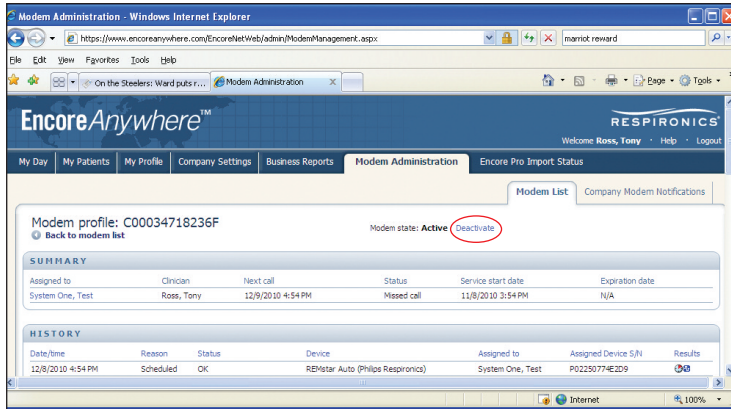

## Deactivating a modem

1. The Modem Manager will access the "Modem List" available in the "Modem Administration" tab.
2. The top of the screen will show "Modem state: **Active**" and the word "**Deactivate**" will be a link (refer to illustration A).
3. By clicking on the "**Deactivate**" link, the modem is now Inactive.
4. The screen will show "Modem state: Inactive" and the word "**Activate**" will now be a link (refer to Illustration B).

## Illustration B

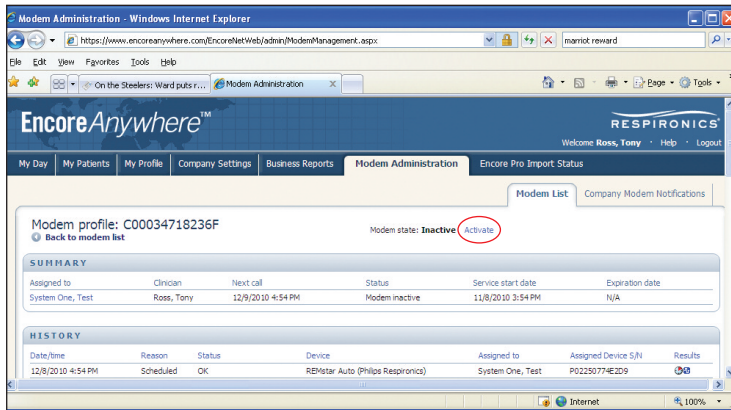

Encore, EncoreAnywhere, System One, and Resprionics are trademarks of Resprionics, Inc. and its affiliates. All rights reserved.



©2011 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.

SB 02/07/11 MCI 4104035

Philips Healthcare is part of Royal Philips Electronics

Resprionics Asia Pacific  
+65 6298 1088  
Resprionics Australia  
+61 (2) 9666 4444  
Resprionics Europe, Middle East, Africa  
+33 1 47 52 30 00  
Resprionics United Kingdom  
+44 800 1300 845

Philips Resprionics  
1010 Murry Ridge Lane  
Murrysville, PA 15668

Customer Service  
+1 724 387 4000  
+1 800 345 6443 (toll free, US only)  
www.philips.com/resprionics