





<b>PHILIPS HEALTHCARE</b>	<b>Product Recycling Passport</b>	Page 1 of 6
-------------------------------	-----------------------------------	-------------

<b>Product name:</b>	<b>Brilliance iCT Bariatric Patient Support</b>	
<b>Identification code(s)</b>	<b>4535 674 80191</b>	
<b>Total weight (in Kg)</b>	<b>395</b>	
<b>Producer/ Manufacturer</b>	Name company:	Philips Medical Systems
	Address:	595 Miner Road Highland Heights, Ohio
	Zip code:	44143
	Country:	United States of America

Printed copies are uncontrolled

<b>Recycle Info</b>	<b>Items:</b>	<b>Location</b>
<b>Special attention</b> 	Removal of Motor: Couch will drop when motor is removed.	Figure 1 (1)
	Green Safety Block: When removed, couch may collapse.	Figure 1 (3)
	Giant Spring: Potential energy hazard.	Figure 1 (4)
<b>Fluids / Gases</b> 	<b>Items:</b>	<b>Location</b>
	Pressure Springs: Must drill 2mm hole to relieve pressure before disposing of springs.	Figure 3 (5)
<b>Batteries</b> 	<b>Type:</b>	<b>Location</b>
<b>To be Removed</b>		
<b>Hazardous</b> 	<b>Substances:</b>	<b>Location</b>
	Printed Circuit Boards	Figure 1 & 2 (2)
<b>To be Removed</b>		

Show locations of materials mentioned on the previous sheet. (Picture information)

Printed copies are uncontrolled



Figure 1: Right Side Couch




Printed copies are uncontrolled




**Figure 2: Left Side of Couch**




Printed copies are uncontrolled

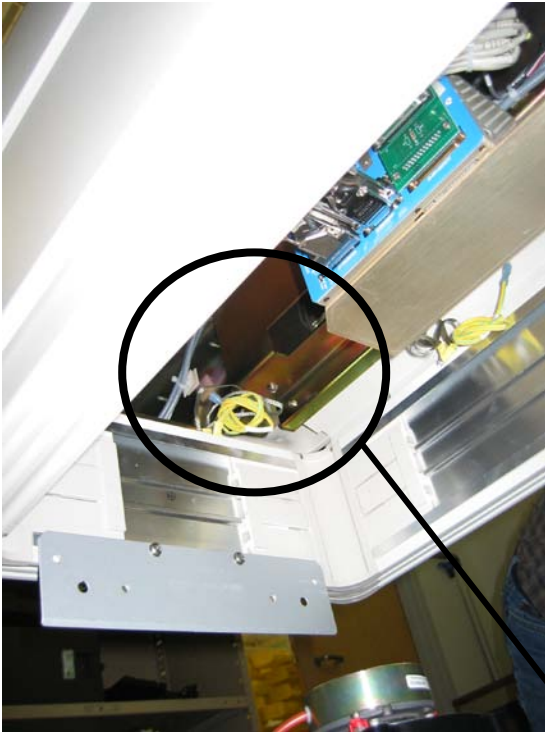

**Figure 3: Underneath Rear of Sub-frame**

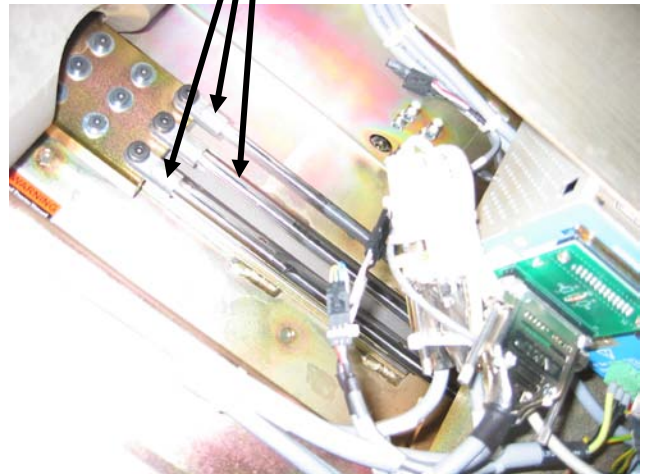

Printed copies are uncontrolled

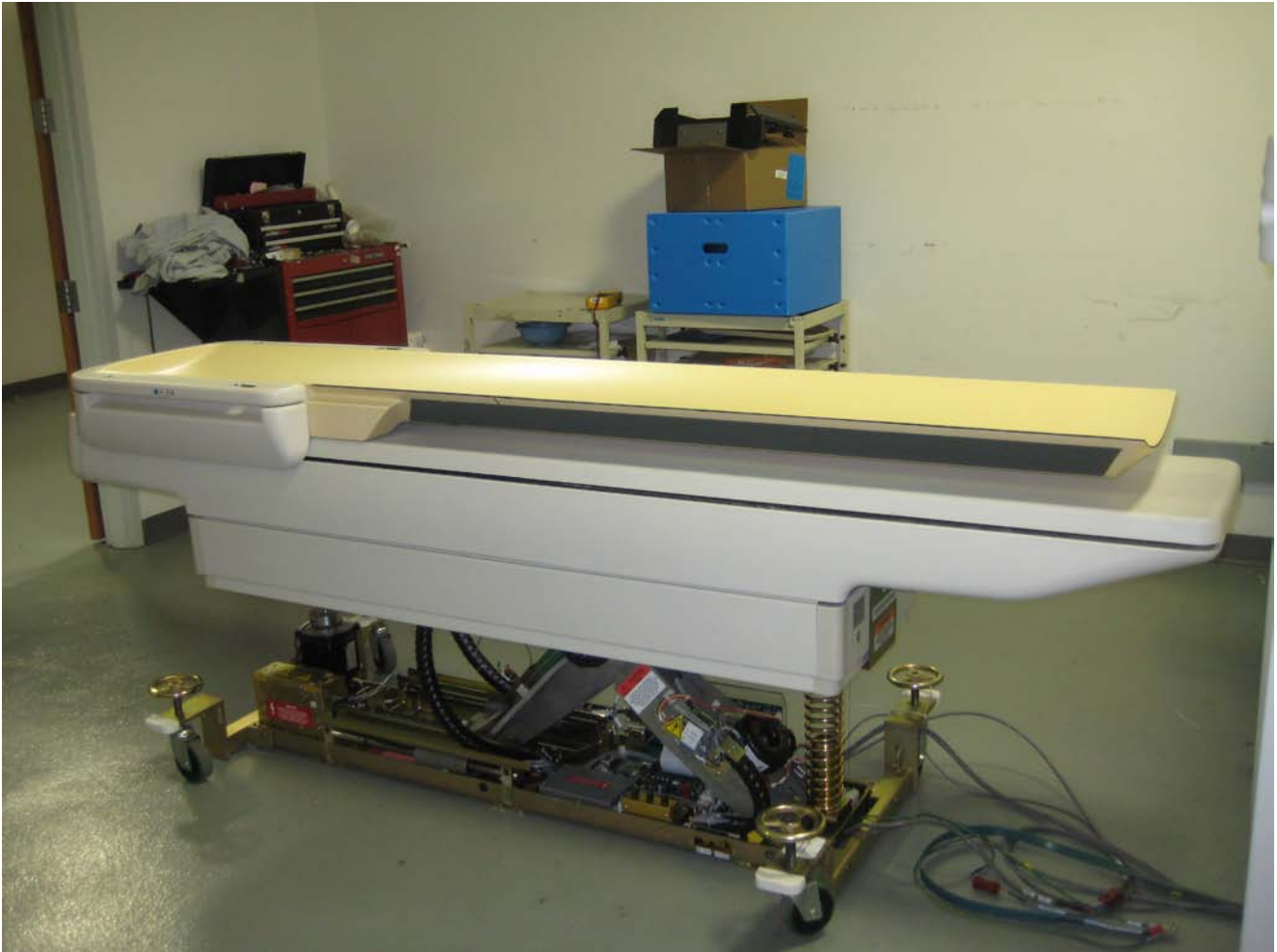

**Figure 4: Full view of Patient Support**

Title: Recycle Passport Brilliance iCT Bariatric Patient Support  
Recycling Passport Number: 4535 675 34241 Rev: B

### Recycling Passport Revision History

Rev	Description	Date
A	CO 2019848-Release Status A	24 February 2009
B	CO 2020759 - Removed superfluous header, added whole patient support picture	09 June 2009

Printed copies are uncontrolled