




Philips Medical Systems / Product Recycling Passport

Product name:	MP5 Patient Monitor	
Identification code	M8105A	
Total weight (in Kg)	4.3 kg	
Producer /Manufacturer	Name company:	Philips Medizin Systeme Boeblingen
	Address:	Hewlett-Packard Strasse 2
	Zip code:	Boeblingen 71034
	Country:	Germany

Recycle info	Substances:	Location
Hazardous  To be removed		
Batteries  To be removed	Type:	Location
	Lithium Ion battery 1x	Figure 2 (item 1)
Special attention 	Item	Location
	TFT-Display	Figure 12 (item 2)
	Item	Location

Material content (global)	Weight in Kg	Materials content (continue)	Weight in Kg
Aluminium	0.1	Battery	0.5
Steel	0.9		
Plastics etc	1.9	All other material types	0.2
Printed Circuits Boards	0.7		

Locations as mentioned in the Passport (Pictures information)

1 Unscrew the two screws securing the handle/bedhanger and pull off the handle/bedhanger.



Figure 1



2 Open the battery compartment, push up the battery compartment latch and remove the battery from the battery compartment by pulling on the tab.

1



Figure 2



3 Open the recorder, remove the recorder paper and locate the two screws inside the recorder.

Figure 3



Figure 4

4 Unscrew the two screws inside the internal recorder and pull out the recorder. Note that these screws will not come out completely but remain in the recorder housing.

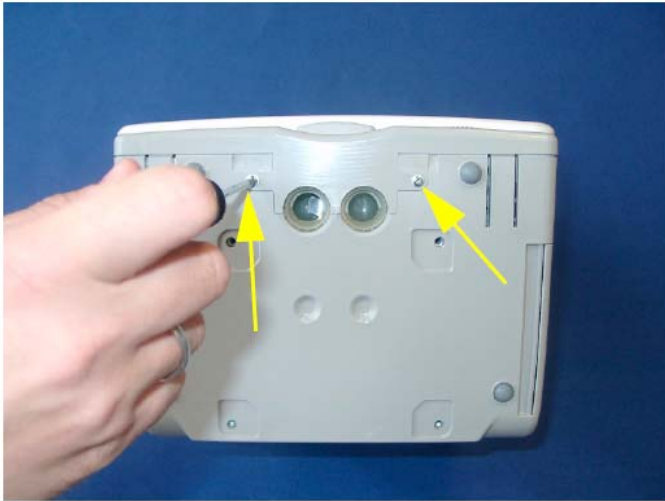


Figure 5

5 Remove the two screws securing the quick mount cover.

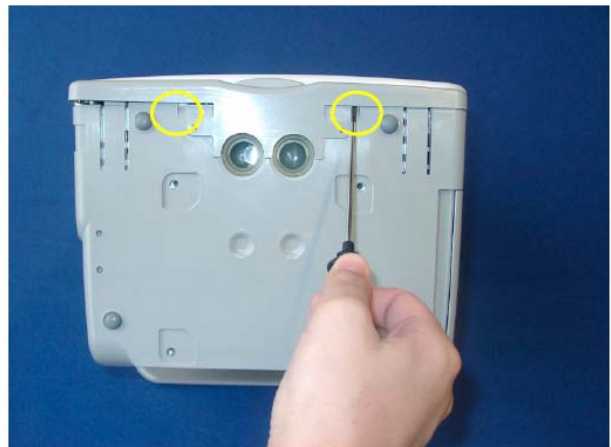


Figure 6

6 Insert a screwdriver into the slots at the bottom of the quick mount cover and push the cover off on each side. Then pull the quick mount cover off the monitor.



Figure 7

7 Remove the cover branding clip.



Figure 8

8 Release the white pins at the top and the bottom of the monitor by turning them counterclockwise with a flat-tipped screwdriver and then pull them out as shown below.



Figure 9

9 Insert a screwdriver into the outer slots (on the outside of the latches of the white housing) on both sides to loosen the rear cover and then lift the back of the monitor off.

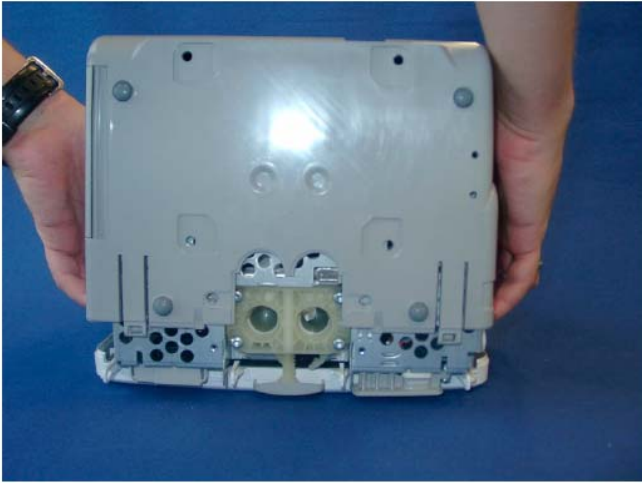


Figure 10

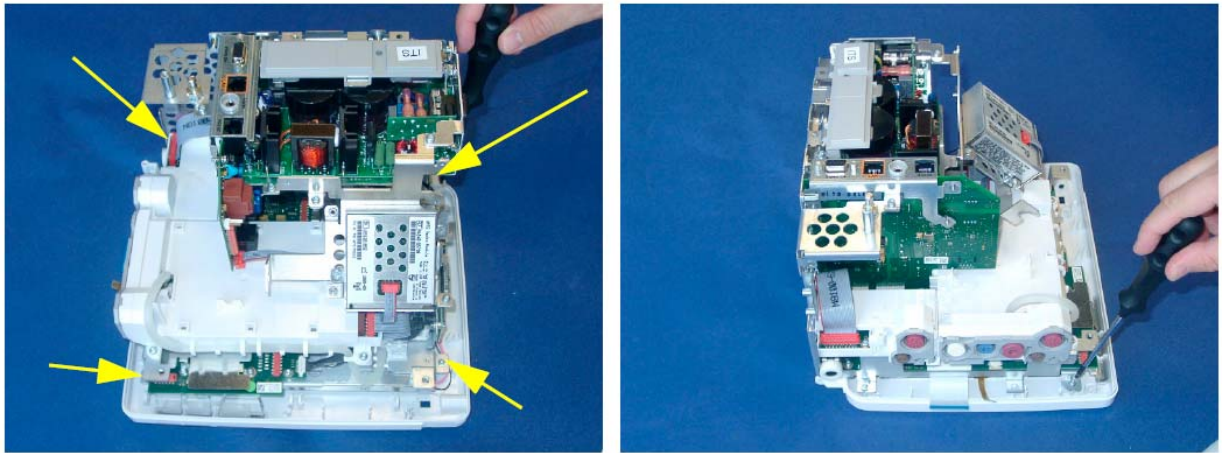


Figure 11

10 Unscrew the four screws securing the touch assembly.



2



Figure 12