

SureSigns VS2i/VS2+ Recycling Passport

Notice **UNCONTROLLED IN PRINTED FORM UNLESS STAMPED IN RED**

Document Ownership The document owner is listed below:





Function	Title
Regulatory System Department	Regulatory Engineer

Revision Summary The current document revision is described below:

Rev.	Change Order #	Change Summary
A	See eDMR	As Issued

SureSigns VS2i/VS2+ Recycling Passport

Product name:		SureSigns VS2i/VS2+
Identification code(s)		(Model) SureSigns VS2i/VS2+ (Reference num) 863275, 863276, 863277/863278, 863279
Total weight (in Kg)		<2.8 kg (including battery)
Producer/ Manufacturer	Name company:	Philips Medical Systems
	Address:	3000 Minuteman Road, Andover MA
	Zip code:	01810
	Country:	USA

Recycle Info	Items:	Location
Special attention 	Warning: Before you attempt to open or disassemble the monitor, disconnect the AC power from the monitor and remove the battery. Remove any cables from the measurement modules, and the nurse call and network connections.	Figure 1, 2, 3, 4
 Fluids / Gases	Items:	Location
	N/A	
 Batteries To be Removed	Type:	Location
	Lithium Ion battery	Figure 4
 Hazardous To be Removed	Substances:	Location
	Tin lead solder on the PCB boards	Figure 14

SureSigns VS2i/VS2+ Recycling Passport

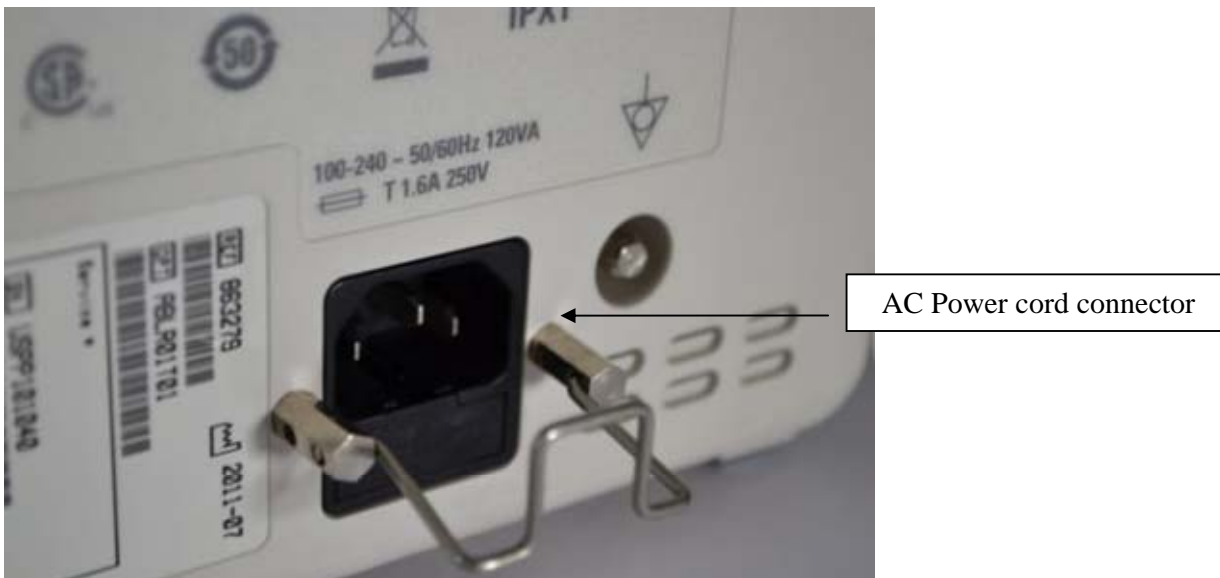


Figure 1 – AC Power - disconnect

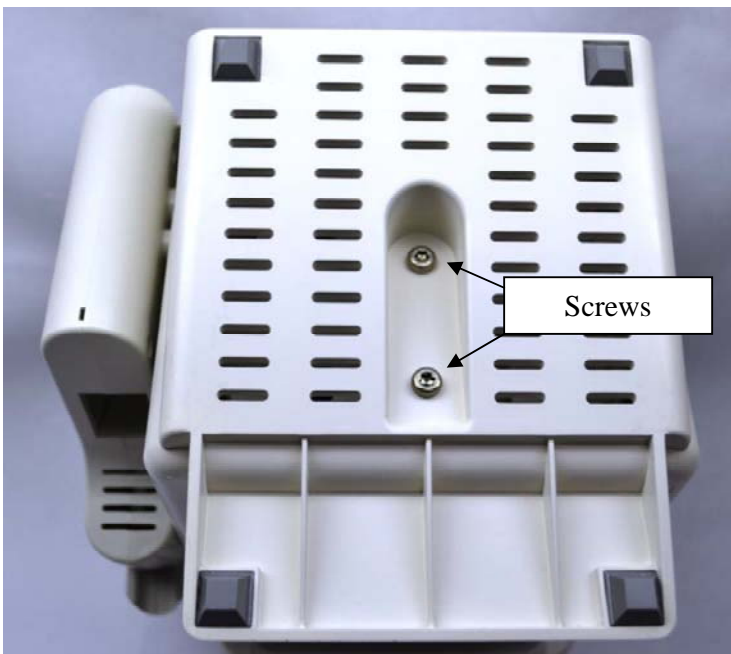


Figure 2 - If your monitor has a tabletop base, remove it by removing the two Torx T20 screws from the bottom of the base.

SureSigns VS2i/VS2+ Recycling Passport

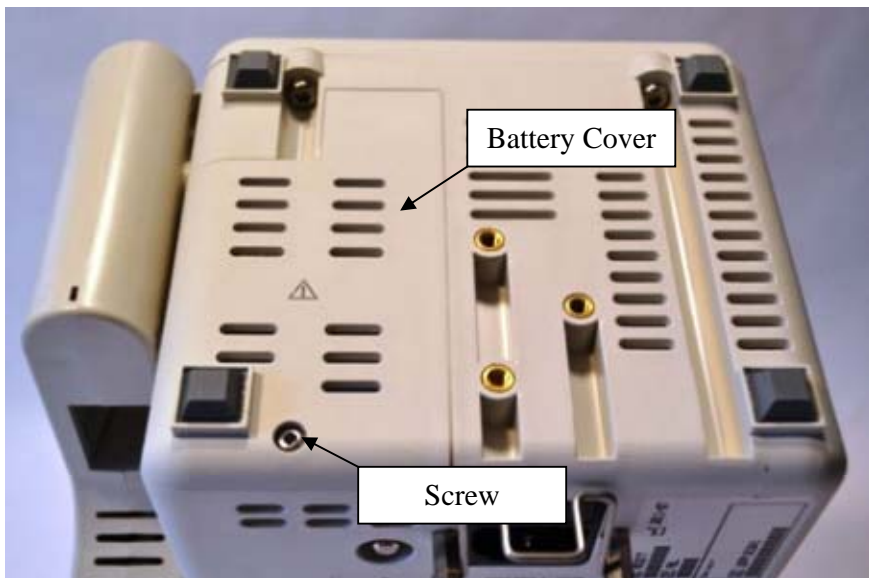


Figure 3 - Remove the Torx T10 screw from the battery cover on the bottom of the monitor.



Figure 4 - Disconnect the battery cable from the connector and remove the battery from the monitor

SureSigns VS2i/VS2+ Recycling Passport

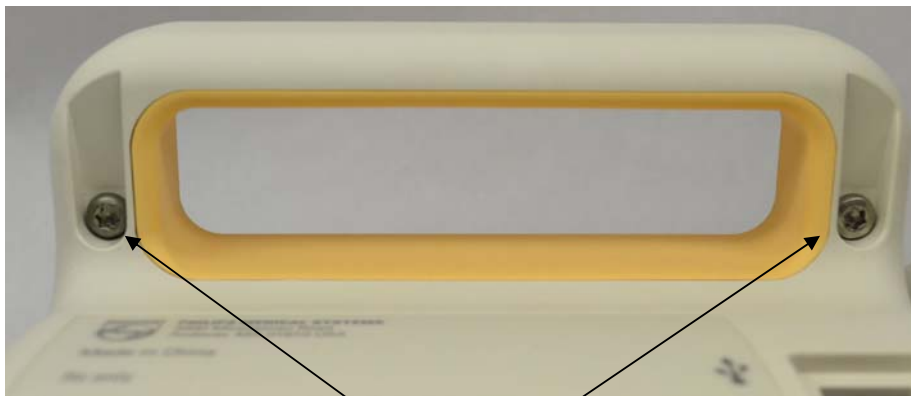


Figure 5

Screws

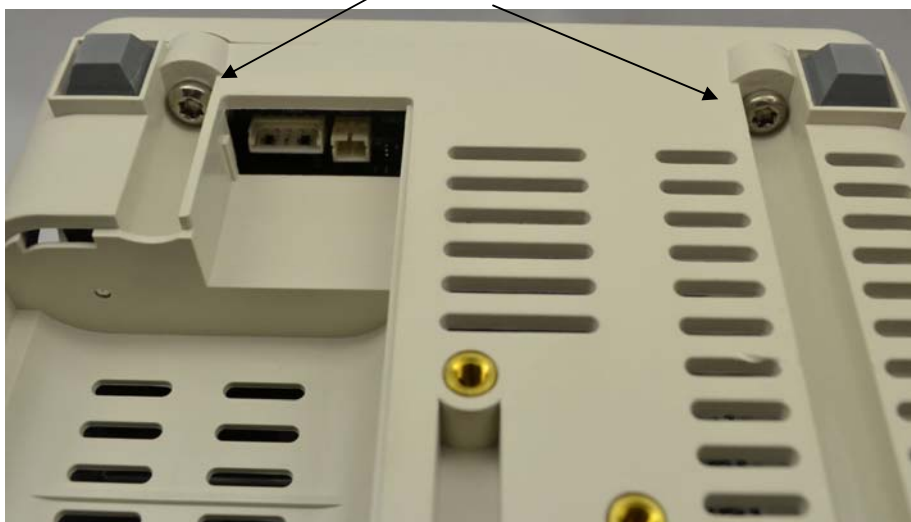


Figure 6

SureSigns VS2i/VS2+ Recycling Passport

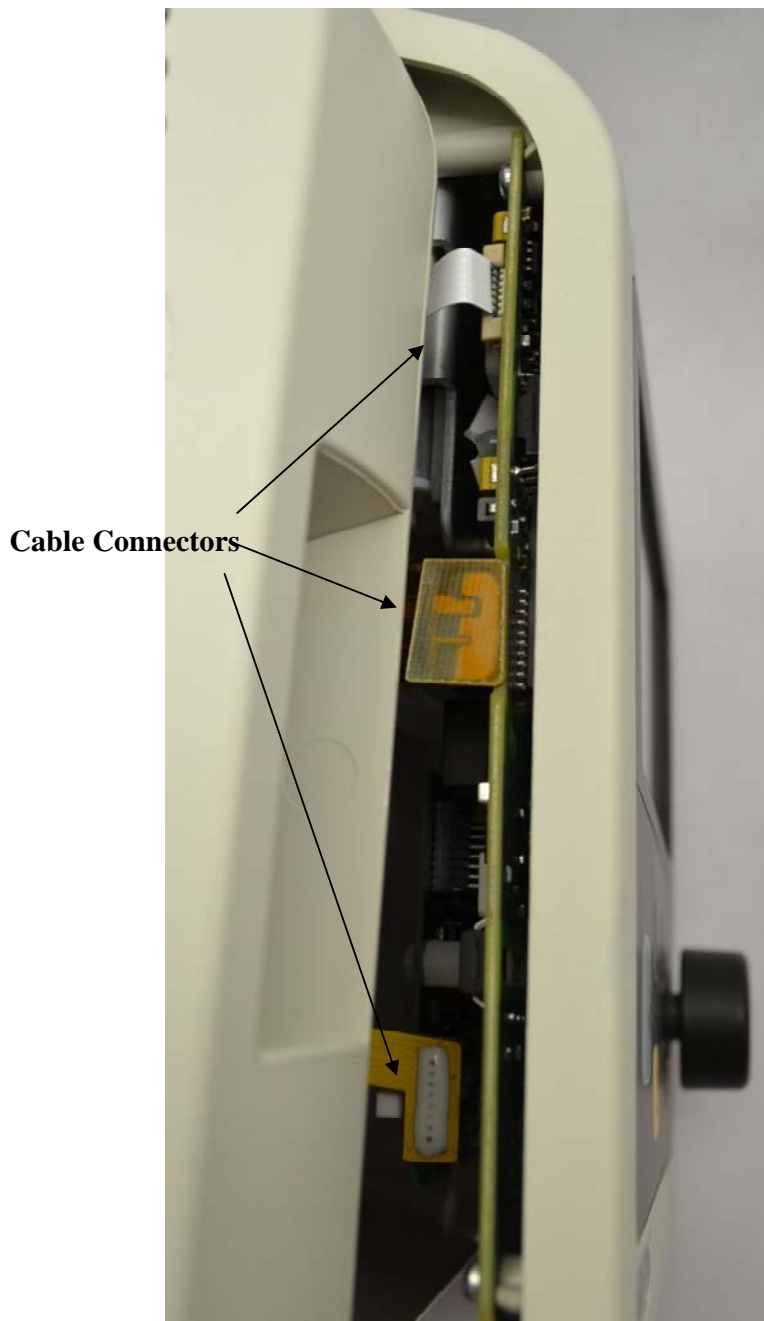


Figure 7

SureSigns VS2i/VS2+ Recycling Passport



Figure 10

PCAs