



PHILIPS
RESPIRONICS

Amara View

Under the nose

Full face mask

High marks for performance

No marks on the bridge of the nose

Full marks for performance

Amara View is the smallest and lightest of all leading full-face masks*.

Its innovative and unique under-the-nose cushion design prevents red marks, discomfort or irritation on the bridge of the nose and delivers comfort and performance for your patient.

Amara View also has the widest field of vision of any leading full face mask*. Perfect if your patient wears glasses or likes to read, watch TV, or use a tablet before sleeping.

* Internal testing: data on file

Ordering information

Mask with headgear (Global version)

	Part number
Small mask	1090602
Medium mask	1090603
Large mask	1090604

Mask with headgear (International version)

Small mask	1090662
Medium mask	1090663
Large mask	1090664

Sizing gauge

Sizing gauge	1090699
--------------	---------



Achieving the right fit is easy

Just follow these instructions.

Unlike most full face masks, the cushion size for Amara View is determined by **measuring the depth of the nose** using the simple sizing gauge. Just place the sizing gauge under your patient's nose **without pushing against the cheeks** and the tip of the nose will indicate whether they need a small, medium or large cushion size. **If your patient measures close to the threshold always choose the smaller size.**



1 Loosen the headgear. Twist the headgear clips to disconnect.



2 Choose the cushion size that covers the mouth and sits at the nose base.



3 Gently holding the mask against the face, pull the headgear over the head.



4 Ensure the nose is over **(not inside)** the cushion nostril opening.



5 Press the headgear clips into place to connect.



6 Pull back the headgear tabs to evenly adjust the top and bottom straps.



7 Adjust the crown strap so the headgear sits loosely and comfortably.



8 Connect to a working CPAP machine to make final adjustments.

Learn more

For other helpful tips please view:

[sleepapnea.com/
products/masks/
amara-view/
resources/](https://sleepapnea.com/products/masks/amara-view/resources/)

Innovative under-the-nose design

prevents red marks on the bridge of the nose and gives your patient the widest field of vision of any of the leading full-face masks*

Easy to use

with one modular frame for all cushion sizes

Small cushion	1090692
Medium cushion	1090693
Large cushion	1090694

Quick release clips

for easy on and off



Headgear clips x 2 1090698

© 2015 Koninklijke Philips N.V. All rights reserved. Specifications are subject to change without notice. Trademarks are the property of Koninklijke Philips N.V. (Royal Philips) or their respective owners.

* Internal testing; data on file

www.philips.com



Soft, comfortable fabric straps

Large headgear	1090696
Standard headgear	1090697

Quick disconnect

Quick release tube 1090689