



See what you've been missing with the world's only digital PET/CT

The Vereos PET/CT takes a leap forward by giving you improved contrast and superb SUV quantitation. It positions you to go beyond the current limitations in PET/CT imaging and improve patient care.

Analog PET scan*

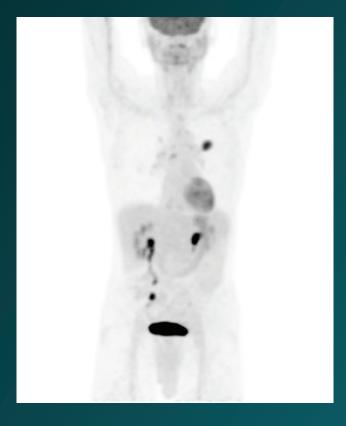

Digital PET scan

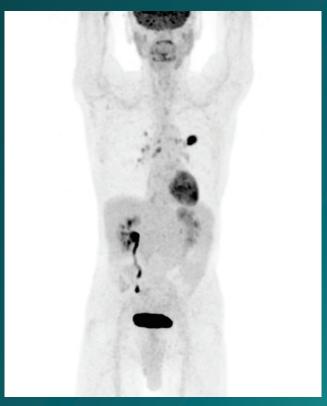

Digital PET sensitivity gain up to ~2x greater than analog.

Sample images acquired in a clinical study of the Vereos PET/CT system at University Hospitals Cleveland. Investigational device limited by law to investigational use.


^{*} GEMINI TF 16

See more with a faster Time-of-Flight


Seeing further requires faster Time-of-Flight (TOF) performance. Vereos PET/CT has it, providing you greater sensitivity gain than analog systems.* This data, which is a result of the improved sensitivity gain, enables you to achieve improved image quality. Our faster TOF performance can also help you manage examination time, giving you quick exams and fast results.

Analog PET scan*



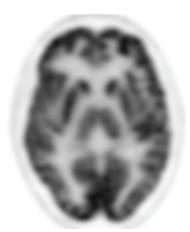

Digital PET scan

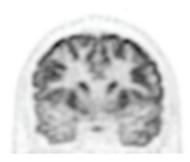



Digital PET volumetric resolution up to ~ 2x greater than analog.*

Digital Photon Counting

At the core of the Vereos revolution is our proprietary Digital Photon Counting (DPC) technology.
Compared to analog*, our DPC technology improves performance in three key parameters:


- ≈ 2x volumetric resolution
- ≈ 2x sensitivity gain
- ≈ 2x quantitative accuracy

Sample images acquired in a clinical study of the Vereos PET/CT system at University Hospitals Cleveland. Investigational device limited by law to investigational use.

^{*} GEMINI TF 16

See your clinical role expand

Philips supports your clinical excellence and your role as a partner with other clinicians in improving patient care. Vereos PET/CT can help you take your diagnostic confidence to the next level.

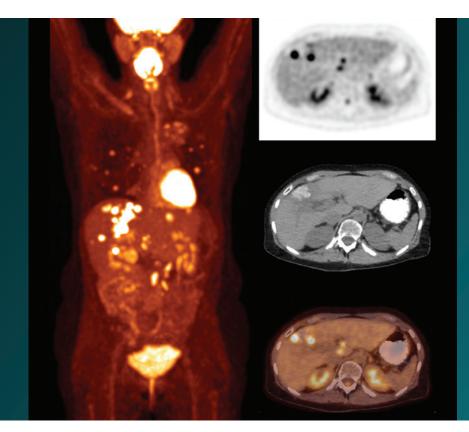

Digital PET

Accurate quantitation using Vereos PET/CT can enhance your position as a preferred site for clinical referrals by offering:

- Enhanced collaboration
- · Enhanced treatment planning
- · Increased diagnostic confidence

Vereos PET/CT can also help you improve workflow efficiency and patient management through:

- Fast workflows
- Fast scans using shortest bore in the industry
- High image quality at low dose (with iDose⁴ Premium Package)


Vereos* specifications

Number of detectors	23,040
System spatial resolution	4.1 mm
Effective system sensitivity	22.0 kcps/MBq
Effective system sensitivity/cm	>1,304 cps/MBq
Effective peak NECR	650 kcps @ 50 kBq/mL
Maximum trues	>675 kcps
System timing resolution	325 ps
Quantitative accuracy	+/- 5%

^{*} Preliminary performance data subject to change.

Sample images acquired in a clinical study of the Vereos PET/CT system at University Hospitals Cleveland. Investigational device limited by law to investigational use.

Philips **exclusive** Digital Photon Counting (DPC) technology

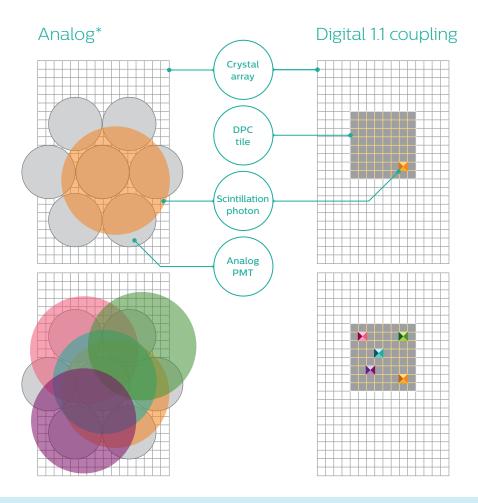

Our revolutionary digital breakthrough in PET imaging

DPC converts scintillation light directly to a digital signal, with zero analog noise.

DPC reduces the traditional tradeoffs between sensitivity gains and resolution gains.

DPC compared to analog* enables:

- Faster TOF
- · Faster timing resolution
- Faster post-processing and image fusion
- Faster throughput and workflow

Resolution gains

Our DPC technology is the only true digital detection technology available in the industry. The system uses 1:1 coupling of scintillators to digital detectors. That means spatial, timing, and energy resolution is improved throughout the entire multidimensional field of view, for improved volumetric resolution.

Sensitivity gains

Our DPC technology converts light directly to a digital signal. Because there is virtually no noise introduced in this conversion, the system improves the signal-to-noise ratio which enables improved sensitivity. Coupled with fast timing resolution, image contrast improves leading to potentially enhanced lesion detectability.

Quantitative gains

Our DPC technology provides a linear count rate performance, which makes the peak count rate higher. An increased rate plus fast timing resolution capabilities gives an improvement in quantitative accuracy. The imaging chain maintains digital data integrity through the entire imaging chain. Add to that our reconstruction, and you get outstanding images with fast post-processing.

≈ 2x volumetric resolution

≈ 2x sensitivity gain

≈ 2x quantitative accuracy

See how your patients benefit

Your patients can benefit from the revolutionary Vereos PET/CT system.


- iPatient: iPatient offers personalized patient-centric workflow built for iterative reconstruction techniques and high image quality at low dose levels to drive confidence and consistency 24/7.
- iDose⁴ Premium Package: iDose⁴ improves image quality* through artifact prevention and increased spatial resolution at low dose. O-MAR reduces artifacts caused by large orthopedic implants. Together they produce high image quality with reduced artifacts.
- Fast scanning: Patients spend less time in scanner.
- Short bore length: Less time in bore may enhance patient comfort.
- Gentle lighting: An improved industrial design contributes to your patients' peace of mind.



Without O-MAR



With O-MAR

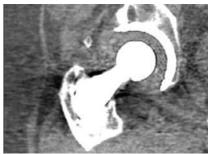





Superb diagnostic CT detail, fast scanning, low radiation dose.



Patient Patient-centric workflow

iPatient ExamCards provide easy PET/CT acquisition setup. Customized ExamCards may be created to suit your preference.

* Improved image quality is defined by improvements in spatial resolution and/or noise reduction as measured in phantom studies.

Go beyond in your practice

Expand your leadership as far as your vision takes you in molecular imaging and personalized medicine.



The images and descriptions contained herein provide technical specifications and optional features which may not be included with the standard system configuration. Contact your local Philips Representative for complete specific system details.

Some or all of the products, features, and accessories shown or described herein may not be available in your market. Please contact your local Philips Representative for availability.

