

## Installation Guide

### 8" GCX Channel for Modular Services or Amico Vertical Rail

*The purpose of this guide is to describe mounting of the 8" GCX Channel to Amico Vertical Rail*




**Warning:** GCX makes no claim regarding the capacity of the vertical rail to carry a load. Refer to Amico / Modular Services Installation Guides and weight limits for Tracks and Headwalls.

#### Parts Reference

*The following parts and hardware are included with this installation kit (hardware not shown):*

Item #	Description	Qty
1	8" Channel with Dovetail Runner	1
2	Dovetail Runner	1
3	10-32 X 3/8 Flat Head Machine Screw (FHMS)	3

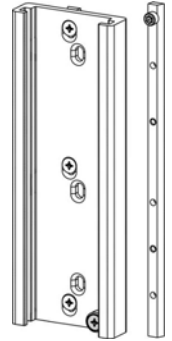

**Tools Required:** Phillips screwdriver

#### Installing the 8" Channel onto Modular Services or Amico Vertical Rail

1. Position 8" channel with attached Dovetail Runner into Vertical Rail (below left).
2. Guide separate Dovetail Runner into Vertical Rail as shown (below center).
3. Using a Phillips screwdriver, install three (3) 10-32 X 3/8 FHMS through 8" Channel and into Dovetail Runner. Tighten all six (6) 10-32 X 3/8 FHMS to secure 8" Channel to Amico Vertical Rail (below right).

