


# Brilliance CT 64 channel


## Preferred Room Layout

The layout shown below is based upon a typical equipment configuration and should be considered as a general design guideline. Site conditions, application requirements, customer preferences, and/or equipment configuration may significantly impact suite design and equipment layout. It is recommended to request site-specific drawings from a Philips representative early in the design process.

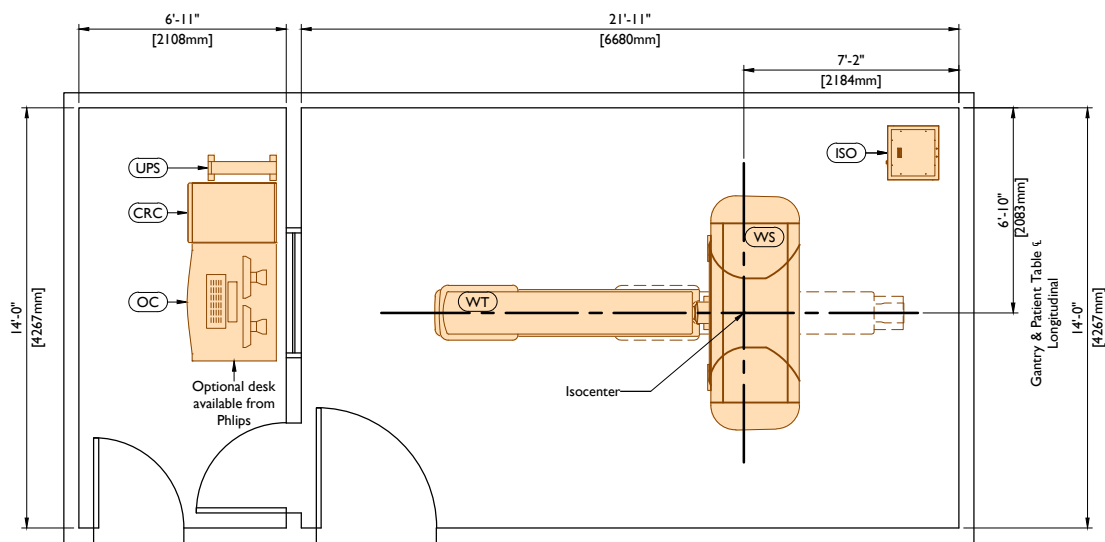

## Equipment Layout

Recommended Ceiling Height: 9'-0" (2743mm)  
Minimum Ceiling Height: 8'-0" (2440mm)

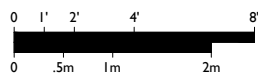

Equipment Legend				
A Furnished and installed by Philips B Furnished by customer/contractor and installed by customer/contractor C Installed by customer/contractor D Furnished by Philips and installed by contractor E Existing F Future G Optional item furnished by Philips				
Equipment Designation				
		Description	Weight lbs [kg]	Heat Load Btu/hr [W]
A	WS	Brilliance CT Scanner Gantry	4280 [1941]	18000 [5275]
A	WT	Patient Table	850 [386]	--
A	OC	Operator's Console	45 [20]	3572 [1046]
A	CRC	CIRS Recon/COM Cabinet	269 [122]	--
G	ISO	Teal MCT 100/480 Isotran Plus Power Unit	697 [316]	2350 [689]
G	UPS	3kVA Console UPS	155 [71]	850 [250]

## Environmental Requirements for General Equipment Locations

Operating temperature range within the CT Exam Room is 64° F (18° C) to 75° F (24° C) [ideal stable room temperature setting: 72° F (22° C)] at 35% to 70% relative humidity (non-condensing). Operating temperature change per hour throughout the CT Exam Room must not exceed 5° F (3° C).

Operating temperature range throughout the rest of the CT Suite is 59°-75° F (15°-24° C) [ideal stable room temperature setting: 72° F (22° C)] at 35% to 70% relative humidity (non-condensing). Operating temperature change per hour throughout the CT Suite must not exceed 5° F (3° C).

The above conditions must be maintained at all times, including overnight, weekends, and holidays. Heat output in one area of the CT Suite must not affect temperature and humidity in other areas. It is strongly recommended that any definable areas within the suite, i.e. equipment closets, control areas, etc. (if applicable), be individually environmentally controlled as required to meet ambient ranges specified.

## Power Requirements

Supply Configuration: 3 phase, 3 wire power, Earth (with Teal Isotran, Philips approved power device)

Nominal Line Voltage: 480 VAC (+/- 10%), 50/60 Hz (+/- 3 Hz)

Branch Power Capacity: 112.5 kVA min. (150 kVA preferred)

## Remote Service Diagnostics

Medical Imaging equipment to be installed by Philips is equipped with a service diagnostic feature which allows for remote and on-site service diagnostics. To establish this feature, a RJ45 type Ethernet 10/100/1000 Mbit network connector must be installed. Access to the customer's network via their remote access server is needed for Remote Service Network (RSN) connectivity. All costs associated with this feature are the responsibility of the customer.



© Koninklijke Philips N.V. 2014.  
All rights reserved.

Reproduction in whole or in part is  
prohibited without prior written consent of  
the copyright holder.

Rev. 14.00