



© 2018 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.

www.philips.com/respironics

RRDPGH TM 12/1/17 MCI 4108048 PN 1136797
1010 Murry Ridge Lane, Murrysville, PA 15668
800-345-6443 • 724-387-4000

A close-up photograph of a man and a woman sleeping in a bed. The man, in the foreground, is wearing a white CPAP mask over his nose and mouth. He has a beard and is wearing a blue t-shirt. The woman is behind him, also sleeping. The bed has white pillows and a white sheet. The lighting is soft and warm, suggesting a bedroom at night.

PHILIPS

RESPIRONICS

DreamWear

This mask
changes everything

Like wearing nothing at all

Patients can sleep how they want with the freedom and comfort of DreamWear

The Philips Respironics DreamWear frame supports full face, nasal and gel pillows cushions to meet patients' needs. DreamWear's remarkable design directs airflow through the frame so patients can sleep comfortably, and the tubing connection located at the top of the head allows patients to sleep in any position they desire—on their stomach, side, or back.



Users told us that DreamWear gave them more freedom to move, had a better fit than their prescribed mask, and comes closest to making them feel like they don't have anything on their face during therapy.*

* DreamWear Under The Nose: Data analysis at 30 days of use of 2015 Philips sponsored patient preference trial (n=98). Prescribed masks include ResMed Swift FX, ResMed Mirage FX, Philips Wisp, and ResMed P10. DreamWear Gel Pillows: Data analysis at 30 days of use of 2017 Philips sponsored patient preference trial (n=80). Prescribed masks include Resmed Swift FX and P10, F&P Opus and Pilairo, and Philips Nuance and Nuance Pro. Data analysis at 30 days of use of 2017 Philips sponsored patient preference trial (n=80). Prescribed masks include Resmed Swift FX and P10, F&P Opus and Pilairo, and Philips Nuance and Nuance Pro. DreamWear Full Face: Data analysis of Aug 2017 Patient Preference trial where n = 85 and prescribed masks include Resmed Airfit F10 and F20, Respironics Amara View and F&P Simplus.



DreamWear

Soft silicone frame

Light, flexible material designed for comfort.

Open field of vision

Wear glasses, read, and watch TV.

No red marks*

Leaves no marks on the bridge of the nose.

Innovative

Under-the-nose design.

*Mask does not directly contact the bridge of the nose or nostrils.





**Top-of-head
tube connection**

Provides freedom of
movement throughout
the night.

In-frame airflow

Unique design allows
airflow through the
frame to help patients
sleep comfortably.



Interchangeable cushions

Easily switch between full face, nasal and gel pillows cushions.

One mask

fits all three cushion types

Simply switch cushions without changing masks

Choosing the right mask for your patients is one of the most difficult jobs you face. The non-stop trial and error involving the masks, cushions, and headgear needed to fit your patients becomes tedious and time-consuming.

On top of that, you must manage patients in their initial acceptance period who struggle for a variety of reasons including poor fit, leaking, nasal congestion and others.

Now, DreamWear's design allows you to quickly and easily switch between full face, nasal and gel pillows cushions without changing masks. With fewer mask decisions to make, you can focus on what's really important: your patients.

Benefits

- Can accommodate full face, nasal and gel pillows cushions
- Allows for easy switching between cushion types
- May reduce the number of masks needed to set up a patient
- Helps streamline the amount of inventory to be managed
- Can increase efficiency by requiring education on only one mask



Clinicians rated the DreamWear mask with nasal and gel pillows cushions as taking less time to change than another pillows mask and nasal mask.*



DreamWear **patient fitting guide**

Follow the instructions to get started fitting your full face, nasal or gel pillows mask

Full face

Using the full face sizing gauge, hold horizontally under the nose and select the cushion size based on where the outer edge of your nostrils and tip of your nose contact the gauge. **Note: Four cushion sizes (small, medium, large, medium-wide) are available.**



The cushion opening will be directly under the nostrils. The cushion will hug the nose and mouth, and leak will be minimal.

Note: At no time should the nose be inside of the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly. A different size cushion may be needed.



With the mask assembled, place the cushion under the nose.



Position the frame on the top of the head. Pull the headgear over the back of the head.



Attach the magnetic headgear clips to the mask cushion.



To adjust the mask, peel the headgear tabs away from the fabric. Adjust the length of the top straps. Press the tabs back against the fabric to reattach. Repeat this step with the bottom straps.

Note: Do not overtighten the headgear.



Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head.

Note: Please read the Instructions for use for further fitting information.

Nasal

Using the nasal cushion sizing gauge (included with the mask), hold it horizontally under the nose. Select the cushion size based on where the outer edge of the nostrils and tip of the nose contact the gauge.

Note: Four cushion sizes (small, medium, large, medium-wide) are available.

Gel pillows

Using the gel pillows sizing gauge, place the pillows cushion tips into the nostrils. Choose the cushion size that allows the blue gel bases to sit comfortably under the nose without any gaps.

Note: Three cushion sizes (small, medium, large) are available.

1



The cushion opening will be directly under the nostrils. The cushion will hug the nose and leak will be minimal.

Note: At no time should the nose be inside the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly.



2



With the mask assembled, place the cushion under the nose.



3



Position the frame on the top of the head. Pull the headgear over the back of the head.



4



To adjust the mask, peel the headgear tabs away from the fabric. Adjust the length of the straps. Press the tabs back against the fabric to reattach.

Note: Do not overtighten the headgear.



5



Position the mask until it fits comfortably.



6



When finished, the elbow should rest at the top of the head.

Note: Please read the Instructions for Use for further fitting information.



Making it simple to find the right fit

Achieving the right fit with DreamWear is easy

Medium frame sizing


The medium mask frame will comfortably fit most faces. If the medium frame does not fit a patient's face, a small or large mask frame might better suit your patient's needs.



Medium correct frame

Proper fit

Large frame sizing

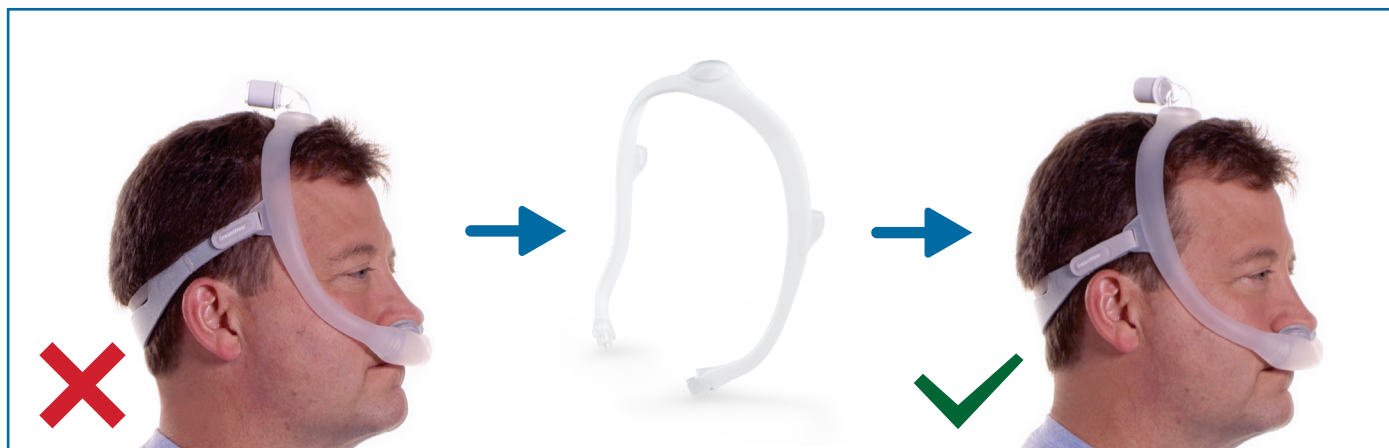


Mask too small

Large correct frame

Proper fit

Small frame sizing

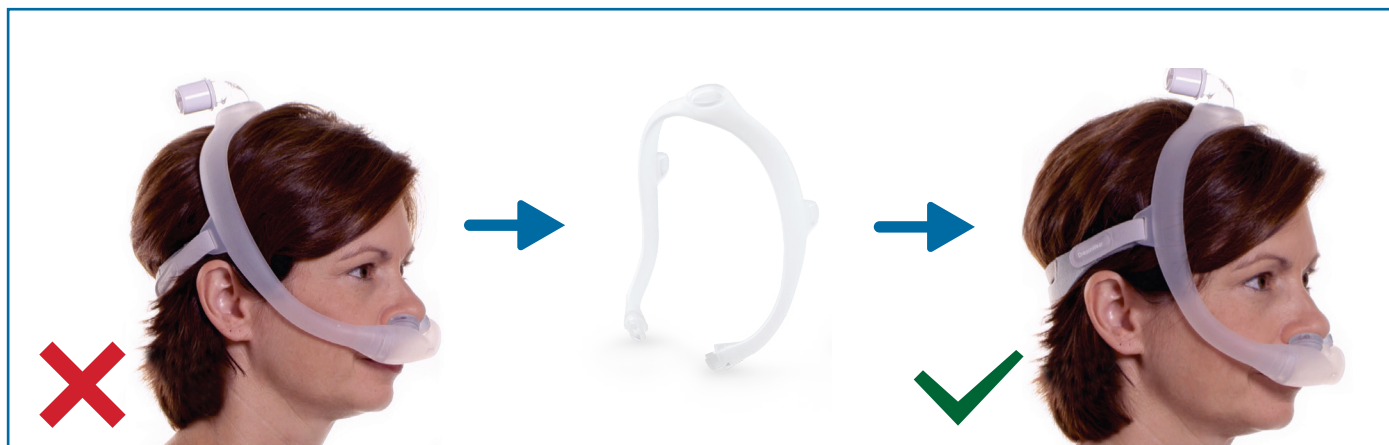

Mask too large

Small correct frame

Proper fit

Ordering information

DreamWear Nasal	Part Number
DreamWear FitPack mask with Headgear (S, M, L, MW cushions)	1116700
S, DreamWear UtN Nsl Cushion, RP	1116740
M, DreamWear UtN Nsl Cushion, RP	1116741
L, DreamWear UtN Nsl Cushion, RP	1116742
MW, DreamWear UtN Nsl Cushion, RP	1116743
DreamWear, Sm Frame, RP	1116745
DreamWear, Med Frame, RP	1116746
DreamWear, Lg Frame, RP	1116747
DreamWear Headgear, RP	1116750
DreamWear Fabric Wraps, RP	1116754
DreamWear Pillow	Part Number
DreamWear Gel Plw Fitpack , Med Frame, W/ HGR, GBL	1124984
S, DreamWear Gel Plw Cushion, RP	1125035
M, DreamWear Gel Plw Cushion, RP	1125036
L, DreamWear Gel Plw Cushion, RP	1125037
DreamWear Full Face	Part Number
S, DreamWear Full, Sm & Med Frm, GBL	1133375
M, DreamWear Full, Sm & Med Frm, GBL	1133376
L, DreamWear Full, Sm & Med Frm, GBL	1133377
MW, DreamWear Full, Sm & Med Frm, GBL	1133378
DreamWear Full, Fitpack, GBL	1133400
S, DreamWear Full Cushion, RP	1133430
M, DreamWear Full Cushion, RP	1133431
L, DreamWear Full Cushion, RP	1133432
MW, DreamWear Full Cushion, RP	1133433
DreamWear Small FRAME, RP	1116745
DreamWear Medium FRAME, RP	1116746
DreamWear Large FRAME, RP	1116747
DreamWear Full Headgear, RP	1133450