

**PHILIPS**  
RESPIRONICS

*DreamStation*

Menu enhancements

## Introducing the **improved DreamStation**

At Philips Respironics we continuously strive to improve our products and services to provide you and your patients our very best.

### **Improved Device Menu**

You've challenged us to make DreamStation even better by streamlining the device menus. We've taken that challenge seriously and are pleased to offer a new Advanced Menu selection.

### **New Advanced Menus option**

This feature defaults to a simplified patient and provider menu, which only includes the essentials. To view the full patient and provider menu (the current DreamStation menu), you simply need to set Advanced Menus to "ON" from the Provider mode.

**Note:** All data is available in *EncoreAnywhere* and *Care Orchestrator* no matter which menu is selected.



### **Designed for:**


- More efficient setup
- Streamlined patient training
- Improved overall experience

# New Menu screen layouts

All DreamStation devices, excluding DreamStation Go, have the Advanced Menus defaulted to "OFF" when they leave the factory.


## Patient Menu (Advanced Menu "OFF")

### My Info

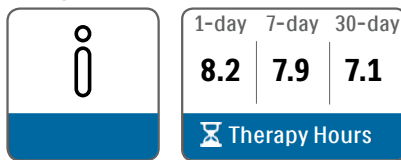


**Advanced Menus "ON" displays:** Therapy Hours, AHI, Mask Fit, Periodic Breathing, 90% Pressure and DreamMapper screens

### My Provider

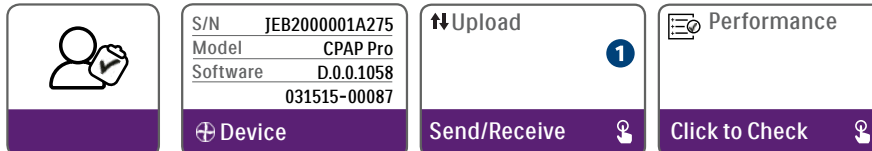

**Advanced Menus "ON" displays:** Device, Phone-in\*, Send/Receive, Performance Check, Compliance, VIC90 and A-Trial screens

### My Setup

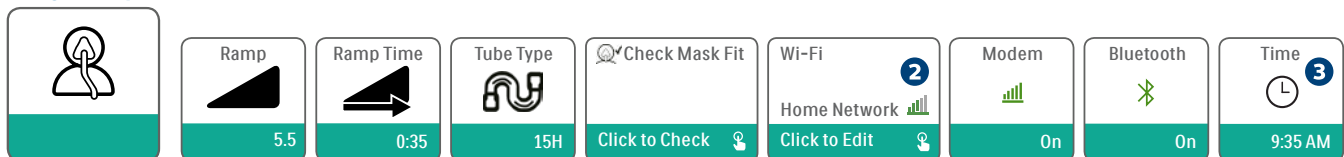


**Advanced Menus "ON" displays:** Ramp, Ramp Time, Flex, Humidification,\*\* Mask Type, Tube Type, Language, Check Mask Fit, Wi-Fi, Modem, Bluetooth and Time screens

### My Provider and My Setup references

- 1 If modem is present    2 If Wi-Fi or modem accessory is present    3 Users not prompted to set time with device start up


## Provider Menu (Advanced Menus "OFF")

### Comfort

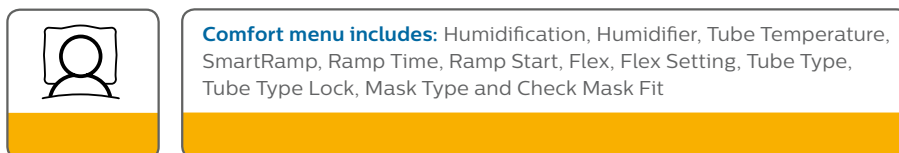

**Advanced Menus "ON" displays:** Humidification, Humidifier, Tube Temperature, SmartRamp, Ramp Time, Ramp Start, Flex, Flex Setting, Tube Type, Tube Type Lock, Mask Type, Mask Type Lock and Check Mask Fit screens


### Device

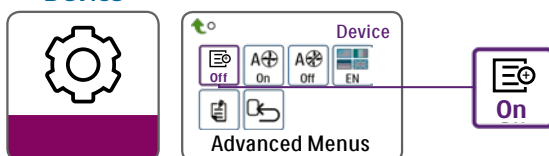

All DreamStation devices\*\*\*\* will have the Advanced Menus defaulted to "OFF." To turn Advanced Menus "ON." Follow these steps:

- Go to "Provider" mode • Navigate to "Device" push control knob • Rotate dial until Advanced Menus icon is "ON" • Push control knob to set

### Info

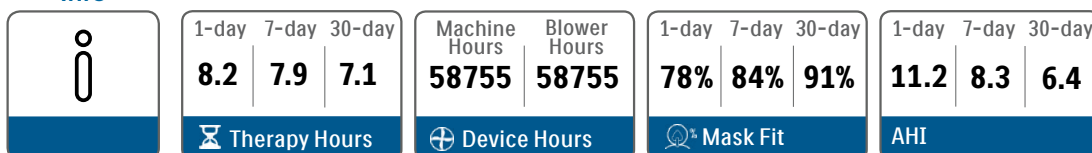

**Advanced Menus "ON" displays:** Phone-In, Compliance, VIC90, Days>4, Therapy Hours, Device Hours, Mask Fit, AHI, Periodic Breathing, 90% Pressure and A-Trial screens

\*Phone-in is only visible with no modem present

\*\*Humidification Type default to adaptive

\*\*\*Flex Lock for markets outside U.S.

\*\*\*\*Excluding DreamStation Go



© 2018 Koninklijke Philips N.V. All rights reserved.  
Specifications are subject to change without notice.

[www.philips.com](http://www.philips.com)

Caution: US federal law restricts these devices to  
sale by or on the order of a physician.

edoc KD 4/13/18 MCI 4108232  
1010 Murry Ridge Lane, Murrysville, PA 15668  
800-345-6443 • 724-387-4000