# PHILIPS

## AED Trainer 2

INSTRUCTIONS FOR USE  
MODE D'EMPLOI  
BEDIENUNGSANLEITUNG  
GEBRUIKSAANWIJZING  
GUÍA DEL USUARIO  
GUIDA UTENTE  
BRUKSANVISNING  
BRUKERHÅNDBOKEN  
KÄYTTÖOPPAASSA  
使用説明  
娑钐璧讨  
MANUAL DO USARIO  
取扱説明書

M3752A  
Edition 6

**HEARTSTART**  
DEFIBRILLATORS

INTENTIONALLY BLANK  
INTENTIONNELLEMENT LAISSÉ EN BLANC  
ABSICHTLICH LEER GELASSEN  
OPZETTELIJK ONBEDRUKT GELATEN  
ESTE ESPACIO SE HA DEJADO EN BLANCO DELIBERADAMENTE  
PAGINA LASCIATA INTENZIONALMENTE VUOTA  
SIDAN AR AVSIKTILIGEN TOM  
SKAL VÆRE TOM  
JÄTETTY TARKOITUKSELLA TYHJÄKSI  
刻意留白  
PÁGINA DEIXADA EM BRANCO INTENCIONALMENTE  
このページは空白です

# AED Trainer 2

## M3752A

|                                       |     |
|---------------------------------------|-----|
| Instructions for Use (ENGLISH) .....  | 1   |
| Mode d'emploi (FRANÇAIS).....         | 11  |
| Gebrauchsanweisung (DEUTSCH) .....    | 21  |
| Gebruiksaanwijzing (NEDERLANDS) ..... | 31  |
| Guía del usuario (ESPAÑOL) .....      | 41  |
| Guida utente (ITALIANO).....          | 51  |
| Bruksanvisning (SVENSKA) .....        | 61  |
| Brukerhåndboken (NORSK) .....         | 71  |
| Käyttöoppaassa (SUOMI).....           | 81  |
| 使用指南 (繁體中文) .....                     | 91  |
| 用戶指南 (簡體中文) .....                     | 101 |
| Manual do usuário (PORTUGUÊS) .....   | 111 |
| 取扱説明書 (日本語) .....                     | 121 |

# PHILIPS

**ENGLISH: About This Edition**

The information in this guide applies to the M3752A AED Trainer 2. This information is subject to change. Please contact Philips at [www.medical.philips.com/heartstart](http://www.medical.philips.com/heartstart) or your local distributor for information on revisions.

**Edition History**

Edition 6  
Publication date: February 2007  
Publication #: M3752-91900  
Assembly #: 011560-0006  
Printed in USA

**Copyright**

© 2007 Philips Electronics North America Corp.

No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

Unauthorized copying of this publication may not only infringe copyright but also reduce the ability of Philips Medical Systems to provide accurate and up-to-date information to users and operators alike.

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# AED Trainer 2

## Contents

|                                          |   |
|------------------------------------------|---|
| Intended Use .....                       | 1 |
| Features .....                           | 2 |
| Overview .....                           | 3 |
| Battery Replacement .....                | 6 |
| Use with Laerdal Training Manikins ..... | 6 |
| Accessories .....                        | 7 |
| Selecting Training Scenarios .....       | 7 |
| Standard Scenarios .....                 | 7 |

## Intended Use

The HeartStart AED Trainer 2 is designed to prepare emergency responders to use the FR2+ automated external defibrillator (AED).


The Trainer provides a variety of simulations, or training scripts, to help responders become familiar with the FR2+ Defibrillator and allow them to demonstrate the basic skills necessary to use the defibrillator in an emergency.

**NOTE:** Before using the FR2+ Defibrillator, the responder should have completed an appropriate responder training program, which may include the use of the AED Trainer 2. Having completed such a course, the responder should demonstrate a proficiency in assessing the patient for lack of responsiveness and absence of breathing, cardiopulmonary resuscitation (CPR), and knowledge of applicable medical protocols and procedures.


## Features

- The AED Trainer 2 provides simulated shock delivery. It has no high-voltage capabilities, ensuring safety during training. *IMPORTANT: Remember handling safety when using working defibrillators that give a high-voltage shock.*
- The AED Trainer 2 is designed for use with the AED Little Anne and Resusci Anne manikins with Laerdal Link Technology by giving responses to pad placement on the manikins. It can also be used with any other manikin. *IMPORTANT: Realistic responses to pad placement feature is only available when used together with the AED Little Anne or Resusci Anne manikins with Laerdal Link Technology.*
- The AED Trainer 2 can be used with an *optional* infrared remote control. The remote control gives the instructor the ability to alter training scenarios while in progress, to test student response.
- The AED Trainer 2 is pre-configured with 10 training scenarios that simulate realistic sudden cardiac arrest episodes. These scenarios are compatible with training programs developed by nationally and internationally recognized responder programs.
- In addition, three custom scenarios can be created using the optional Scenario Builder PC software. The software is available at no charge via the internet at <http://www.medical.philips.com/goto/Trainer2>. Note that this software also enables change of trainer language and configuration settings.
- The AED Trainer 2 can be connected to the serial port of a PC (generally labeled "COM1"). The optional PC software lets you configure custom training scenarios, set-up various protocol parameters and change language output. Connection to a PC serial port requires a standard 1:1 9-pin D-sub serial cable.
- Check with your local medical director and Emergency Medical Services provider for additional training requirements, protocols, and materials.

## Overview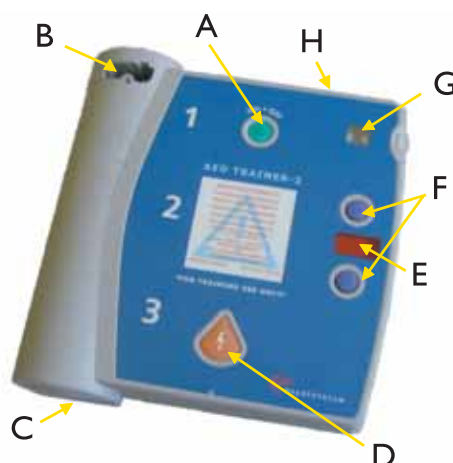

- A On/Off button. Turns unit on, initiates voice prompts. Press once to turn unit on. Press again to turn off.
- B Pads Connector Socket. Insert the training pads connector into the pads connector socket. The LED flashes to indicate the socket location. The LED is covered when the defibrillation pad connector is properly inserted in the socket.
- C Speaker. Provides audible instructions and information about the unit and the patient's status. The speaker volume can be controlled by the two blue Up/Down buttons or the *optional* remote control. The last setting will be stored and maintained even if the unit has been turned off.
- D Shock Button. Controls simulated shock delivery. The button flashes when the unit is ready to deliver a simulated shock.
- E Operator Display and Remote Control Receiver. Indicates voice prompt volume, training scenario selection, and responses to remote control inputs. (See the diagram and button description of the remote control for examples of these indicators.)

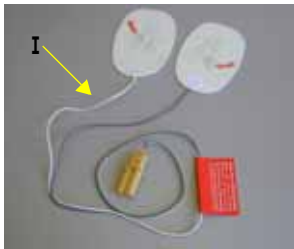- F Up/Down Buttons. Control scenario selection, adjust voice prompt volume during use. *IMPORTANT: These buttons have different functions on the real automated external defibrillators. Please refer to the FR2+ Instructions for Use.*
- G Status Window. This window simulates the FR2+ status display. By pulling on the tab you can switch between the hourglass symbol (ready for use) and the red **X** (troubleshooting required).
- H Battery. The AED Trainer 2 battery resembles that of the FR2+ Defibrillator. Insertion and removal mechanisms are similar. However, the AED Trainer 2 battery case holds six (6) replaceable C-cell alkaline batteries and can only be used with the AED Trainer 2.
- I Training Pads. Training pads resemble those used with the FR2+ Defibrillator, however they do not deliver a shock. Training pads with a yellow connector are for use with any manikin; those with a white connector are for use with the AED Little Anne or Resusci Anne manikins with Laerdal Link Technology.  

*NOTE: The AED Trainer 2 comes with one set of training pads with a yellow connector. When the yellow connector is inserted into the trainer's connector socket, the trainer responds as if the pads are attached to a patient and proceeds through the treatment protocol. The training pads are for training use only and will not function with an actual FR2+ Defibrillator.*
- J Remote Control. The *optional* remote control can be used to control the different features of the AED Trainer 2 during training sessions. (See the table on the next page for descriptions of individual functions. Most remote control actions generate a response in the operator display window on the AED Trainer 2 device as shown in the table.)

## Optional Remote Control


**NOTE:** The remote control is not required to operate the AED Trainer 2. Speaker volume and training scenario can be selected via the blue Up/Down keys. See “Overview” and “Selecting Training Scenarios” for more information.

|                                                                                     |                                                                                                                                                                                                                                                                                                             |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | This function is for future use.                                                                                                                                                                                                                                                                            |
|    |  Select one of the 10 standard scenarios.                                                                                                                                                                                  |
|    |  Select one of the 3 custom scenarios.                                                                                                                                                                                     |
|    |  Show the selected scenario in the operator display for 2 seconds.                                                                                                                                                         |
|    | Pause/Resume scenario and AED Trainer 2 activity. Pause (  ) will freeze all activity in the trainer. Pause is indicated by flashing symbol shown. Resume (▶) resumes the activity from the point at which it was paused.                                                                                   |
|    |  Simulate motion artifact. Analysis will be interrupted and a voice prompt asking to stop all motion will be issued.                                                                                                       |
|   |  Simulate loose electrodes connection.                                                                                                                                                                                    |
|  |  Simulate good electrodes connection. This function will also override the detection of poor physical pad placement.                                                                                                     |
|  |  Simulate a shockable rhythm. Overrides the running scenario.                                                                                                                                                            |
|  |  Simulate a non-shockable rhythm. Overrides the running scenario.                                                                                                                                                        |
|  |  Light electrode pads position LEDs on manikin chest (for use only with manikins and pads using Laerdal Link Technology, such as AED Little Anne).                                                                       |
|  |  Simulate low battery, press once for Low Battery. Press again for Replace Battery.                                                                                                                                      |
|  |  Simulate error condition. Unit will stop operation and give “chirping” sound every 10 seconds. “Er” is continuously displayed in the operator display. This state represents a red X in the status display of the FR2+. |
|  |  Adjust speaker volume.                                                                                                                                                                                                  |

## Battery Replacement

To install the battery case into the AED Trainer 2, slide the battery case into the trainer's battery receptacle, with the battery case's label pointed upwards. The battery case locks into place when inserted properly, and the Trainer 2 automatically turns on and enters "set-up" mode, which permits selection of the training scenario via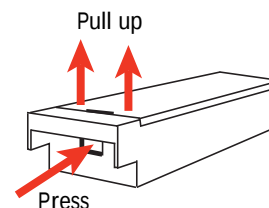

Up/Down keys. If battery power falls below a preset point, a voice message, "Low Battery," is provided when the trainer is turned on. When you hear the message, replace the battery as follows:

1. Remove the battery case. Lift the battery case handle up until the two locking tabs release their hold, then remove the battery case from the Trainer 2.
2. Lift the lid off of the battery case by pressing inwards against the locking tab at the end of the case until it releases (see illustration).
3. Replace the alkaline batteries. Remove the old batteries and install 6 new LR14 (C-cell) alkaline batteries. Be sure to follow the battery-placement icons etched into the inside of the battery case.
4. Reposition the lid, and reinsert the battery case into the Trainer 2.

## Use with Laerdal Training Manikins

The AED Trainer 2 can be used with the AED Little Anne and Resusci Anne training manikins with Laerdal Link Technology. This technology allows realistic response to electrode placement on the manikin surface; they also have an LED in the chest skin indicating the center of correct pad placement. The LED lights up automatically when pads are attached. If an electrode is loose or incorrectly placed on the manikin, the trainer will repeat the voice prompt to apply the electrode pads. *NOTE: If using the AED Little Anne or Resusci Anne manikin with Laerdal Link Technology, electrode training pads with a white connector are required.*

## Accessories

- 07-10900.....Defibrillator Training Pads (for use with any training manikin)
- M3871A.....FR2 Infant/Child Training Pads
- M3753A.....Remote Control for AED Trainer 2 (*optional*)
- M3754A.....Programming Kit for AED Trainer 2 (optional software and cable for configuring trainer parameters, language, custom scenarios)  
*NOTE: The software is also available at no charge via the internet at <http://www.medical.philips.com/goto/Trainer2>.*
- M3755A.....Training Pads for AED Little Anne and Resusci Anne with Laerdal Link Technology
- M3756A.....AED Little Anne Training System (AED Trainer 2, Instructions for Use, carrying case, quick reference card, remote control, AED Little Anne training manikin, and 1 set training pads for AED Little Anne)


## Selecting Training Scenarios

You can select a standard training scenario either by using the *optional* remote control or via the Up/Down buttons in the AED Trainer 2's "set-up" mode. To activate the set-up mode, make sure the electrode pads are not connected, then remove and reinsert the battery. Alternatively, press and hold down the two blue Up/Down buttons while simultaneously pressing the green On/Off button. When the Trainer is in set-up mode, the operator display shows the currently selected scenario. To change the scenario selection, use the Up/Down buttons to select the desired scenario, then press the On/Off button to exit the set-up mode. The last scenario selection is stored in the unit even if the battery is taken out.

**NOTE:** The Trainer 2 provides a CPR/patient care interval after each shock delivery and each No Shock decision.

## Standard Scenarios

| symbol                                                                            | meaning                                                      |
|-----------------------------------------------------------------------------------|--------------------------------------------------------------|
|  | <i>(Simulated) shockable rhythm detected by Trainer.</i>     |
|  | <i>(Simulated) non-shockable rhythm detected by Trainer.</i> |
|  | <i>(Simulated) pads problem detected by Trainer.</i>         |
|  | <i>(Simulated) motion detected by Trainer.</i>               |
|  | <i>CPR First selected.</i>                                   |
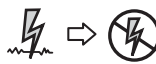
| scenario number                                                                                   | scenario description                                                                                                                                                                                                                                                                                |
|---------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| scenario 1<br> | <i>Shockable rhythm detected, one shock needed for conversion</i> <ul style="list-style-type: none"> <li>Trainer detects a shockable rhythm, instructs user to deliver a shock.</li> <li>Trainer detects a non-shockable rhythm.</li> </ul>                                                         |
| scenario 2<br> | <i>Shockable rhythm detected, four shocks needed for conversion</i> <ul style="list-style-type: none"> <li>Trainer detects a shockable rhythm, instructs user to deliver a shock. Repeats for total of four shocks.</li> <li>Trainer detects a non-shockable rhythm.</li> </ul>                     |
| scenario 3<br> | <i>Troubleshooting pads, one shock needed for conversion</i> <ul style="list-style-type: none"> <li>Trainer detects poor pad contact.</li> <li>After 15 seconds, Trainer detects a shockable rhythm, instructs user to deliver a shock.</li> <li>Trainer detects a non-shockable rhythm.</li> </ul> |

| scenario number                                                                                       | scenario description                                                                                                                                                                                                                                                                                                                                                                                                                                           |
|-------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p>scenario 4</p>    | <p><i>Shockable rhythm detected, conversion, return to shockable rhythm, conversion</i></p> <ul style="list-style-type: none"> <li>• Trainer detects a shockable rhythm, instructs user to deliver a shock.</li> <li>• Trainer detects a non-shockable rhythm.</li> <li>• Trainer detects refrillation (return to a shockable rhythm), instructs user to deliver a shock.</li> <li>• Trainer detects a non-shockable rhythm.</li> </ul>                        |
| <p>scenario 5</p>    | <p><i>Non-shockable rhythm detected</i></p> <ul style="list-style-type: none"> <li>• Trainer detects a non-shockable rhythm throughout.</li> </ul>                                                                                                                                                                                                                                                                                                             |
| <p>scenario 6</p>   | <p><i>Shockable rhythm detected, two shocks needed for conversion</i></p> <ul style="list-style-type: none"> <li>• Trainer detects a shockable rhythm, instructs user to deliver a shock.</li> <li>• Trainer still detects a shockable rhythm, instructs user to deliver another shock.</li> <li>• Trainer detects a non-shockable rhythm.</li> </ul>                                                                                                          |
| <p>scenario 7</p>  | <p><i>CPR First has been selected, shockable rhythm (late VF) detected, Trainer instructs user to perform CPR, one shock needed for conversion</i></p> <ul style="list-style-type: none"> <li>• Trainer detects a shockable rhythm (late VF, indicative of long down-time), instructs user to perform CPR.</li> <li>• Trainer still detects a shockable rhythm, instructs user to deliver shock.</li> <li>• Trainer detects a non-shockable rhythm.</li> </ul> |
| <p>scenario 8</p>  | <p><i>Troubleshooting pads, two shocks needed for conversion</i></p> <ul style="list-style-type: none"> <li>• Trainer detects poor pad contact.</li> <li>• After 15 seconds, Trainer detects a shockable rhythm, instructs user to deliver a shock.</li> <li>• Trainer still detects a shockable rhythm, instructs user to deliver another shock.</li> <li>• Trainer detects a non-shockable rhythm.</li> </ul>                                                |

| scenario number                                                                                  | scenario description                                                                                                                                                                                                                                                                                                                                                                                                            |
|--------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| scenario 9<br>  | <i>Shockable rhythm detected</i> <ul style="list-style-type: none"> <li>• Trainer detects a shockable rhythm throughout.</li> </ul>                                                                                                                                                                                                                                                                                             |
| scenario 10<br> | <i>Motion detected, one shock advised, one shock needed for conversion, low battery</i> <ul style="list-style-type: none"> <li>• Trainer detects motion artifact, instructs user to stop all motion.</li> <li>• Trainer detects a shockable rhythm, instructs user to deliver a shock.</li> <li>• Trainer detects a non-shockable rhythm.</li> <li>• Trainer detects low battery, instructs user to replace battery.</li> </ul> |

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Niets uit deze uitgave mag in enige vorm of op enige manier worden verveelvoudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.
Philips Medizin Systeme Boeblingen GmbH
Philips Electronics Hong Kong Ltd.