PerforMax

INSTRUCTIONS FOR CLEANING AND DISINFECTION

CAUTION

- For Clinical Use Only.
- The following processes are validated for the PerforMax multi-patient use mask when used in accordance with the instructions provided by the manufacturer of the cleaning and disinfection products and/or processes. Any deviation from said instructions or the cleaning agents listed below may impact the performance or durability of the product. Review those instructions for additional warnings and cautions
- Open or remove the port cap located on the elbow prior to cleaning and disinfection.

WARNING

- Inspect the mask components after disinfection and replace any components that are damaged. See the *Instructions for Use* included with the product for additional warnings and cautions.
- Exhalation/Air Entrainment (EE) Valve Elbow only: Verify that the air entrainment valve flapper moves freely after cleaning and disinfection. Replace the elbow if the flapper does not function correctly.

NOTES

- The headgear cannot be processed using the disinfection methods listed. The headgear is for single use and must be replaced before the mask is reused. See the *Instructions for Use* included with the product.
- After disinfection, discoloration of the cushion or a slight odor is normal.

CLEANING

- Disassemble the mask per the *Instructions for Use* included with the product.
- Clean the mask using a soft bristle brush to adequately remove adhering substances from each component while soaking in a commercially available, anionic detergent such as MediZyme (used with thermal treatments and autoclave) or Manu Klenz (used with chemical and gas plasma treatments). Extra attention should be given to the crevices and cavities during cleaning.
- Rinse the mask and components with potable water (minimum 5 liters per mask) and air dry out of direct sunlight.

HIGH-LEVEL DISINFECTION

The following high-level disinfection processes are intended for PerforMax multi-patient use mask in a clinical or hospital/institution environment only. For single patient use refer to the *Instructions for Use* included with the product.

Autoclave

TEMPERATURE	TIME	MAXIMUM CYCLES	NOTES
121°C	15 minutes	5	Gravity displacement cycle No sterilization bag

Thermal Disinfection: Apply only one treatment option following the manufacturer's instructions.

TEMPERATURE	TIME	MAXIMUM CYCLES
70°C	100 minutes	10
or		
75°C	30 minutes	10
or		
80°C	10 minutes	10
or		
90°C	1 minute	10
or		
93°C	10 minutes	10

Continued

PHILIPS

Chemical Disinfection: Apply only one treatment option following the manufacturer's instructions.

TREATMENT	EXPOSURE TIME	ACTIVE INGREDIENTS	SOLUTION	RINSE	MAXIMUM CYCLES
CIDEX Activated Dialdehyde Solution	45 minutes	2.4% Glutaraldehyde	Full Strength	Potable Water, 5 Liters per Mask (minimum)	10
or					
CIDEX OPA	12 minutes	0.55% Ortho-Phthalaldehyde	Full Strength	Potable Water, 5 Liters per Mask (minimum)	10
or					
WAVICIDE	45 minutes	2.65% Glutaraldehyde	Full Strength	Potable Water, 5 Liters per Mask (minimum)	10
or					
CONTROL III	10 minutes	10% Alkyl Dimethyl Benzyl Ammonium Chloride 10% Alkyl Dimethyl Ethyl Benzyl Ammonium Chloride	8 ml per liter	Potable Water, 5 Liters per Mask (minimum)	10

Hydrogen Peroxide Gas Plasma

TREATMENT	EXPOSURE TIME	MAXIMUM CYCLES
STERRAD 100S	Machine Controlled	10

Visit www.respironics.com or contact customer service to obtain PerforMax multi-patient use cleaning and disinfection instructions.
QTY=1 REF 1067125, QTY=25 REF 1050854.

Respironics: 1-800-345-6443 (USA or Canada only) or 724-387-4000

Respironics Deutschland: +49 8152 93060

Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668
USA

Respironics Deutschland
Gewerbestrasse 17
82211 Herrsching, Germany
[EC REP]

Respironics is a registered trademark, and PerforMax is a trademark, respectively, of Respironics, Inc. and its affiliates. CIDEX, CIDEX OPA, and STERRAD are registered trademarks of the Advanced Sterilization Products. Wavicide is a registered trademark of Medical Chemical Corporation (MCC). Control III is a trademark of Maril Products, Inc. MediZyme is a registered trademark of MEDICHEM International Ltd. ©2009 Respironics, Inc. and its affiliates. All rights reserved.

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