HeartStart FR3 Language Card

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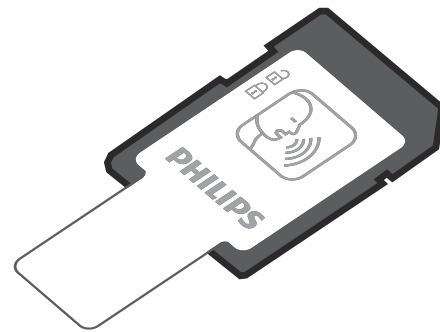
ENGLISH: About this edition

The information in this guide applies to the 989803150101 FR3 Language Card used with the 861388 and 861389 HeartStart FR3 Defibrillators. This information is subject to change. Please contact Philips at www.philips.com/productdocs or your local distributor for information on revisions.

Edition history

Edition 2

Publication date: July 2011

Publication number: 453564069392

Copyright

© 2011 Koninklijke Philips Electronics N.V. No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.

Device manufacturer

Philips Medical Systems, Seattle, WA, USA
98121-1825

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Philips Medical Systems Seattle, WA,
Etats-Unis 98121-1825

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Philips Medizin Systeme Boeblingen GmbH
Hewlett-Packard Strasse 2
71034 Boeblingen, Allemagne
(+49) 7031-463-2254

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Philips Medical Systems Seattle,
WA 98121-1825, VS

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Fabricante del dispositivo

Philips Medical Systems Seattle,
WA, EE.UU. 98121-1825

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Philips Medical Systems Seattle, WA, USA 98121-1825

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Philips Medizin Systeme Boeblingen GmbH
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
(+49) 7031-463-2254

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Philips Medical Systems, Seattle, WA,
USA 98121-1825

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Philips Medizinsysteme Boeblingen GmbH
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
(+49) 7031 463-2254

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Philips Medical Systems Seattle, WA, USA
98121-1825

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Philips Medizin Systeme Boeblingen GmbH
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
(+49) 7031-463-2254

FR3 LANGUAGE CARD

The HeartStart FR3 language card 989803150101 is intended for use with the HeartStart FR3 Defibrillator (FR3) only.* The FR3 can be configured to operate in any language that is provided on the FR3 language card. In addition, if the bilingual option is enabled, the FR3 allows the user to choose a configured secondary language at the time of use.

The language card can be used in three ways:

1. To change the FR3 default primary language only.
2. In conjunction with HeartStart Configure software† [REF: 861487], available separately from Philips, to enable the bilingual option and select a secondary language.
3. In conjunction with HeartStart Configure, to make changes to other FR3 configuration settings.

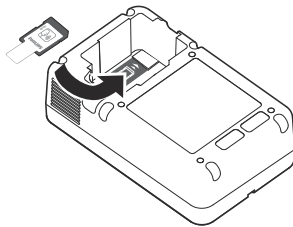
NOTE: Changing the primary language and/or enabling the bilingual option and choosing a secondary language does not change default or previously reconfigured settings for device operation, self-test options, patient care, defibrillation, and advanced mode parameters.

CHANGING THE PRIMARY LANGUAGE ONLY

The primary language is the language the FR3 automatically uses for voice and text prompts. The FR3 uses the language card to identify available languages.‡ To change the primary language, follow these steps:

INSTALLING THE LANGUAGE CARD


1. Remove the battery from the FR3 and remove the data card, if present. To remove the data card, grasp the card by its tail, press the retention clip away from the card, and pull the card out. See the illustration on page 3.
2. Insert the FR3 language card in the data card slot in the bottom of the battery compartment. A label on the floor of the battery

* See the *Guide to Setup, Operation, and Maintenance* provided with the FR3 for an explanation of symbols on this accessory and its packaging.

† This product requires a prescription in the United States.

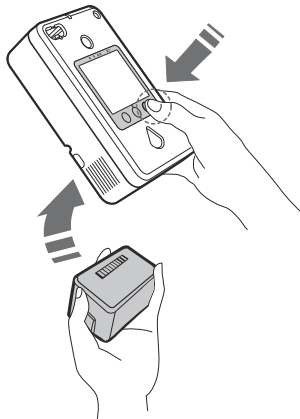‡ In the unlikely event that the FR3 ever loses its configured language, it defaults to U.S. English.


compartment indicates correct orientation for inserting the card. A retention clip helps hold the card in the slot. The tail on the data card serves as a visual reminder that a card is installed and aids in card removal.

3. Reinstall the battery **while holding down the right option button**. Keep the button pressed until the language selection screen is displayed.

NOTE: The FR3 ignores the lock switch on the side of the language card.

SELECTING A NEW PRIMARY LANGUAGE

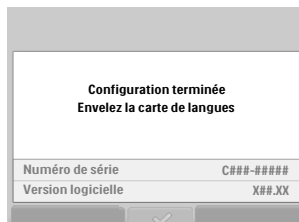1. The FR3 display screen automatically provides a list of languages. There may be more than one page of languages.
2. Press the ▼ option button to scroll to the language you want, then press the ✓ option button to select it. (If no selection is made within 60 seconds, the FR3 shuts down. To restart the process, remove and reinsert the battery while holding down the right option button.)
3. The new language file automatically loads. The FR3 displays the language name and a progress bar. During language installation, the FR3 cannot be turned off. (If the new language installation does not complete, the FR3 reverts to the default primary language and displays a setup error screen.)
4. When the language load is complete, the FR3 operates in the new language. It displays a screen announcing that the setup is complete, and reminding you to remove the language card.


REMOVING THE LANGUAGE CARD

1. Now remove the battery and the FR3 language card. This will turn off the device. Store the language card in a safe place for future use or reference.

NOTE: To remove the language card, grasp the data card by its tail, press the retention clip away from the card, and pull the card out.

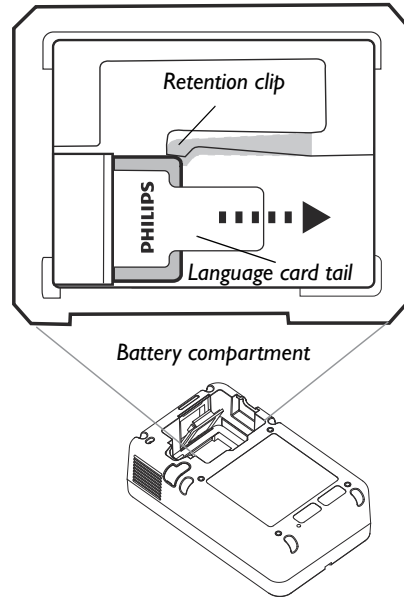2. Insert an FR3 data card, if desired, in the data card slot, and reinstall the battery. As soon as the battery is installed, the FR3 automatically runs a user-initiated test. Press the buttons when directed, or the test will fail.
3. On completion of the test, the FR3 displays the results and then displays the main Administration menu. If the FR3 clock has not yet been set, the FR3 will provide a reminder message. The device will automatically turn off after a few minutes. Thereafter, any time you turn on the FR3, it will operate using the new primary language.

NOTE: If you change the primary language or choose a secondary language for bilingual operation, Philips recommends that you relabel the green On/Off button on the FR3 with the appropriate language decal(s) from the set provided with the language card.

ENABLING THE BILINGUAL OPTION AND/OR CHANGING OTHER CONFIGURATION SETTINGS

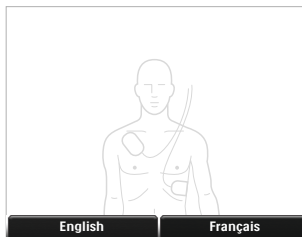
Under the supervision of the Medical Director, you can change the primary language, and also enable the bilingual option, select a secondary language for bilingual operation, or make other changes to the FR3 configuration. These changes require both the language card and HeartStart Configure, separately available software. HeartStart Configure uses the FR3 language card to identify available languages and provides directions to changing the configuration settings. To change configuration settings, follow these steps:

1. Install HeartStart Configure on a computer.
2. From the computer's Start menu, open HeartStart Configure.
3. Insert the FR3 language card into the data card slot on the computer, or into an optional USB data card reader [REF: M3524A], available separately from Philips, connected to the computer.
4. Follow the directions for enabling the bilingual option, if desired, and selecting the secondary language, *OR* for making changes to additional configuration settings, if desired.
5. Follow the directions for saving the revised setup on the computer and writing the saved setup to the FR3 language card.
6. Remove the language card from the computer or the USB data card reader. Protect the card from environmental extremes and store it in a safe place for future use or reference.

LOADING THE NEW SETUP CONFIGURATION

1. Install the language card in the FR3 as described on page 1. Reinstall the battery while holding down the right option button.
2. The language and new configuration changes automatically load to the FR3. During setup loading, the FR3 cannot be turned off.
3. When the new setup has been successfully loaded, the FR3 displays a screen for five minutes, announcing that the setup is complete, and reminding you to remove the language card. (If an error occurs during loading, the FR3 displays a SETUP ERROR message in the default language.)
4. Follow the directions for removing the language card as described on page 2. Insert an optional FR3 data card, if desired, in the data card slot.
5. Reinstall the battery.
6. The FR3 automatically runs a user-initiated test. Press the buttons when directed, or the test will fail. Thereafter, any time you turn on the FR3, it will operate using the new configuration settings.

NOTE: If you choose a secondary language for bilingual operation, Philips recommends that you relabel the green On/Off button on the FR3 with the appropriate language decal(s) from the set provided with the language card. The decals allow for relabeling the primary language and identifying the secondary language.


DEVICE BEHAVIOR IN BILINGUAL OPERATION

When the FR3 is configured for bilingual operation, you can select between a primary and a secondary language after turning on the FR3 for use. The device immediately prompts you to select either the primary language or the secondary language by pressing one of the option buttons. The selection prompts are provided in each language.

If you select the secondary language, the FR3 announces the language selection and uses that language for all operation until the FR3 returns to standby mode. If you do not select the secondary language within 10 seconds, the FR3 continues to operate in the primary language.

You must select either the primary or the secondary language each time you turn on the FR3 for use.

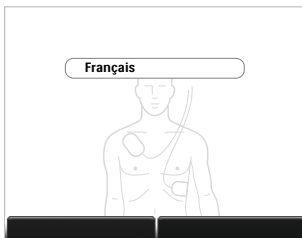When you select the language, the FR3 displays a screen acknowledging the language selection.

NOTE: If you select the wrong language by mistake, you can press the On/Off button twice to reset the FR3 and select the correct language.

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© 2011 Koninklijke Philips Electronics N.V. Ohne vorherige Zustimmung seitens des Copyright-Inhabers darf kein Teil dieser Veröffentlichung in irgendeiner Form vervielfältigt, reproduziert, elektronisch verarbeitet, in einem Datenabrufsystem gespeichert oder in eine andere natürliche oder Computer-Sprache übersetzt werden.
© 2011 Koninklijke Philips Electronics N.V. Niets uit deze uitgave mag in enige vorm of op enige manier worden vervoelvoudigd, verzonnen, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.
© 2011 Koninklijke Philips Electronics N.V. Queda prohibida la reproducción, transmisión, transcripción, almacenamiento en un sistema de recuperación, o traducción a cualquier idioma de parte alguna de esta publicación en cualquier forma o por cualquier medio sin el previo consentimiento del propietario del copyright.
Electronics Ltd.
Philips Electronics Hong Kong Ltd.