HeartStart FR3 Infant/Child Key

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PHILIPS

ENGLISH: About This Edition
The information in this guide applies to the 989803150031 Infant/Child Key for the HeartStart FR3 defibrillator. This information is subject to change. Please contact Philips at www.philips.com/productdocs for information on revisions.

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No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.

Device Manufacturer
Philips Medical Systems
Seattle, WA 98121-1825, USA

Authorized European Representative
Philips Medizin Systeme Boeblingen GmbH
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
(+49) 7031-463-2254

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
HEARTSTART FR3 INFANT/CHILD KEY

DESCRIPTION AND INTENDED USE

The FR3 Infant/Child Key 989803150031 is for use *only* with the HeartStart FR3 Defibrillator (“FR3”) to treat a suspected victim of sudden cardiac arrest who is under 55 lbs (25 kg) or 8 years old. If you are uncertain of the child’s exact weight or age, use the FR3 without the Infant/Child Key.*

For instructions for using the FR3, refer to the *Guide to Setup, Operation, and Maintenance* provided with the FR3.†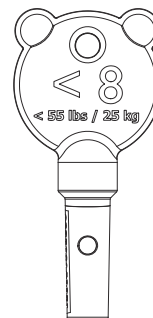

WARNING: Most cardiac arrests in children are not caused by heart problems. When responding to cardiac arrest in an infant or child:

- Provide infant/child CPR while a bystander calls EMS and brings the FR3.
- If no bystander is available, provide 1-2 minutes of CPR before calling EMS and retrieving the FR3.
- If you witnessed the child's collapse, call emergency medical services immediately and then get the FR3.
- Turn on the FR3 and install the Infant/Child Key.‡

Alternatively, follow your local protocol.

With the Infant/Child Key inserted, the FR3 announces “Infant/Child Mode” and automatically adjusts the defibrillation therapy to a level more appropriate for infants and children. In addition, the FR3 provides instructions for infant/child CPR as part of any configured CPR protocol.

No special defibrillator pads are required to use the FR3 to treat an infant or child. The HeartStart SMART Pads III provided with the defibrillator are designed for use on all patients.


* The Infant/Child Key should not be used when treating an adult, because the lower defibrillation therapy level may not be effective.

† See the *Guide to Setup, Operation, and Maintenance* provided with the FR3 for an explanation of symbols on this accessory and its packaging.


‡ NOTE: It does not matter whether you insert the Infant/Child Key before or immediately after turning on the FR3.

The FR3 Infant/Child Key is supplied with a tether cable designed to secure it to an FR3 system case, if used. *The tether is not required for use of the Infant/Child Key.* To use the tether, follow these steps:


1. Insert the tether through the hole in the top of the Infant/Child Key.
2. Thread the “winged” end of the tether (A) through the single hole in its other end (B).
3. Straighten out the tether and tighten the loop around the Key.
4. Insert the “winged” end of the tether into the slot inside the Infant/Child Key compartment in the upper lid of the case (A). Feed the tether into the slot, then snap the Key into the compartment (B).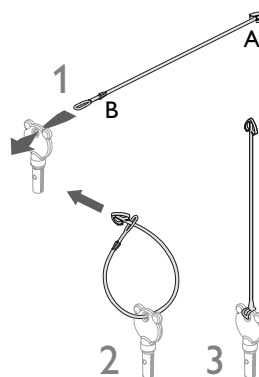

Thread tether into Key.

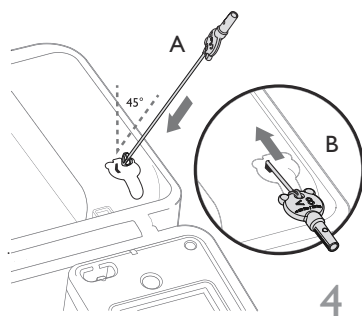Install Key in case.

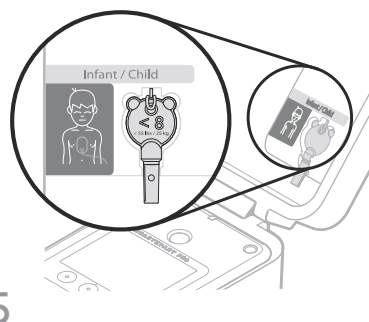
Apply decal to case.


5. For increased visibility of the Infant/Child Key, Philips recommends that you label the Infant/Child Key compartment in the FR3 case, as illustrated, with the appropriate language(s) from the set of decals provided with the Key. The decal is designed to permit labeling in two languages for situations in which the FR3 is configured for bilingual operation.

DIRECTIONS FOR USE

1. Press the Key firmly into the port at the top right of the front panel of the FR3. When the FR3 is turned on, it announces “Infant/Child Mode” and displays infant/child pads placement on the screen.*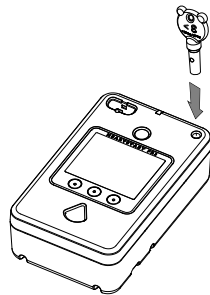

Insert Key into FR3.

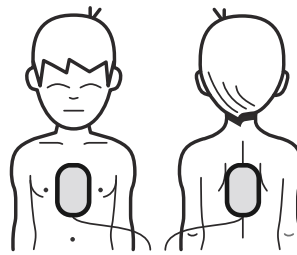Place pads.

2. Place the pads, one on the chest and one on the back. *It does not matter which pad is placed on the chest or the back.*
3. Follow the FR3 voice instructions. It will talk you through each step of using it.

If the Infant/Child Key is removed during use of the FR3, the FR3 will announce “Adult Mode.” Any shocks delivered will be at the adult energy dose, and any CPR instructions given will be for adults

CAUTION: Do not leave the Infant/Child Key inserted in the FR3 when not in use. The lower defibrillation therapy level delivered with the Key inserted may not be effective for treating an adult. The FR3 provides a prompt to remove the Infant/Child Key on the status screen when shutting down. In addition, if the Infant/Child Key is left inserted, the FR3 will give alarm chirps the next time it runs a self-test.

* If the bilingual option has been configured for the FR3, the screen provides a choice of languages first.

TRAINING

The Infant/Child Key can be used with the HeartStart FR3 Training Pack [REF: 989803150191] to enable training scenarios for treating infants and children. CPR for infants and children is different than for adults. It is recommended that anyone who may use the Infant/Child Key be trained in infant/child CPR.

STORAGE

Keep the Infant/Child Key in its compartment in the FR3 system carry case or, if a custom case is used, with the FR3.

MAINTENANCE

The Infant/Child Key can be cleaned with a soft cloth dampened in soapy water, chlorine bleach (2 tablespoons per quart or liter of water), an ammonia-based cleaner, or isopropyl alcohol (70% solution).

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PHILIPS

Philips Medical Systems is part of
Royal Philips Electronics

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Philips Medical Systems er en del av
Royal Philips Electronics

Philips Medical Systems on osa
Royal Philips Electronics -konsernia

Philips Medical Systems 隸屬 Royal
Philips Electronics

Philips Medical Systems 隶属于:
Royal Philips Electronics

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Canada

Philips Healthcare, a Division of Philips Electronics Ltd.
281 Hillmount Road
Markham, Ontario
L6C 2S3, Canada
(800) 291-6743

Europe, Middle East, and Africa

Philips Medizin Systeme Boeblingen GmbH
Cardiac and Monitoring Systems
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
+49 7031-463-2254

Latin America

Philips Medical Systems Ltda.
Rua Verbo Divino - 1400 - 7o Andar
04719-002 -Sao Paulo, Brazil
0800 7017789

Asia Pacific

Philips Electronics Hong Kong Ltd.
6/F, Core Building I
1 Science Park East Avenue
Hong Kong Science Park
Shatin, New Territories, Hong Kong
(852) 2821 5888

453564069362

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Niets uit deze uitgave mag in enige vorm of op enige manier worden verveel- voudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.