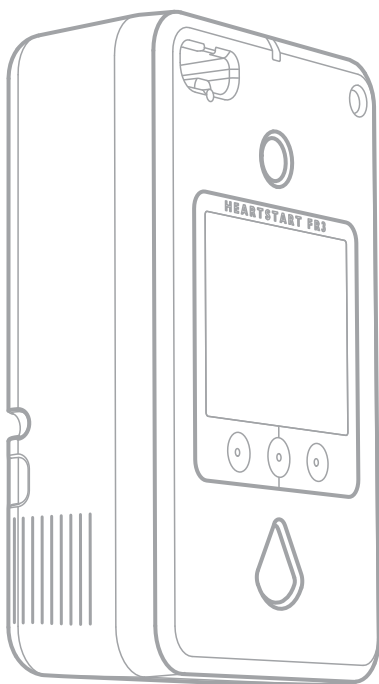
SUPPLEMENTAL INFORMATION TO HEARTSTART FR3 DEFIBRILLATOR INSTRUCTIONS

INFORMATIONS SUPPLÉMENTAIRES

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About this edition

The information in this guide applies to the HeartStart FR3 defibrillator. This information is subject to change. Please contact Philips at www.philips.com/productdocs or your local Philips representative for information on revisions.

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No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

Philips Medical Systems reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequence resulting from use of this publication.

Authorized EU Representative

Philips Medizin Systeme Boeblingen GmbH
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
(+49) 7031 463-2254

CAUTION

Federal law (USA) restricts this device to sale by or on the order of a physician.

Device Manufacturer

Philips Medical Systems
22100 Bothell Everett Highway
Bothell, WA 98021-8431, USA

For Technical Support

If you need technical support, please contact your local Philips representative or go to www.philips.com/AEDsupport.

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ENGLISH — Supplemental Information

The following information is intended to supplement the HeartStart FR3 Defibrillator user documentation:

- If you use *Bluetooth* with HeartStart Configure to set the FR3 clock, the FR3 time setting may be several seconds later than the computer clock.
- Although FR3 may indicate that a data card is invalid, patient use data recorded on the card may still be accessible. Do not discard the card without attempting to retrieve the patient data.
- The data card creates a folder each time the device is activated, whether or not there is an associated patient use. To ensure room for new patient use data, periodically download and delete data files.
- The 861394 charger begins the charging process when the temperature of any FR3 rechargeable battery is between 55° F and 104° F (13° C and 40° C).
- The FR3 language card is supplied with certain versions of the FR3. It is not available separately. To modify the default language of the FR3, or to enable the bilingual option and select a second language, use HeartStart Configure software, version 3.1 or higher.

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Niets uit deze uitgave mag in enige vorm of op enige manier worden verveelvoudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.
Philips Healthcare, a Division of Philips Electronics Ltd.
Philips Electronics Hong Kong Ltd.