



High image quality, dramatic dose reduction

Philips MicroDose Mammography system, US/Canada

PHILIPS
sense and simplicity

Enhancing patient care and business

By combining features that enhance image quality and speed with those that provide outstanding patient comfort and low dose, Philips MicroDose Mammography system can contribute to your business growth and productivity while helping you provide outstanding patient care.

Philips MicroDose Mammography provides diagnostic quality mammography images at low radiation dose. The use of a photon counting X-ray detector gives you outstanding image quality with 18 to 50 % lower radiation dose than used on other digital mammography systems, with an average dose reduction of 40%* 1,2,3,4

MicroDose delivers future-proof technology that will allow you to be competitive and deliver excellent patient care. Its exclusive photon counting detector technology opens up opportunities for advanced future breast applications.

We're there for you

Philips Healthcare Services supports you in delivering quality breast imaging. We help keep your screening and diagnostic solutions performing at their peak, motivate and educate your staff, and help drive your operational efficiency. Our service solutions help you to enhance your workflow efficiency while reducing costs, and are always focused on helping you deliver the best care possible.

Grow profits with MicroDose marketing

The considerable benefits possible with MicroDose give your business a strong competitive edge. Our MicroDose marketing program includes a variety of printed information for your waiting room, plus a powerful market launch package with news releases and marketing tools that help you boost awareness of the benefits of your MicroDose mammography system.

Dose matters

Breast tissue is sensitive to radiation. With Philips MicroDose, clinicians have achieved substantially lower radiation dose than with other mammography systems, without compromising image quality.


Designed to enhance comfort

MicroDose is designed to provide a less stressful mammography experience. The exam – including image acquisition – takes less than five minutes, offering speed and convenience that patients appreciate. The system also provides a curved patient support and a comfortably warm positioning surface. The ergonomic operation makes it easy for users to focus on patients and put them at ease. The result? Shorter examination times and high-patient throughput.



* The actual result of the average dose reduction will differ based on variations of digital mammography systems.

productivity

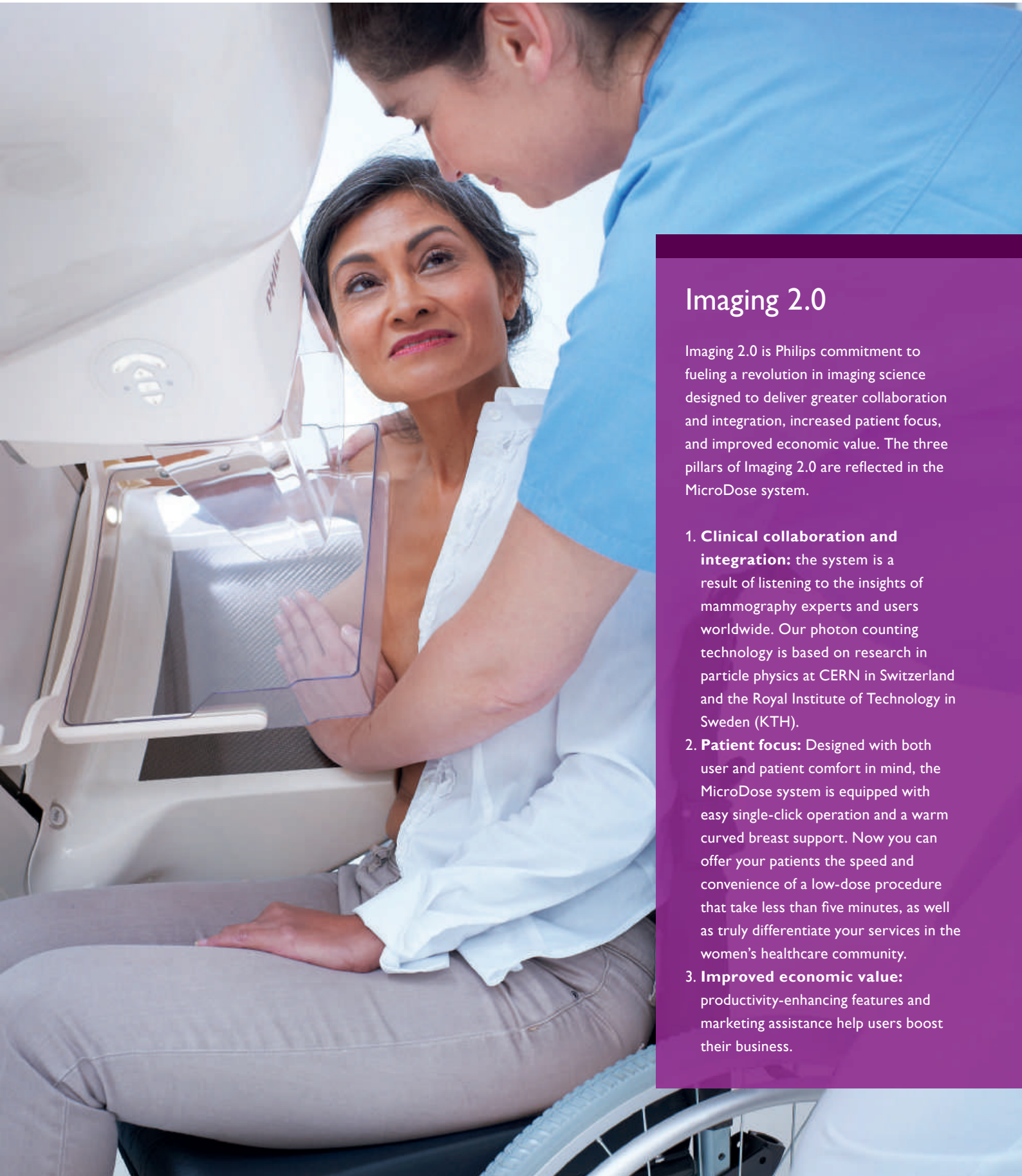

Imaging 2.0

Imaging 2.0 is Philips commitment to fueling a revolution in imaging science designed to deliver greater collaboration and integration, increased patient focus, and improved economic value. The three pillars of Imaging 2.0 are reflected in the MicroDose system.

- 1. Clinical collaboration and integration:** the system is a result of listening to the insights of mammography experts and users worldwide. Our photon counting technology is based on research in particle physics at CERN in Switzerland and the Royal Institute of Technology in Sweden (KTH).
- 2. Patient focus:** Designed with both user and patient comfort in mind, the MicroDose system is equipped with easy single-click operation and a warm curved breast support. Now you can offer your patients the speed and convenience of a low-dose procedure that take less than five minutes, as well as truly differentiate your services in the women's healthcare community.
- 3. Improved economic value:** productivity-enhancing features and marketing assistance help users boost their business.

Exclusive photon counting technology

The key factor in reducing breast cancer mortality rates is the early detection of small lesions. This requires highly effective equipment with superb image quality that increases diagnostic confidence.⁵

A number of factors, such as electronic noise, scattered radiation and information loss during analog-to-digital signal conversion can negatively affect image quality. To compensate, most FFDMs have to increase the radiation dose.

In contrast, MicroDose removes these factors almost entirely, using photon counting to facilitate confident detection at low radiation dose. Our patented detector counts the digital X-rays one by one, resulting in the following benefits:

- **No scattered radiation or electronic noise:** the image is acquired by a scanning method that uses a patented multi-slit collimator, eliminating 97% of scattered radiation.⁶ In addition, the signals from the individual photons are clearly separated from the background electronic noise. Together, these factors significantly reduce the noise level in the image, allowing for high dose efficiency.
- **No signal conversion and no data loss:** the direct capture of the individual X-rays occurs without the analog-to-digital conversion steps used by other FFDM systems, so no information is lost in conversion.

- **No ghost images:** because Philips photon counting detector is fast enough to be ready when the next photon arrives, there are no “ghost images” that can interfere with image interpretation or slow down your workflow.
- **50 µm pixel size:** a pixel size of 50 µm enables visualization of fine structures in the breast, such as microcalcifications and spiculations.
- **100% fill factor:** flat panel systems typically have part of the detector surface covered by read out electronics, leading to a waste of dose and active pixel area. Thanks to the design of the photon counting detector, 100% of the detector surface is active, putting every single photon to use.
- **No additional dose from scout images with SmartAEC:**⁷ SmartAEC continuously adjusts the radiation exposure depending on the density of the breast throughout the scan. This provides correct exposure without the need for scout images.

Photon counting gives you diagnostic quality images at up to 50% of the radiation dose used on other full-field digital mammography systems, as shown by ACR phantom testing.	Photon counting at glance	
	Detector technology	Photon counting
	Detector material	Crystalline silicon
	Dynamic resolution	1 photon
	Resolution	25 megapixels (4800 x 5200)
	Pixel size	50 µm
	Field of view	24 x 26 cm
	Fill factor	100%
	Pixel warranty	100%

yields high dose efficiency

100% pixel warranty

MicroDose's scanning technology eliminates dead pixels and associated image data loss, making faulty pixels and possible obscuring of microcalcifications a thing of the past. Photon counting gives you images you can trust.



Simplified and enhanced workflow

MicroDose is designed to fit the way you work.

The logical mobile choice

Mobile mammography brings breast examinations to women who might not otherwise have them. Philips MicroDose can be installed in almost any trailer, and the stable detector handles transport and storage temperatures from -10° to 50° C, providing year-round performance.

MicroDose in a mobile setting brings the same high quality, ergonomics and reliability as when it is in a stationary clinic, resulting in confidence and comfort for both the technologist and the woman.

Easy positioning

With MicroDose, positioning is easy and efficient.

- Scanning technology allows flexible positioning because the breast can be placed anywhere in the field of view (FOV).
- The system features a curved patient support and a comfortably warm positioning surface designed to relax patients.
- The optimal size of the FOV (24 x 26 cm) decreases the need to capture extra views for larger breasts.⁸ Small breasts are equally easy to position.

High-patient throughput

Equipped with one of the fastest and most efficient detectors on the market, MicroDose users perform as many as 15 four-image examinations per hour, and the system allows for even higher productivity if needed. By eliminating the need for scout images,⁷ SmartAEC also adds to the system's throughput efficiency.

Single click operations

1. Dedicated shortcut keypad to automatically control your workflow: angulations, selection of patients and users and even daily QA.
2. Easy access to rotation and height adjustment in four different places.
3. Motorized compression with foot pedals.
4. Isocentric rotation. No need for height adjustments between projections.
5. X-ray exposure from the acquisition workstation.
6. Foot switch for X-ray exposure offers you flexibility (optional).
7. Height adjustable table.



Dedicated compression paddles are available in the following sizes: standard, high edge, small, medium, spot.





**Philips Healthcare is part of
Royal Philips Electronics**

How to reach us

www.philips.com/healthcare
healthcare@philips.com

Asia
+49 7031 463 2254

Europe, Middle East, Africa
+49 7031 463 2254

Latin America
+55 11 2125 0744

North America
+1 425 487 7000
800 285 5585 (toll free, US only)

1. Oduko, J.M., Young, K.C., Burch, A., 2010. A Survey of Patient Doses from Digital Mammography Systems in the UK in 2007 to 2009. *Digital Mammography, IWDM 2010*, pp. 365–370.
2. Baldelli P, et. al., 2010. Comprehensive Dose Survey of Breast Screening in Ireland, *Radiation Protection Dosimetry*, 145(1), pp. 52–60.
3. Leitz W, Almén A., 2010. Patientdoser från röntgenundersökningar i Sverige – utveckling från 2005 till 2008. *Swedish Radiation Safety Authority, 2010-14*, ISSN 2000-0456, available online (in Swedish) at www.stralsakerhetsmyndigheten.se.
4. White paper, Comparison of Dose Levels in a National Mammography Screening Program, Philips Healthcare.
5. Keavey, E. et al., 2011. Comparison of the clinical performance of three digital mammography systems in a breast cancer screening programme. Published online before print November 17, 2011, *The British Institute of Radiology*, doi: 10.1259/bjr/29747759.
6. Åslund, M., Cederström, B., Lundqvist, M., Danielsson, M., 2006. Scatter rejection in multi-slit digital Mammography. *Medical Physics*, 33(4), pp.933–40.
7. Åslund, M. et al., 2005. AEC for scanning digital mammography based on variation of scan speed. *Medical Physics*, 32, p 11.
8. Hoffmeister, S., 2009. A comparison of digital mammography field of view detector sizes and the need for extra views. BSc DCR.

Please visit www.philips.com/microdose



© 2012 Koninklijke Philips Electronics N.V.
All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Printed in The Netherlands.
4522 962 86201 * MAY 2012