Interconnect Tubing for NIBP Cuffs

Instructions for Use

Tubing for Neonatal Cuffs

REF M1596B 1.5m Tubing

REF M1597B 3.0m Tubing

PHILIPS

About This Edition

Notice

Copyright © 2009

Koninklijke Philips Electronics N.V.
All rights are reserved.

Printing History

Current Edition/Revision:

Printed in Germany December 2009
Part Number 453564187771

Previous Edition/Revision:

Printed in Germany December 2007
Part Number 453564077031

Contents

Instructions for Use.....	1	Návod na obsluhu.....	31
Manuel d'utilisation	3	Navodila za uporabo	34
Gebrauchsanweisung	6	Instrucțiuni de utilizare	36
Gebruiksaanwijzing	8	Указания за употреба.....	38
Instrucciones de uso	10	Kasutusjuhend.....	41
Instruções de uso	12	Naudojimo instrukcija.....	43
Istruzioni d'uso	14	Lietošanas instrukcijas	45
Bruksanvisning	16	Инструкция по эксплуатации	
Bruksanvisning	18		47
Käyttöopas	20	Οδηγίες Χρήσης.....	50
Brugsanvisning	22	Kullanim Talimatları.....	53
Használati útmutató	24	ユーザーズ・ガイド.....	55
Instrukcja obsługi.....	26	使用説明.....	57
Návod k obsluze.....	29	使用说明书.....	59

USA Restriction

R_xonly United States law restricts this device to sale by or on the order of a physician.

Warnings

- Never connect intra-arterial or intra-venous lines, or any other Luer connectors, to the NIBP tubing. This can cause serious injury or death.
- These products are for use only in combination with Philips Patient Monitors when listed as accessories in the monitor's user documentation.

Cleaning

To keep tubing free of dust and dirt, clean with a lint-free cloth, moistened with either soap and warm water (40 °C/104 °F maximum), a diluted non-caustic detergent, or one of the approved cleaning agents listed below. Do not allow sustained contact with cleaners—remove immediately by wiping tube with a clean, water-dampened cloth. Never immerse or soak in any liquid. Fluid in tubing will damage monitoring equipment.

Deteriorated or damaged tubing should be replaced. Do not continue use for patient monitoring.

Recommended Cleaning Agents and Brands	
Soaps	mild soaps
Tensides	dishwasher detergents: Edisonite Schnellreiniger®, Alconox®
Ammonias	Dilution of Ammonia <3%, Window cleaner
Alcohol based	Ethanol 70%, Isopropanol 70%, Window cleaner

Disinfecting

Prior to disinfecting, tubing should first be cleaned (see *Cleaning*). To avoid long term damage to the tubing, disinfect only when necessary as specified by your facility. Do not allow sustained contact with disinfectants—remove immediately by wiping tube with a clean, water-dampened cloth. Never immerse or soak in any liquid. Fluid in tubing will damage monitoring equipment.

Recommended Disinfecting Substances	
Alcohol based	Ethanol 70%, Isopropanol 70%
Aldehyde based	Cidex [®]

Sterilizing

CAUTION: Do not autoclave the tubing or use bleaches containing sodium hypochlorite (e.g., Clorox[™]).

Prior to sterilizing, tubing should first be cleaned (see *Cleaning*). To avoid long term damage to the tubing, sterilize only when necessary as specified by your facility.

The tubing has been tested to withstand Ethylene Oxide (EtO) gas sterilization. Be sure that all safety precautions regarding aeration after EtO exposure are followed.

Ordering information

The tubing is for use with Philips monitoring equipment only. Orderable neonatal NIBP tubing includes:

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- **REF** M1596B (1.5 米)
- **REF** M1597B (3.0 米)

For more information, please call your local Philips sales office listed in your telephone directory or a Philips regional office listed below for the location of your nearest sales office.

United States:

Philips Medical Systems
North America Corporation
3000 Minuteman Road
Andover, MA 01810-1099
(800) 225-0230

Canada:

Philips Medical Systems Canada
281 Hillmount Road
Markham, ON
L6C 2S3
(800) 291-6743

Latin America Headquarters:

Philips Argentina S.A.
Depto Sistemas Medicos, Vedia 3892,
Casilla
3479, 1429 Buenos Aires
Tel: +54 11 4546 7699
Fax: +54 11 4546 7697

Europe, Middle East and Africa:

Philips Medizin Systeme
Böblingen GmbH
Hewlett-Packard Str. 2
71034 Böblingen, Germany
Tel: (+49) 7031 463 2254
Fax: (+49) 7031 463 1552

Asia Pacific Headquarters:

Philips Medical Systems
30/F Hopewell Centre
17 Kennedy Road
Wanchai
Hong Kong
Tel: (852) 2821 5888
Fax: (852) 2527 6727

<http://www.philips.com/healthcarestore>

Philips Medical Systems
3000 Minuteman Road
Andover, MA 01810-1099
USA
(800) 225-0230

GMDN: 15640

Manufactured in Germany

Subject to modification
Copyright © 2009
Koninklijke Philips Electronics N.V.
Printed in Germany December 2009
Part No. 453564187771