
Easy Care Reusable NIBP Cuffs

Instructions for Use

1-Tube Cuff	1-Tube Kit (5 Cuffs)	2-Tube Cuff	
REF	REF	REF	
M4552B	M4552B5	M4562B	10.0-15.0 cm
M4553B	M4553B5	M4563B	14.0-21.5 cm
M4554B	M4554B5	M4564B	20.5-28.5 cm
M4555B	M4555B5	M4565B	27.5-36.0 cm
M4556B	M4556B5	M4566B	27.5-36.0 cm
M4557B	M4557B5	M4567B	35.0-45.0 cm
M4558B	M4558B5	M4568B	35.0-45.0 cm
M4559B	M4559B5	M4569B	44.0-56.0 cm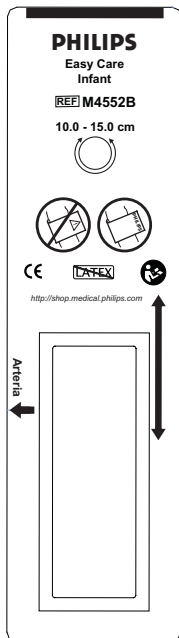

PHILIPS

About This Edition

Notice

Copyright © 2008

Koninklijke Philips Electronics N.V.
All rights are reserved.

Printing History

Current Edition/Revision:

Printed in USA February 2008
Part Number 453564085371

Previous Editions/Revisions:

None

Contents

Instructions for Use	1	Návod k obsluze	53
Manuel d'utilisation	5	Návod na obsluhu	57
Gebrauchsanweisung	9	Navodila za uporabo	61
Gebruiksaanwijzing	13	Instrucțiuni de folosire	65
Instrucciones de uso	17	Указания за употреба	69
Instruções de Uso	21	Kasutusjuhend	73
Istruzioni d'uso	25	Naudojimo instrukcija	77
Brugerhåndbok	29	Lietošanas instrukcijas	81
Bruksanvisning	33	Инструкция по	
Käyttöopas	37	эксплуатации	85
Brugsanvisning	41	Οδηγίες Χρήσης	89
Használati útmutató	45	Kullanım Talimatları	93
Instrukcja obsługi	49	使用説明	97
		使用说明书	101

United States law restricts this device to sale by or on the order of a physician.

Micropel[®] is a registered trademark of Troy Corporation.

Product Overview

Philips Easy Care, Reusable, Non-Invasive Blood Pressure (NIBP) Cuffs are available in single- or dual-tube configuration and accommodate a broad range of patient types and sizes (see *Applying Cuff to Patient*). The cuffs are latex- and PVC-free, color-coded for easy size selection, and designed to be easily cleaned and disinfected when needed.

Easy Care cuffs have a durable fabric cover treated with Micropel[®], an antimicrobial agent which prevents growth of fungal, bacterial, and algal organisms on the fabric, extends service life, and makes these cuffs ideal for emergency room and other rigorous multi-patient use environments. The mechanism of action of this agent does not contribute to the development of drug-resistant microbial strains. Reduction in colonization or microbial growth on the device has not been shown to correlate with a reduction in patient infections. Clinical studies to evaluate reduction in infection have not been performed.

Intended Use

Philips Easy Care NIBP Cuffs are to be used with identified devices intended for use by, or under the supervision of, a licensed physician or other healthcare provider for the non-invasive measurement of human blood pressure.

Compatibility

Easy Care MxxxxB cuffs are compatible with the same Philips, Agilent, and Hewlett-Packard NIBP systems that list Philips model MxxxxA cuffs as an accessory; e.g. M4555B replaces M4555A, etc. The cuffs are also compatible with many other manufacturers' NIBP systems. Consult your sales representative or visit <http://shop.medical.philips.com> for the most recent list of compatible NIBP monitoring systems.

Definition of Product Symbols

					
Proper Application	Improper Application	Patient Limb Circumference Range	Refer to Instruction Manual/Booklet	Contains No Latex	Caution Consult Instructions

Warnings

- Periodically inspect and replace damaged or deteriorated cuffs.
- Replace cuff if hook and loop fastener fails to hold during inflation.
- Use cuff only under direct supervision by a trained healthcare professional when attached to automated monitors without fail-to-read alarms.
- Prevent damage to cuff when used on monitors with auto-BP functions by shutting off auto-function, or removing cuff from monitor when it is not applied to patient.
- Avoid placing cuff on patient in a manner that can lead to a cuff tube becoming kinked (tube kinking may cause inaccurate readings).
- Be sure cuff is correctly matched to patient type and size (limb circumference ranges are printed on each cuff). Cuffs that are too small may produce false high blood pressure readings; cuffs that are too large may produce false low pressure readings.
- Easy Care NIBP Cuffs should be cleaned when soiled, but never sterilized (refer to *Cleaning and Low Level Disinfection*).


Applying Cuff to Patient

Before applying cuff to a patient, read and understand all Warnings in the instruction manuals for both your NIBP monitor and cuffs.

- Using the guide below, select a cuff that accommodates your monitor type (single- or dual-tube) and your patient's limb circumference. Appropriate limb circumference size ranges are printed on each cuff.

 (cm)	10.0–15.0 Infant	14.0–21.5 Pediatric	20.5–28.5 Small Adult	27.5–36.0 Adult	27.5–36.0 Adult Long	35.0–45.0 Large Adult	35.0–45.0 Large Adult Long	44.0–56.0 Thigh
Cuff Color	Orange	Green	Blue	Navy	Navy	Burgundy	Burgundy	Grey
Single-tube	M4552B	M4553B	M4554B	M4555B	M4556B	M4557B	M4558B	M4559B
Dual-tube	M4562B	M4563B	M4564B	M4565B	M4566B	M4567B	M4568B	M4569B


- Place open cuff to arm (or thigh) such that the side with the Philips logo is *not* in contact with the patient. Align arrow (↓) labeled **Arteria** to the brachial (or femoral) artery. Wrap cuff snugly around arm or thigh.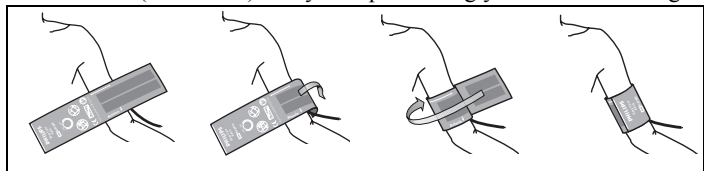

- Verify that the index line (located along the top of the Philips logo) falls within the size range arrow, then fasten.

- Connect cuff tubing to instrument hose:

- Single-tube.* Join tube and hose connectors. Push until connectors snap together.
- Dual-tube.* Screw connectors into mating connectors on double-lumen hose.

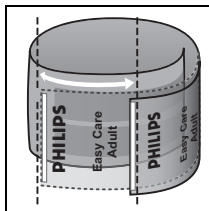Cleaning and Low-Level Disinfection

Easy Care cuffs may be cleaned and disinfected, but never sterilized.

Cleaning Warnings

- Use only validated cleaning agents and disinfectants (listed below) in accordance with labeled instructions. Use of non-listed cleaners may damage the cuff, shorten product life, or cause safety hazards.
- Do not sterilize cuffs. Do not use bleach.
- Wash and rinse temperatures should not exceed 40° C.
- Prior to cleaning, place end-cap (reorder part numbers: single-tube—9898 031 51601; dual-tube—9898 031 51611) on connector(s) to prevent fluid from entering tubing. Fluid in tubing will damage monitoring equipment.

Cleaning and Disinfecting

Philips Easy Care reusable cuffs have been determined to withstand cleaning with the classes of cleaners listed below when used in accordance with labeled instructions.

Anionic cleaners (e.g., *Alconox*)

Isopropyl alcohol 70%

Quaternary ammonium cleaners (e.g., *Steris Coverage Plus NPD*)

Mild soap and water

Enzymatics (e.g., *Metrazyme*)

Consult your sales representative or visit <http://shop.medical.philips.com> for the most recent list of tested cleaners and disinfectants.

Cuff Disposal

Any cuff that becomes damaged or deteriorated should be discarded. Always follow approved disposal methods for medical waste as specified by your facility (hospital, clinic, etc.) or local regulations. (Note: Philips Easy Care Cuffs do not contain PVC.)

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For more information, please call your local Philips sales office listed in your telephone directory or a Philips regional office listed below for the location of your nearest sales office.

United States:

Philips Medical Systems
North America Corporation
3000 Minuteman Road
Andover, MA 01810-1099
(800) 225-0230

Canada:

Philips Medical Systems Canada
281 Hillmount Road
Markham, ON
L6C 2S3
(800) 291-6743

Latin America Headquarters:

Philips Medical Systems
1550 Sawgrass Corporate Parkway #300
Sunrise, FL 33323 USA
Tel: (954) 835-2600
Fax: (954) 835-2626

 This product complies with the requirements of the Council Directive 93/42/EEC June 1993 (Medical Device Directive).

Cuffs made in USA

US Status:

Manufactured for Philips Medical Systems

Manufacturing Address:

Philips Medical Systems
3000 Minuteman Road
Andover, MA 01810-1099
USA
(800) 225-0230

Europe, Middle East and Africa:

Philips Medizin Systeme
Böblingen GmbH
Hewlett-Packard Str. 2
71034 Böblingen, Germany
Fax: (+49) 7031 463 1552

Asia Pacific Headquarters:

Philips Medical Systems
30/F Hopewell Centre
17 Kennedy Road
Wanchai
Hong Kong
Tel: (852) 2821 5888
Fax: (852) 2527 6727

Contains no Latex

<http://shop.medical.philips.com>

Subject to modification

Copyright © 2008
Koninklijke Philips Electronics N.V.
Printed in USA February 2008

Part No. 453564085371