
Reusable NIBP Comfort Cuffs

Instructions for Use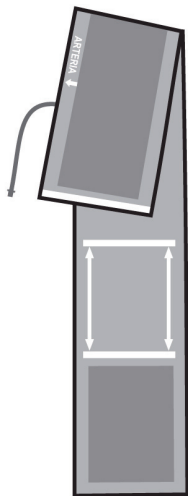

REF M1571A 	REF M1572A 
REF M1573A 	REF M1574A 
REF M1575A 	REF M1576A 

PHILIPS

About This Edition

Notice

Copyright © 2007

Koninklijke Philips Electronics N.V.
All rights are reserved.

Printing History

Current Edition/Revision:

Printed in Germany December 2007
Part Number 453564077021

Contents

Instructions for Use.....	1	Návod na používanie.....	29
Manuel d'utilisation.....	3	Navodila za uporabo	31
Gebrauchsanweisung	5	Instrucțiuni de utilizare	33
Gebruiksaanwijzing	7	Указания за употреба	35
Instrucciones de uso	9	Kasutusjuhend.....	37
Instruções de Uso	11	Naudojimo instrukcija.....	39
Istruzioni d'uso	13	Lietošanas instrukcijas.....	41
Brugerhåndbok.....	15	Инструкция по эксплуатации	43
Bruksanvisning	17	Οδηγίες Χρήσης	45
Käyttöohjeet.....	19	Kullanim Talimatları.....	47
Brugerhåndbog.....	21	ユーザーズ・ガイド.....	49
Használati útmutató	23	使用説明.....	51
Instrukcja obsługi.....	25	使用说明书.....	53
Návod k obsluze.....	27		

USA Restriction

United States law restricts this device to sale by or on the order of a physician.

Warnings

- Use cuff only under direct supervision by a trained healthcare professional when attached to automated monitors without fail-to-read alarms.
- This product is for use only in combination with Philips instruments when listed as accessories in the instrument use documentation.
- Liquids in the tube or air bladder will damage the monitor. Place end-cap (reorder number: 9898 031 51601) on connector prior to cleaning to prevent fluid from entering tubing. Never immerse the air bladder or tube in any liquid.

When to Replace Cuff

Replace the cuff if it is damaged, clogged, or visibly contaminated.

Replace the cuff if the monitor displays a message that a leak has been detected.

Definition of Product Symbols

Limb circumference




Correct cuff application

Incorrect cuff application

Instructions

1. Place open cuff to arm (or thigh) such that the side with the Philips logo is *not* in contact with the patient. Align arrow (▼) labeled **ARTERIA** to the brachial (or femoral) artery.
2. Wrap the cuff snugly around arm or thigh (see below).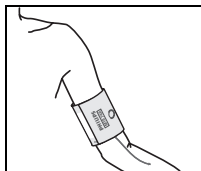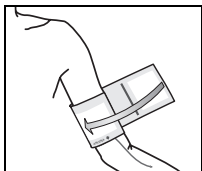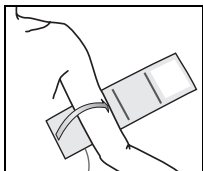

3. Verify white line at the end of cuff closest to tubing falls within the size range lines (| |) printed on the cuff interior. If it does not, select a cuff that better accommodates the limb circumference.

Cleaning and Disinfecting

Wipe cuffs with a solution of mild soap and water. If the cover requires more rigorous cleaning, remove the air bladder first. Allow the cover to thoroughly air dry before use.

Cuffs have been tested to withstand the following recommended disinfectants: Cidex, Sporicidin, Microzid, Isopropyl-alcohol 70%, Ethanol 70% Buraton liquid.

Sterilizing

Cuffs have been tested to withstand Ethylene Oxide (EtO) gas sterilization. Be sure to follow all aeration safety precautions following EtO exposure. *Do not autoclave.*

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For more information, please call your local Philips sales office listed in your telephone directory or a Philips regional office listed below for the location of your nearest sales office.

United States:

Philips Medical Systems
North America Corporation
3000 Minuteman Road
Andover, MA 01810-1099
(800) 225-0230

Canada:

Philips Medical Systems Canada
281 Hillmount Road
Markham, ON
L6C 2S3
(800) 291-6743

Latin America Headquarters:

Philips Medical Systems
1550 Sawgrass Corporate Parkway #300
Sunrise, FL 33323 USA
Tel: (954) 835-2600
Fax: (954) 835-2626

Philips Medical Systems
3000 Minuteman Road
Andover, MA 01810-1099
USA
(800) 225-0230

Europe, Middle East and Africa:

Philips Medizin Systeme
Böblingen GmbH
Hewlett-Packard Str. 2
71034 Böblingen, Germany
Fax: (+49) 7031 463 1552

Asia Pacific Headquarters:

Philips Medical Systems
30/F Hopewell Centre
17 Kennedy Road
Wanchai
Hong Kong
Tel: (852) 2821 5888
Fax: (852) 2527 6727

<http://shop.medical.philips.com>

GMDN: 34978

Manufactured in Germany

Subject to modification
Copyright © 2007
Koninklijke Philips Electronics N.V.
Printed in Germany December 2007
Part No. 453564077021

+H1794535640770213-