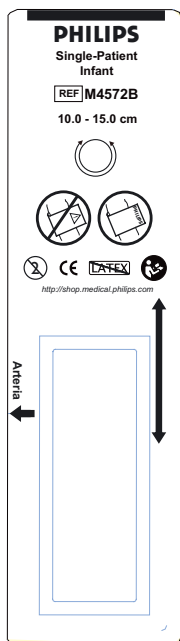
Single-Patient NIBP Cuffs

Instructions for Use

1-Tube Cuffs

REF	
M4572B	
M4573B	
M4574B	
M4575B	
M4576B	
M4577B	
M4578B	
M4579B	

2-Tube Cuffs

M4582B	10.0-15.0 cm
M4583B	14.0-21.5 cm
M4584B	20.5-28.5 cm
M4585B	27.5-36.0 cm
M4586B	27.5-36.0 cm
M4587B	35.0-45.0 cm
M4588B	35.0-45.0 cm
M4589B	44.0-56.0 cm

PHILIPS

About This Edition

Notice

Copyright © 2008

Koninklijke Philips Electronics N.V.
All rights are reserved.

Printing History

Current Edition/Revision:

Printed in USA October 2008
Part Number 453564121921

Previous Editions/Revisions:

Printed in USA January 2008
Part Number 453564083261

Contents

Instructions for Use	1
Manuel d'utilisation	4
Gebruiksaanwijzing	7
Gebruiksaanwijzing	10
Instrucciones de uso	13
Instruções de Uso	16
Istruzioni d'uso	19
Brugerhåndbok	22
Bruksanvisning	25
Käyttöohjeet	28
Brugerhåndbog	31
Használati útmutató	34
Instrukcja obsługi	37

Návod k obsluze	40
Návod na používanie	43
Navodila za uporabo	46
Instrucțiuni de utilizare	49
Указания за употреба	52
Kasutusjuhend	55
Naudojimo instrukcija	58
Lietošanas instrukcijas	61
Инструкция по эксплуатации	64
Οδηγίες Χρήσης	67
Kullanım Talimatları	70
ユーザーズ・ガイド	73
使用僧明	76
使用说明书	79

United States law restricts this device to sale by or on the order of a physician.

Product Overview

Philips Single-Patient Non-Invasive Blood Pressure (NIBP) Cuffs are available in single- or dual-tube configuration and accommodate a broad range of patient types and sizes (see *Applying Cuff to Patient*). The cuffs are latex- and PVC-free, color-coded for easy size selection, and designed for single-patient use only.

All of these MxxxxB cuffs are compatible with the same Philips, Agilent, and Hewlett-Packard NIBP systems that list Philips model MxxxxA cuffs as an accessory; e.g. M4575B replaces M4575A, etc. The cuffs are also compatible with many other manufacturers' NIBP systems. Consult your sales representative or visit <http://shop.medical.philips.com> for the most recent list of compatible NIBP monitoring systems.

Intended Use

Philips Single-Patient NIBP Cuffs are to be used with identified devices intended for use by, or under the supervision of, a licensed physician or other healthcare provider for the non-invasive measurement of human blood pressure.

Definition of Product Symbols

						
Proper Application	Improper Application	Patient Limb Circumference Range	Refer to Instruction Manual/Booklet	Single-Patient Use Only	Contains No Latex	Caution Consult Instructions

Warnings

- Periodically inspect and replace damaged or deteriorated cuffs.
- Replace cuff if hook and loop fastener fails to hold during inflation.
- Use cuff only under direct supervision by a trained healthcare professional when attached to automated monitors without fail-to-read alarms.
- Prevent damage to cuff when used on monitors with auto-BP functions by shutting off auto-function, or removing cuff from monitor when it is not applied to patient.
- Avoid placing cuff on patient in a manner that can lead to a cuff tube becoming kinked (tube kinking may cause inaccurate readings).
- Be sure cuff is correctly matched to patient type and size (limb circumference ranges are printed on each cuff). Cuffs that are too small may produce false high blood pressure readings; cuffs that are too large may produce false low pressure readings.
- Single-patient cuffs are not intended for use on more than one patient.
- Single-patient cuffs are not intended to be cleaned or disinfected and if soiled should be discarded.

Cuff Disposal

Always follow approved methods for disposal of medical waste as specified by facility (hospital, clinic, etc.) or local regulations. **Note that Philips Single-Patient NIBP Cuffs do not contain PVC.**


Applying Cuff to Patient

Before applying to a patient, read and understand all Warnings in the instruction manuals for both your NIBP monitor and cuffs.

1. Using the guide below, select a cuff that accommodates monitor type and limb circumference. Size ranges are printed on each cuff.


 (cm)	10.0–15.0 Infant	14.0–21.5 Pediatric	20.5–28.5 Small Adult	27.5–36.0 Adult	27.5–36.0 Adult Long	35.0–45.0 Large Adult	35.0–45.0 Large Adult Long	44.0–56.0 Thigh
Cuff Text Color	Orange	Green	Blue	Navy	Navy	Burgandy	Burgandy	Grey
Single-tube	M4572B	M4573B	M4574B	M4575B	M4576B	M4577B	M4578B	M4579B
Dual-tube	M4582B	M4583B	M4584B	M4585B	M4586B	M4587B	M4588B	M4589B

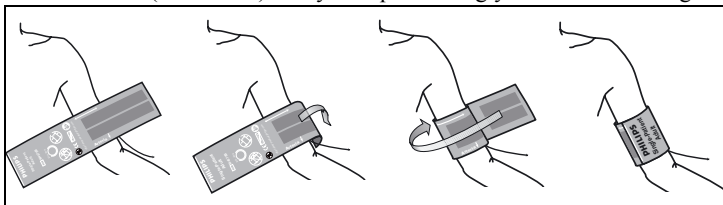2. Place open cuff to arm (or thigh) such that the side with the Philips logo is *not* in contact with the patient. Align arrow (↓) labeled **Arteria** to the brachial (or femoral) artery. Wrap cuff snugly around arm or thigh.

3. Verify index line (located above Philips logo) falls within the size range arrow, then fasten.
4. Connect cuff tubing to instrument hose:

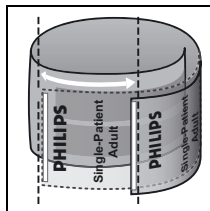- *Single-tube.* Join tube and hose connectors. Push until connectors snap together.
- *Dual-tube.* Screw connectors into mating connectors on double-lumen hose.

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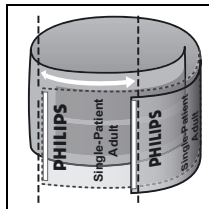
For more information, please call your local Philips sales office listed in your telephone directory or a Philips regional office listed below for the location of your nearest sales office.

United States:

Philips Healthcare
North America Corporation
3000 Minuteman Road
Andover, MA 01810-1099
(800) 225-0230

Canada:

Philips Healthcare Canada
281 Hillmount Road
Markham, ON
L6C 2S3
(800) 291-6743

Latin America Headquarters:

Philips Healthcare
1550 Sawgrass Corporate Parkway #300
Sunrise, FL 33323 USA
Tel: (954) 835-2600
Fax: (954) 835-2626

 This product complies with the requirements of the Council Directive 93/42/EEC June 1993 (Medical Device Directive).

Cuffs made in USA

US Status:
Manufactured for
Philips Healthcare

Manufacturing Address:
Philips Healthcare
3000 Minuteman Road
Andover, MA 01810-1099
USA
(800) 225-0230

Europe, Middle East and Africa:

Philips Medizin Systeme
Böblingen GmbH
Hewlett-Packard Str. 2
71034 Böblingen, Germany
Fax: (+49) 7031 463 1552

Asia Pacific Headquarters:

Philips Healthcare
30/F Hopewell Centre
17 Kennedy Road
Wanchai
Hong Kong
Tel: (852) 2821 5888
Fax: (852) 2527 6727

<http://www.philips.com/healthcarestore>

Contains no Latex

Subject to modification

© 2008
Koninklijke Philips Electronics N.V.
Printed in USA October 2008

Part No. 453564121921-L-IFU Rev A