


Philips Medical Systems

***DICOM***

CONFORMANCE STATEMENT

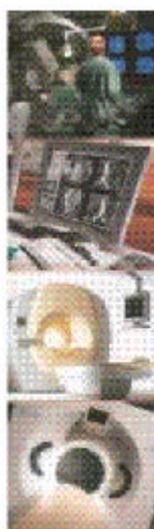

**EnVisor B.0**

2003-10-29

Copyright Philips Medical Systems Nederland B.V. 2002  
All rights reserved

 **PHILIPS**

*Let's make things better.*

**COPYRIGHT KONINKLIJKE PHILIPS ELECTRONICS N.V. 2003.**

ALL RIGHTS RESERVED. NO PART OF THIS PUBLICATION  
MAY BE REPRODUCED OR TRANSMITTED IN ANY FORM OR  
BY ANY MEANS, ELECTRONIC OR MECHANICAL, INCLUDING  
PHOTOCOPY, RECORDING, OR ANY INFORMATION STORAGE  
AND RETRIEVAL SYSTEM, WITHOUT PRIOR WRITTEN  
PERMISSION FROM THE COPYRIGHT OWNER.

© PHILIPS MEDICAL SYSTEMS NEDERLAND B.V. 2003

# Table of Contents

<b>0. INTRODUCTION</b> .....	<b>4</b>
<b>0.1 PURPOSE OF THIS DOCUMENT</b> .....	<b>4</b>
<b>0.2 INTENDED AUDIENCE</b> .....	<b>5</b>
<b>0.3 OVERVIEW OF DICOM PRODUCT OFFERING</b> .....	<b>6</b>
<b>0.4 SOURCES FOR THIS DOCUMENT</b> .....	<b>8</b>
<b>0.5 IMPORTANT NOTE TO THE READER</b> .....	<b>9</b>
<b>0.6 ACRONYMS, ABBREVIATIONS AND GLOSSARY OF TERMS</b> .....	<b>10</b>
<b>1. IMPLEMENTATION MODEL</b> .....	<b>16</b>
<b>1.1 APPLICATION DATA FLOW DIAGRAMS</b> .....	<b>19</b>
1.1.1 NETWORK RELATIONSHIPS .....	19
1.1.2 REMOVABLE MEDIA RELATIONSHIPS.....	22
<b>1.2 FUNCTIONAL DEFINITION OF ENVISOR AE</b> .....	<b>23</b>
1.2.1 STORAGE OF DICOM STUDIES TO A PACS.....	24
1.2.2 ISSUING OF STORAGE COMMITMENT REQUESTS TO AN IMAGE MANAGER .....	25
1.2.2.1 Batch Mode and Manual Export .....	25
1.2.2.2 Send-As-You-Go Mode .....	26
1.2.3 VERIFICATION OF THE EXISTENCE OF DICOM SERVER ON THE HOSPITALS NETWORK.....	27
1.2.4 PRINTING DICOM STUDIES TO A B&W OR COLOR PRINTER.....	28
1.2.4.1 Monochromization and Intelli-print .....	28
1.2.4.2 Batch Mode and Manual Print.....	28
1.2.4.3 Send-As-You-Go Mode .....	29
1.2.5 RESPONDING TO A VERIFICATION REQUEST FROM A REMOTE DICOM SERVER .....	29
1.2.6 SAVING A DICOM STUDY TO REMOVABLE MEDIA.....	30
1.2.7 READING A DICOM STUDY FROM REMOVABLE MEDIA .....	30
1.2.8 WRITING AND READING A STUDY TO/FROM REMOVABLE MEDIA IN NON-DICOM FORMATS.....	31
1.2.9 OBTAINING A LIST OF SCHEDULED WORK FROM THE HIS VIA THE MWL SERVER.....	31
1.2.10 UPDATING THE STATUS OF A SCHEDULED PROCEDURE USING THE MPPS SERVER .....	32
<b>1.3 SEQUENCING OF REAL-WORLD ACTIVITIES</b> .....	<b>32</b>
<b>2. APPLICATION ENTITY SPECIFICATIONS</b> .....	<b>33</b>
<b>2.1 ENVISOR AE SPECIFICATION</b> .....	<b>33</b>
2.1.1 ASSOCIATION ESTABLISHMENT POLICIES .....	33
2.1.1.1 General .....	33
2.1.1.2 Number of Associations .....	34
2.1.1.3 Asynchronous Nature .....	35
2.1.1.4 Implementation Identifying Information .....	36
2.1.2 ASSOCIATION INITIATION BY REAL-WORLD ACTIVITY .....	37
2.1.2.1 Storage of DICOM studies to a PACS .....	37

2.1.2.2	Issuing of Storage Commitment requests to an image manager .....	82
2.1.2.3	Verification of the existence of DICOM server on the hospitals network .....	87
2.1.2.4	Printing DICOM studies to a B&W or color printer.....	91
2.1.2.5	Obtaining a list of scheduled work from the HIS via Modality Worklists .....	112
2.1.2.6	Updating the status of a scheduled procedure .....	123
2.1.3	ASSOCIATION ACCEPTANCE POLICY .....	132
2.1.3.1	Responding to a verification request from a remote DICOM server .....	132
<b>3.</b>	<b>ENVISOR AS A MEDIA STORAGE APPLICATION.....</b>	<b>135</b>
3.1	FILE META INFORMATION FOR THE ENVISOR AE.....	135
3.2	REAL-WORLD ACTIVITIES.....	136
3.2.1	SAVING A DICOM STUDY TO REMOVABLE MEDIA .....	136
3.2.2	READING A DICOM STUDY FROM REMOVABLE MEDIA .....	138
3.2.2.1	Import DICOM Object Specifications .....	139
3.2.2.2	Media Storage Application Profiles .....	141
3.2.2.3	SOP Specific Conformance For "DICOM Directory" SOP Class.....	142
3.2.2.4	File-Set Identification Module.....	143
3.2.2.5	Directory Information Module .....	144
<b>4.</b>	<b>COMMUNICATIONS PROFILES .....</b>	<b>149</b>
<b>5.</b>	<b>EXTENSIONS/SPECIALIZATIONS/PRIVATIZATIONS .....</b>	<b>150</b>
5.1	2D.....	150
5.2	3D STILLs AND LOOPS, AND PANVIEW.....	150
<b>6.</b>	<b>CONFIGURATION.....</b>	<b>152</b>
<b>7.</b>	<b>SUPPORT FOR EXTENDED CHARACTER SETS .....</b>	<b>153</b>
7.1	SUPPORT FOR RUSSIAN AND JAPANESE MARKETS.....	154
7.2	ADDITIONAL SUPPORT FOR JAPANESE MARKETS.....	154
7.3	SUPPORT FOR CHINESE MARKETS.....	155

## 0. Introduction

This chapter provides general information about the purpose, scope and contents of this Conformance Statement.

### 0.1 Purpose of this Document

Context: Expansion of Headings and sub-headings

- Introduction
  - Purpose of this Document

The Digital Imaging and Communications in Medicine (DICOM) standard was originally developed by a joint committee of the American College of Radiology (ACR) and the National Electrical Manufacturers Association (NEMA) to

*“Facilitate the open exchange of information between digital imaging computers”.*

It specifies how medical images and related clinical information are passed between medical devices.

The DICOM Conformance Statement (DCS) is a required document for any device that claims conformance to DICOM. Since the DICOM standard specifies the structure and content of this document (PS3.2 - 2001) a DCS describes the DICOM capabilities and key features of a particular product in a standardized, defined manner.

This DCS defines the DICOM capabilities and key features of Philips Medical Systems' EnVisor ultrasound imaging system.

For a hospital's Information Technology (IT) department, matching DICOM Conformance Statements between vendor product offerings is a key element to determine interconnectivity between vendors' devices.

This Conformance Statement should be read in conjunction with the DICOM standard and its addenda [DICOM].

## 0.2 Intended Audience

Context: Expansion of Headings and sub-headings

- Introduction
- Intended Audience

This Conformance Statement is intended for:

- Potential customers
- System integrators of medical equipment
- Marketing staff interested in system functionality
- Software designers implementing DICOM interfaces

It is assumed that the reader is familiar with the DICOM standard.

### 0.3 Overview of DICOM product offering

Context: Expansion of Headings and sub-headings

- Introduction
  - Overview of DICOM product offering

The services supported by EnVisor’s DICOM subsystem are derived from the following customer needs:

Name	Customer Need	Options package
Optioning	Ability to purchase some features and not others.	N/A
Archival	Archival of digital images to: <ol style="list-style-type: none"> <li>1. Removable media or</li> <li>2. Across the network</li> </ol>	<ol style="list-style-type: none"> <li>1. DICOM Media</li> <li>2. DICOM Basic</li> </ol>
Printing of medical images	Printing to a DICOM compatible printer.	DICOM Basic
Verification	Ability to verify the existence of and communicate with a DICOM server on the network.	DICOM Basic
Modality WorkList (MWL)	Ability to obtain lists of patients and procedures from the hospital’s information system.	DICOM Advanced
Modality Performed Procedure Step (MPPS)	Ability to update the information in the hospital’s information system with regard to the status of a scheduled procedure.	DICOM Advanced

The base EnVisor system will be sold with no DICOM services enabled. Customers requiring functionality beyond that provided by the base system purchase DICOM services as options on top of the base system.

Philips Medical Systems offers customers three DICOM options:

1. DICOM Media      Capability to read/write studies from/to a floppy, CD, or MOD.
2. DICOM Basic      Capability to store studies across a network, transfer ownership of studies to the PACS and print a hardcopy to a DICOM printer.
3. DICOM Advanced      Capability to request lists of scheduled work from the hospital's information system and the ability to update study status information in the hospital's information system.

While the DICOM Conformance Statement is not intended to be a complete EnVisor product specification, some areas of this document will refer to system operation where it is necessary to add a context for the discussion or to help explain a capability.

## 0.4 Sources for this Document

Context: Expansion of Headings and sub-headings

- Introduction
  - Sources for this Document

The source for this document is:

- American College of Radiology-National Electrical Manufacturers Association (ACR-NEMA) Digital Imaging and Communications in Medicine (DICOM) V3.0. 2001

## 0.5 Important Note to the Reader

Context: Expansion of Headings and sub-headings

- Introduction
  - Important Note to the Reader

This Conformance Statement by itself does not guarantee successful interoperability of Philips equipment with non-Philips equipment. The user (or user's agent) should be aware of the following issues:

### Interoperability

Interoperability refers to the ability of application functions, distributed over two or more systems, to work successfully together. The integration of medical devices into an IT environment may require application functions that are not specified within the scope of DICOM. Consequently, using only the information provided by this Conformance Statement does not guarantee interoperability of Philips equipment with non-Philips equipment. It is the user's responsibility to analyze thoroughly the application requirements and to specify a solution that integrates Philips equipment with non-Philips equipment.

### Validation

Philips equipment has been carefully tested to assure that the actual implementation of the DICOM interface corresponds with this Conformance Statement. Where Philips equipment is linked to non-Philips equipment, the first step is to compare the relevant Conformance Statements. If the Conformance Statements indicate that successful information exchange should be possible, additional validation tests will be necessary to ensure the functionality, performance, accuracy and stability of image and image related data. It is the responsibility of the user (or user's agent) to specify the appropriate test suite and to carry out the additional validation tests.

### New versions of the DICOM Standard

The DICOM Standard will evolve in future to meet the user's growing requirements and to incorporate new features and technologies. Philips is actively involved in this evolution and plans to adapt its equipment to future versions of the DICOM Standard. In order to do so, Philips reserves the right to make changes to its products or to discontinue its delivery. The user should ensure that any non-Philips provider linking

to Philips equipment also adapts to future versions of the DICOM Standard. If not, the incorporation of DICOM enhancements into Philips equipment may lead to loss of connectivity (in case of networking) and incompatibility (in case of media).

## 0.6 Acronyms, Abbreviations and Glossary of Terms

Context: Expansion of Headings and sub-headings

- Introduction
  - Acronyms, Abbreviations and Glossary of Terms

DICOM definitions, terms and abbreviations are used throughout this Conformance Statement. For a description of these, see NEMA PS 3.3 and PS 3.4.

ACC-----	American College of Cardiology
ACR-----	American College of Radiology Initiated the DICOM standardization effort in the early 1980's.
AE -----	Application Entity A software process that implements DICOM. EnVisor uses a single AE.
AE Title-----	AE's require a unique 'AE Title', typically set up during installation through EnVisor's setup key on the control panel. Application Entities (AE's) identify themselves to each other via the AE Title at application level.
ANSI -----	American National Standard Institute
Association-----	A connection between AE's for DICOM exchange.
ASCII-----	American Standard Code for Information Exchange Known for standardizing codes for text.
Attribute-----	Attributes are the components of an object (IOD), describing its properties. Examples of attributes are Patient Name, Patient ID etc. Each attribute has a unique DICOM tag.
Big Endian -----	An encoding method for storing the most significant Byte in the high order bits of a 16 bit word. Motorola based CPU's expect information to be stored this way.

Exchanged as part of the transfer syntax. Conversion to little endian would require swapping each byte within the words.

- CD-R----- Compact Disk, Read Only  
An option for the physical specification for the DICOM media exchange standard and used by EnVisor as a removable media device.
- DICOM ----- Digital Imaging and Communications In Medicine  
Version 3.0 is the current defined version and is that used by this in this document.
- DICOM Media----- A DICOM option that can be purchased by the customer, it allows the user to write DICOM study's to removable media.
- DICOM Basic ----- A DICOM option that can be purchased by the customer, it allows the user to perform network export of DICOM study's and DICOM print. It includes DICOM Media.
- DICOM Advanced ----- A DICOM option that can be purchased by the customer; it allows the user to select a procedure from a Modality Worklist and to send study status information to the department scheduler. It includes DICOM Basic and DICOM Media.
- DICOMDIR ----- The standard directory structure specified for DICOM media exchange.
- DIMSE----- DICOM Message Service Element.  
The DICOM set of commands (e.g. C\_ECHO, C\_STORE, etc.)
- EnVisor ----- Philips Medical Systems' EnVisor ultrasound system.
- Explicit VR ----- Explicit Value Representation  
A transfer syntax which is negotiated by which the receiver is explicitly told the VR syntax and context. This is in contrast to Implicit VR.
- FSC----- File-set creator  
The function of a device that creates DICOM exchange media, typically an acquisition device such as an ultrasound. EnVisor is a FSC.

FSR-----	File-set reader The function of a device that reads DICOM exchange media - typically a viewing station. EnVisor is also a FSR and can view studies created by other EnVisor systems.
FSU-----	File-set updater The function of a device that can read DICOM exchange media and update the DICOMDIR.
Implementation Class UID	A unique number, which is exchanged during the set up of the association, by which EnVisor identifies itself. This UID is guaranteed not to change for a specific release of EnVisor.
Implicit VR-----	Implicit Value Representation A transfer syntax which is negotiated by which the receiver is assumed to know the VR syntax and context.
HIS -----	Hospital Information System
IOD -----	Information Object Definition Specification of a DICOM object such as a US Image Object.
ISO-----	International Standards Organization
JPEG -----	An image compression technique created by the Joint Photographic Experts Group
Little Endian -----	An encoding method by which the least significant byte is stored in the high order bits in a 16 bit word. This is how Intel based CPU's store data.
LUT-----	Look Up Table Specifying mapping from specific values such as pixel values into luminance.
MOD -----	Magneto Optical Disk Used by EnVisor as one of the physical exchange media options for the DICOM exchange standard.
Modality-----	An acquisition system such as CT, MR, US. EnVisor is an ultrasound (US) modality.

Module -----	A group of attributes, specified as a unit for convenience reasons. Examples are Patient module, Study module etc.
MPPS -----	Modality Performed Procedure Step Used by EnVisor for informing a department scheduler of the status of a study.
MWL -----	Modality WorkList Used to provide on the EnVisor system a Worklist of scheduled procedures.
NEMA -----	National Electrical Manufacturers Association US trade organization, members of which defined the first version of the DICOM standard together with the ACR.
PACS-----	Picture Archiving and Communications System An image archive.
PaletteColor-----	A color format for images in which the pixel values are indices into a color table. See RGB and YBR for other color formats.
PDU-----	Protocol Data Unit Packet that is created at the DICOM lower-level protocol.
Pixel-----	Smallest, single element or dot of an image.
RGB-----	A color format for images in which the pixel values contain the red, green and blue color intensities. See PaletteColor and YBR for other color formats.
RIS-----	Radiology Information System Which typically schedules and maintains patient demographic information.
RLE -----	Run Length Encoding A lossless image compression scheme.
SCP-----	Service Class Provider DICOM AE that functions as a server or 'provides' a service such as Storage, Print etc.
SCU-----	Service Class User DICOM AE that functions as a client, or uses a service, i.e. for printing, storage etc.

Service Class -----	A service class is a group of one or more SOP classes e.g. the Storage Service class contains all the storage SOP classes (CT_STORE, US_STORE etc).
SOP-----	Service Object Pair Combination of a service such as US_STORE and an object such as image.
TCP/IP -----	Transmission Control Protocol/Internet Protocol The communication standard supported by DICOM.
Transfer Syntax -----	Encoding specification of DICOM messages, negotiated while setting up an association. Examples of different transfer syntaxes are Little Endian or Big Endian, Implicit or Explicit VR, or a compression scheme (such as RLE or JPEG).
Type-----	Specification of rule for whether an attribute has to be present in an object. Type 1 attributes are required; Type 2 are required but can be left blank when unknown; Type 3 are optional.
U/U -----	Usage specification for a specific service, meaning (user-) optional for SCU and mandatory for SCP.
U/M-----	Usage specification for a specific service, meaning (user-) optional for both SCU and SCP.
UID -----	Unique Identifier A world-wide unique numbering scheme which is used by the NEMA to, for example, identify SOP classes, syntaxes etc and vendors for identifying SOP instances.
US -----	Ultrasound
VM-----	Value Multiplicity Defining whether or not an attribute can have multiple elements, for example multiple phone numbers.
VR -----	Value Representation The definition of rules and encoding of groups of similar attributes. For example the VR Person Name (PN) specifies exactly the sequence of last name, first name etc.

- YBR ----- A color format for images in which the pixel values contain one luminance and two chrominance planes. See PaletteColor and RGB for other color formats.
- 3D/PanView----- Philips Medical System's EnVisor Fetal-3D and Panoramic View options

## 1. Implementation Model

Context: Expansion of Headings and sub-headings

- Implementation Model

This section describes the functional relationship between the device and the DICOM services:

Customer Need	Provided in options package	Functionality	DICOM Service Classes Required
Optioning	Bundled	Ability to install/remove optional features	
Archive to Media	Bundled	Saving BMP's, AVI's, and HTML docs to media	
		Formatting removable media (floppy, MOD)	
	DICOM Media	Saving DICOM studies to removable media.	Media Storage Service Class – File Set Creator Media Storage Service Class – File Set Updater
Retrieval from Media	DICOM Media	Reading DICOM studies from removable media	Media Storage Service Class – File Set Reader

Customer Need	Provided in options package	Functionality	DICOM Service Classes Required
Archive to Network	DICOM Basic	Network export of DICOM studies.	Storage SCU
		Transfer ownership of acquired images to an image management system.	Storage Commitment SCU
Print	Bundled	Print images to PC based printers, non-DICOM film printers.	
	DICOM Basic	Print studies to a DICOM printer – both color and B&W.	Print Management SCU
Modality Worklist (MWL)	DICOM Advanced	Request modality worklists from the Modality Worklist Server.	MWL SCU
Modality Performed Procedure Step (MPPS)	DICOM Advanced	Inform the hospital of the status of a performed procedure.	MPPS SCU
Setup	DICOM Basic	Verification that a network device is a DICOM server.	Verification SCU


Customer Need	Provided in options package	Functionality	DICOM Service Classes Required
		Response to requests from the network to verify that EnVisor is a DICOM device.	Verification SCP
		Set the AE Title for EnVisor; Specify which network server is the storage SCP, storage commit SCP; List servers, add servers etc	

## 1.1 Application Data Flow Diagrams

### 1.1.1 Network Relationships

The diagram in Figure 1.1-1 represents the relationship between EnVisor's Application Entity and its use of DICOM to real-world activities. Figure 1.1-1 shows the relationships for DICOM activities on the network.

Figure 1.1-1 Network Relationships

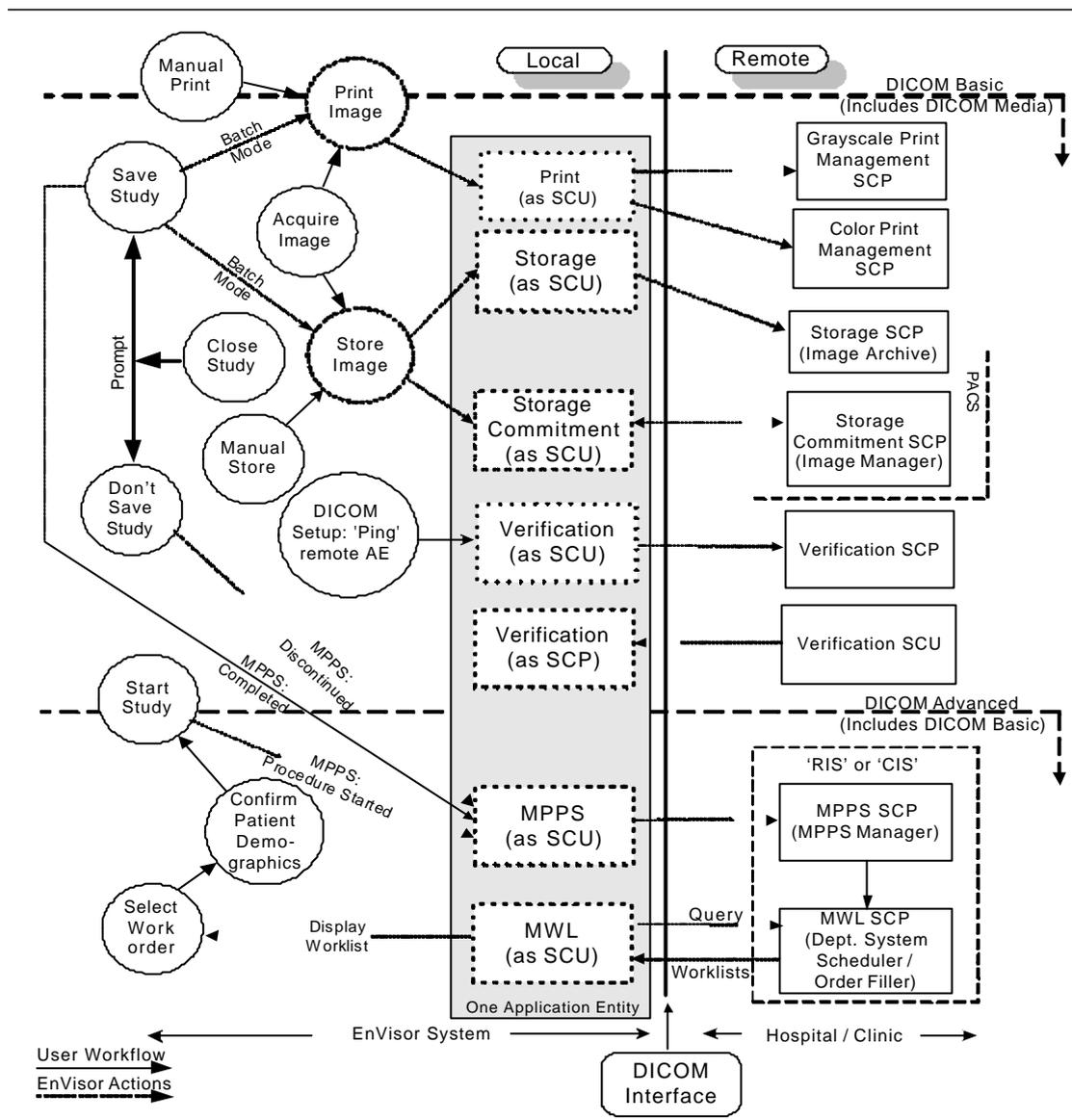

The left side of the diagram (labeled 'Local') represents the EnVisor system being described in this DICOM Conformance Statement. The right side (labeled 'Remote') represents equipment that EnVisor is meant to exchange information with (the Hospital/Clinic), and the vertical line in between is the DICOM Interface.

The long rectangular gray box represents the one and only Application Entity that is used in the implementation of all EnVisor's DICOM services. This single AE supports all the EnVisor services: print, storage, storage commitment, verification, MWL and MPPS.

Since an AE must have a unique AE Title across a hospital's network (HIS), the user can configure the AE's title through setup. The dotted rectangular boxes within the Application Entity represent the various DICOM services used (SCU) and supported (SCP).

The circles, on the left side of the diagram, represent real-world activities that a user can perform with the EnVisor system, such as saving a study and acquiring an image.

The diagram shows that EnVisor supports storing images to a remote PACS, as well as transferring ownership of the images to an image manager so that the study can automatically be deleted from EnVisor's hard-drive.

Images can be:

1. Sent to the Storage SCP as soon as they are acquired (circle 'Acquire Image'), this is called send-as-you-go mode.
2. Batched up and sent all at once each time the study is saved (circle 'Save Study').
3. A study can be selected, by the user, from a list of studies on EnVisor's local hard-drive, and manually exported (circle 'Manual Store').

EnVisor also supports printing studies to a grayscale or color DICOM printer. As with storing studies to a remote PACS, images can be printed as soon as they are acquired (but only when there are enough to fill a page); they can be batched for printing all at once when the study is saved, or the study can be selected manually for printing.

If color images are sent to a grayscale printer, they will be converted to grayscale. If both a color and grayscale printer is configured, color images will be routed to the color printer and grayscale images will be routed to the grayscale printer.

EnVisor supports Modality Worklists (MWL) and Modality Performed Procedure Step (MPPS.) These two capabilities work together to allow EnVisor to communicate with a Hospital Information System (HIS) to obtain and display lists of patients scheduled for imaging procedures and to update the HIS whenever the status of a scheduled procedure changes (for example, when a scheduled study is completed.)

The diagram shows that when a study is started (circle 'Start Study'), EnVisor sends an MPPS Study Started message to the department system scheduler (MPPS SCP) and when the user finishes the study a MPPS Study Completed message is sent to the department system scheduler (circle 'Save Study'). It also shows that the user can discontinue a study (circle 'Don't Save Study'), in which case a MPPS Study Discontinued message is sent to the department system scheduler.

The user can disconnect the network cable and use EnVisor in walk-about or portable mode. When reconnected to the network, EnVisor will process any queued jobs including storage, printing and Storage Commitment. Queued MPPS status updates will also be performed. Also, on reconnect, EnVisor resumes the periodic retrieval of the modality worklist.

### 1.1.2 Removable Media Relationships

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Application Data Flow Diagrams
    - Removable Media Relationships

The diagram in Figure 1.1-2 represents the relationship between EnVisor's Application Entity and it's use of DICOM to real-world activities. Figure 1.1-2 shows the relationships for DICOM activities involving local storage to removable media.

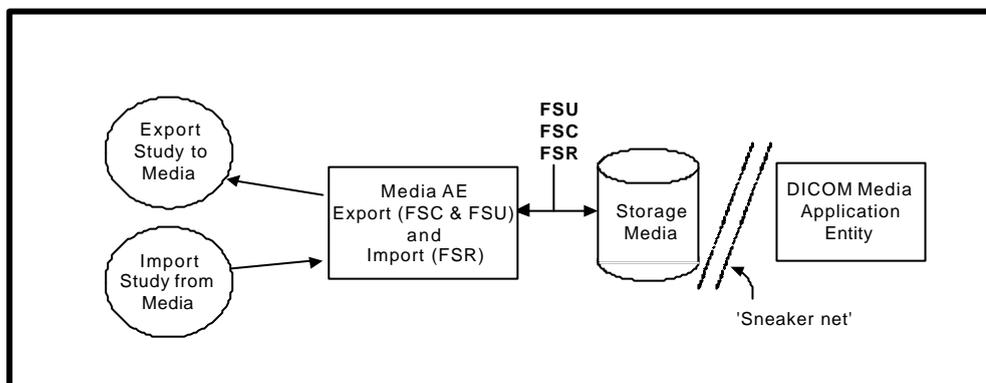


Figure 1.1-2 Removable Media Relationships

As with the previous diagram, circles represent real-world activities performed on the EnVisor system.

The diagram shows that EnVisor supports the writing of DICOM study's to the ultrasound systems removable media (floppy diskette, CD or MOD). This is useful for exporting the study's to the image archive when the hospitals network is down. It is also useful for long term archival to CD of study's for sites that have not purchased the networking capability provided in the 'Advanced' package.

EnVisor can also read back into the system study's that it (or another EnVisor system) have previously written to removable media. However, since EnVisor is not an image archive but an image modality, it will not allow a user to read study's into the system that were not generated by an EnVisor system.

## 1.2 Functional Definition of EnVisor AE

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE

EnVisor is implemented as a single AE. The DICOM AE Title and Port number are configurable by the user through the 'Setup' screens. The default AE Title that EnVisor will use is the host name of the computer. Since AE Titles must be unique across a hospital's network and computer names must also be unique, some institutions institute a policy where the AE Title is derived from the computer name. EnVisor supports this by allowing the user to specify a fixed string for a prefix and suffix. The AE Title is then generated from the prefix, the computer's name and the suffix.

The default port number is 104 but as with the AE Title, the port number can be configured by the user.

There are ten real-world activities that the EnVisor AE performs. These are:

1. Storage of DICOM studies to a PACS,
2. Issuing of Storage Commitment requests to an image manager,
3. Verification of the existence of DICOM servers on the hospital's network,
4. Printing DICOM studies to a B&W or color printer,
5. Responding to a verification request from a remote DICOM server,
6. Saving a DICOM study to removable media,
7. Reading a DICOM study from removable media,
8. Writing and Reading a study to/from removable media in Non-DICOM formats
9. Obtaining a list of scheduled work from the HIS via the MWL Server, and
10. Updating the HIS whenever a scheduled procedure changes using the MPPS Server.

These real-world activities are described, in general terms, in the following sub-sections.

### 1.2.1 Storage of DICOM studies to a PACS

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Storage of DICOM studies to a PACS

EnVisor acts as a Service Class User (SCU) of the 'Ultrasound Image Store' SOP Class using DIMSE C-STORE commands to transmit images to the storage server. It provides a set of DICOM configuration settings used to set up the network interface and storage options. The configurable options include specification of the DICOM storage server (hostname, port number and AE Title). These options can be accessed through the DICOM Setup screen.

Just before the first image is sent from the system, the storage AE establishes an association with the storage SCP and maintains the open association as long as images for storage are in the queue to that SCP. If the queue empties, the storage AE will close the association. This process will repeat for subsequent images. Therefore, images sent quickly one after the other would share the same association. This reduces overhead and improves performance. Therefore

- In Batch Mode, where all the images are sent to the storage SCP when the user closes (and saves) the study, all the images will be sent on the same association.
- In send-as-you-go mode, where the images are sent one-at-a-time as the user acquires them, images acquired quickly one after the other would be sent on the same association, however if the user has a delay between acquiring one image and acquiring the next image then each image would be sent on a separate association.

If any images in a study are not successfully stored to the Storage SCP, then the study is marked with an icon indicating failure in the 'Search for Study' screen. If the user subsequently manually exports the study, all the images will be resent to the Storage SCP (both those that were previously successfully stored and those that failed). The Storage SCP will detect, without detrimental consequences (per the DICOM standard), that some images are duplicates.

## 1.2.2 Issuing of Storage Commitment requests to an image manager

EnVisor serves as a Storage SCU and sends images to a remote DICOM PACS.

### 1.2.2.1 Batch Mode and Manual Export

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Issuing of Storage Commitment requests to an image manager
      - Batch Mode and Manual Export

If the user has configured, through DICOM setup, a Storage Commitment server, then after the last image of the study is stored to the storage SCP, EnVisor will generate an N-Action to request Storage Commitment by the Storage Commitment SCP of all the images that were successfully stored. Storage Commitment will not be requested for images that the Storage SCP rejected.

The N-Action command contains a list of image transaction UIDs. EnVisor then closes the association and waits for a reply from the STORAGE COMMITMENT server. Some time later, the Storage Commitment SCP will open an association with EnVisor's AE using reverse-role negotiation, and will send an NEvent Report with a list of the image transaction UIDs that were successfully committed and if applicable, a list of those that were not.

EnVisor will reject an association requested by a Storage Commitment SCP that does not employ role-reversal.

### 1.2.2.2 Send-As-You-Go Mode

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Issuing of Storage Commitment requests to an image manager
      - Send-As-You-Go Mode

If the user has configured, through DICOM setup, a Storage Commitment server, then after each image of the study is successfully stored to the storage SCP, EnVisor will generate an N-Action to request Storage Commitment of the image by the Storage Commitment SCP.

Therefore, in send-as-you-go mode, where the images are sent one-at-a-time as the user acquires them, multiple N-Action requests are generated and sent to Storage Commitment SCP. One N-Action request corresponds to one image.

The N-Action command contains the transaction UID of the just acquired image. EnVisor then, as with batch mode, closes the association and waits for a reply from the STORAGE COMMITMENT server.

### 1.2.3 Verification of the existence of DICOM server on the hospitals network

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Verification of the existence of DICOM server on the hospitals network

When the user configures one of the SCP servers (for example the Storage SCP or B&W printer SCP), he/she can optionally 'ping' the SCP to verify it is a DICOM server, it is on-line and it is enabled to communicate with this EnVisor system.

When the user requests a 'DICOM Ping', the verification SCU will initiate an association with the remote server and send a C-Echo request to the server.

## 1.2.4 Printing DICOM studies to a B&W or color printer

EnVisor serves as a print SCU and sends images to a remote DICOM print device.

### 1.2.4.1 Monochromization and Intelli-print

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Printing DICOM studies to a B&W or color printer
      - Monochromization and intelli-print

The operator can configure up to two print SCPs: one B&W and one COLOR. If only a B&W print SCP is configured, then color images will be converted to grayscale. If both B&W and color print SCP's are defined then EnVisor uses an "intelli-print" process to send color images to the color SCP and grayscale images to the B&W SCP.

### 1.2.4.2 Batch Mode and Manual Print

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Printing DICOM studies to a B&W or color printer
      - Batch Mode and Manual Print

In Batch Mode, where all the images are printed when the user closes (and saves) the study, all the images will be sent on the same association. Each page will contain the configured number of images. The last page may be a partial page if there are not enough images to fill the page; this ensures that a printed page cannot have images from multiple studies

### 1.2.4.3 Send-As-You-Go Mode

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Printing DICOM studies to a B&W or color printer
      - Send-As-You-Go Mode

In send-as-you-go mode, as images are acquired they are held until a full page of images is ready for printing. When a full page of images is ready for printing, EnVisor will open an association with the printer, send the images and then close the association.

When the study is closed (or Saved), any partially filled page is printed. As with Batch Mode, this ensures that a printed page cannot have images from multiple studies

### 1.2.5 Responding to a verification request from a remote DICOM server

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Responding to a verification request from a remote DICOM server

The ultrasound system employs a Verification SCP to reply to verification requests sent by remote devices. This will allow the remote devices to ensure the availability of EnVisor on the network, within the constraints of the network topology, and timeout values.

EnVisor will respond to a C-Echo request even if a customer has not purchased any DICOM options.

Some DICOM systems will only respond to C-Echo requests from DICOM Servers that it knows about. EnVisor will respond to a GEcho requests from DICOM systems that are not on its list of known DICOM servers.

### 1.2.6 Saving a DICOM study to removable media

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Saving a DICOM study to removable media

EnVisor is a DICOM file set creator (FSC) and updater (FSU). Studies can be saved (exported) to EnVisor's removable media (MOD, CD-R or floppy disk), in DICOM format, for long-term storage. Also, if a customer chooses not to purchase DICOM Basic, then DICOM media can be used as a 'sneaker-net' to get DICOM studies off EnVisor and onto the PACS.

### 1.2.7 Reading a DICOM study from removable media

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Reading a DICOM study from removable media

EnVisor is a DICOM file set reader (FSR). Studies that EnVisor has saved to removable media may also be loaded into another EnVisor system or even into the same EnVisor system (as long as the original study has already been deleted). Since EnVisor is not an image review station, it will check the originator of the study and only import studies created by another EnVisor system.

### 1.2.8 Writing and Reading a study to/from removable media in Non-DICOM formats

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Writing and Reading a study to/from removable media in Non-DICOM formats

Users that do not purchase DICOM Media can write the images in DICOM studies to removable media as bmp's and AVI's. They can also write a study to removable media in an EnVisor proprietary format that can then be read back into (the same) or another EnVisor system. This format, however, is not DICOM and cannot be read by non-EnVisor systems.

### 1.2.9 Obtaining a list of scheduled work from the HIS via the MWL Server

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Obtaining a list of scheduled work from the HIS via the MWL Server

EnVisor acts as a Service Class User (SCU) of the 'Modality Worklist (MWL)' SOP Class using DIMSE C-FIND commands to retrieve lists of scheduled protocols (imaging sessions) from the HIS.

A set of standard MWL queries is available (e.g. Show today's worklist entries, show today's worklist entries assigned to this system, show yesterday, today and tomorrow's worklist entries etc.). The user can also configure their own queries based on start date, AE Title of performing EnVisor etc.

The current work lists can be retrieved manually (when the EnVisor system is connected to the network) or automatically polled in the background.

### 1.2.10 Updating the status of a scheduled procedure using the MPPS Server

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Functional Definition of EnVisor AE
    - Updating the status of a scheduled procedure using the MPPS Server

EnVisor also acts as a Service Class User (SCU) of the 'Modality Performed Procedure Step (MPPS)' SOP Class.

The start procedure message (N-CREATE) is sent when the user presses the OK button on the Patient ID Window to bring up live imaging. The MPPS Server is also notified, with a N-SET command, when the study is completed (when the study is saved to EnVisor's disk and closed), or when it is discontinued (when the study is closed without saving.)

### 1.3 Sequencing of Real-World Activities

Context: Expansion of Headings and sub-headings

- Implementation Model
  - Sequencing of Real-World Activities

For printing and storing using the Print Gray Image, Print Color Image, and Store Image commands, the user must have previously completed the Patient ID screen (which creates a study). For accessing and updating procedures scheduled by the HIS, the EnVisor user must first select a patient from the Patient Selection screen which displays a list of patients scheduled for procedures on EnVisor.

## 2. Application Entity Specifications

EnVisor is implemented as a single AE.

### 2.1 EnVisor AE Specification

#### 2.1.1 Association Establishment Policies

##### 2.1.1.1 General

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Establishment Policies
      - General

The following Application Context Name will be proposed and recognized by EnVisor:

- DICOM 3.0 Application Context 1.2.840.10008.3.1.1.1

EnVisor contains no limitations for maximum PDU size. The PDU size is configurable with a minimum size of 100. The default PDU size is 100,000.

### 2.1.1.2 Number of Associations

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Establishment Policies
      - Number of Associations

EnVisor **establishes** one association per destination at a time. The total number of associations possible at one time is six: 1 B&W printer, 1 Color printer, 1 Storage Server, 1 Storage Commitment server, 1 MWL server, and 1 MPPS server.

EnVisor accepts simultaneous associations for Storage Commitment and Verification. If multiple servers issue a Storage Commitment or verification request at the same time, EnVisor will accept all the associations. The maximum number of simultaneous associations **accepted** by EnVisor is limited only by resource constraints.

### 2.1.1.3 Asynchronous Nature

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Establishment Policies
      - Asynchronous Nature

EnVisor allows a single outstanding operation on any association. Therefore, EnVisor does not support asynchronous operations window negotiation, other than Storage Commitment reverse-role negotiation for N-Event Report's.

#### 2.1.1.4 Implementation Identifying Information

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Establishment Policies
      - Implementation Identifying Information

Element	Implementation Value
Implementation Class UID	1.2.840.113543.6.6.3.2
Implementation Version Name	EnVisor_B.0

**Table 1: Implementation Identifying Information**

Note: This Class UID and Version Name will be used for the release of EnVisor described in this DICOM Conformance Statement. The Class UID and Version Name may be updated with subsequent EnVisor releases to capture the version level of the new release, however the Implementation Version Name for the EnVisor product will always start with the characters “EnVisor” and the Implementation Class UID will always be of the form 1.2.840.113543.6.6.n.n

## 2.1.2 Association Initiation by Real-World Activity

### 2.1.2.1 Storage of DICOM studies to a PACS

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS

The EnVisor provides standard conformance to the following DICOM V3.0 SOP Classes as an SCU:

SOP Class Name	SOP Class UID	Role
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	SCU
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	SCU

**Table 2: SOP Classes Supported by Network Storage AE**

### 2.1.2.1.1 Associated Real-World activity

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Associated Real-World activity

The real world activities that will trigger EnVisor to initiate an association with the Storage Server is dependent on the mode of operation:

#### 1. In Manual mode

An association is initiated when the user selects a study from the list of studies on EnVisor's local hard-drive and requests that the selected study be exported to the PACS.

#### 2. In Send-As-You-Go mode

An association is initiated when the first image is acquired. It is kept open while images remain to be stored. If no images have been stored within its time-out window, EnVisor will close the association. A new association will be initiated when the next image is acquired. The association will be closed when the study is closed.

#### 3. In Batch mode

An association is initiated whenever the user saves the study. The images that have been acquired since the previous 'save' are stored to the PACS. To enhance network performance images that have already been stored to the PACS are not resent.

Store Association Negotiation - Association Status (Send-As-You-Go mode)

User Action	DICOM Activity – Store Send As You Go Mode
First image acquired from system	Association Negotiation (but will reuse current Association if queue to that Store SCP is non-empty) + C-Store until queue to that SCP is empty when Association Release Request is sent.

Store Association Negotiation - Association Status (Batch Mode & Manual Mode)

User Action	DICOM Activity – Store
Save Study	Association Negotiation then C-Store until all images sent, then Association Release Request is sent.

The user can also configure the photometric interpretation and transfer syntax of the image pixel data so that EnVisor's images can be viewed with a wide range of DICOM viewers.

For a manual export to removable media (or the PACS) the user can specify that the images be sent to the Storage SCP in one of five formats:

- Palette Color (pixels are indices into a color palette), RLE compressed
- Palette Color (pixels are indices into a color palette), uncompressed (little endian, implicit VR)
- RGB (pixels are intensities of Red, Green and Blue color components), RLE compressed
- RGB (pixels are intensities of Red, Green and Blue color components), uncompressed (little endian, implicit VR)
- YBR FULL 422, (pixels are described by one luminance and two chrominance planes, sampled with twice as a much luminance as chrominance), JPEG compressed.

For automatic export (batch or send-as-you-go) to a networked PACS, the user can select the same photometric interpretations available in manual export (Palette Color, RGB or YBR), however...

- If the user has selected Palette Color or RGB as the photometric interpretation, EnVisor will first try to negotiate the transfer using RLE compression. If the Storage SCP does not support this transfer syntax, EnVisor will try to negotiate the transfer using the Little Endian Implicit VR transfer syntax. All PACS are mandated, by the standard, to accept 'Little Endian, Implicit VR'.
- If the user has selected YBR\_FULL\_422 as the photometric interpretation, EnVisor will try to negotiate the transfer using JPEG compression. If the images cannot be sent as YBR JPEG compressed, EnVisor will report an error condition to the user in the 'Search for Studies' screen.

Table 3 describes the behavior of the Network Storage AE in response to various error conditions and C-STORE-RSP status indicators. After all images in the study have been processed, the association is closed.

### Establishing the association

Condition (After C-Store)	Status Codes (C-Store-RSP)	Response
<p>Could not establish the association within 30-second time window (Connect Timeout) due to NO RESPONSE from the Storage Server</p>	<p>Not Applicable</p>	<p>The association attempt is aborted, and after 5-minutes a new association is attempted. EnVisor will make three attempts to open an association with the configured Storage SCP before aborting the storage request and placing the job in an error state. The user can then manually restart the job at some later date. The failure is logged to the DICOM log file as an error.</p> <p>The 5-minute timeout and the number of retries are configurable by the user from the DICOM Setup screens. The 5-minute timeout is mapped to the 'Retry Interval' input control on the 'DICOM Setup screen and the number of retries is mapped to 'Maximum Retries' on the DICOM Setup screen.</p>
<p>Refused</p>	<p>A7xx</p>	<p>If the Storage SCP server refuses the association, then the association attempt is aborted. EnVisor will wait 5-minutes and then reattempt the association. EnVisor will make three attempts to establish the association before aborting the storage request and placing the job in an error state. The user can then manually restart the job at some later date. The failure is logged to the DICOM log file as an error.</p> <p>As an example, the association would be refused if the storage server employs a high security mechanism whereby it only accepts association requests from DICOM Servers that it knows about and the EnVisor's AE Title was not in the PACS database.</p> <p>The 5-minute timeout and the number of retries are</p>

Condition (After C-Store)	Status Codes (C-Store-RSP)	Response
		configurable by the user from the DICOM Setup screens. The 5-minute timeout is mapped to the "Retry Interval" input control on the DICOM Setup screen and the retry is mapped to 'Maximum Retries' on the DICOM Setup Screen.

**During image transfer**

Condition (After C-Store)	Status Codes (C-Store-RSP)	Response
After association has been accepted, there is no response to a request within 5-minute time window (Read Timeout).	Not Applicable	<p>If the association is lost during active image transfer to the Storage SCP server, EnVisor will initiate a new association after 5 minutes, and attempt to store all the images. If during transfer, the association is again lost, EnVisor will wait another 5 minutes and try again. EnVisor will make three attempts to send all the images before aborting the storage request and placing the job in an error state. The user can then manually restart the job at some later date. The failure is logged to the DICOM log file as an error.</p> <p>The 5-minute timeout and the number of retries are configurable by the user from the DICOM Setup screens. The 5-minute timeout is mapped to the "Retry Interval" input control on the DICOM Setup screen and the retry is mapped to 'Maximum Retries' on the DICOM Setup Screen.</p>
Error	A9xx, Cxxx,	If the Storage SCP rejects an image, then EnVisor will keep a record of the image that failed and will continue with the remaining images. The failure is

<b>Condition</b> (After C-Store)	<b>Status Codes</b> (C-Store-RSP)	<b>Response</b>
	0122, Other	<p>logged to the DICOM log file as an error. Once all the images have been attempted, EnVisor will close the association and wait for 5-minutes. The job, as viewable by the user from the job manager, will be in the 'retry' status.</p> <p>After it's 'retry interval' has expired, EnVisor will move the job to 'in progress', establish another association and try storing the images that failed in the earlier association. Successfully stored images are not re-exported, only those that earlier failed.</p> <p>EnVisor will make three attempts to complete the transfer of all the images. If, after three attempts, some images are still not successfully stored to the PACS, EnVisor will notify the user (through an icon on the list of studies) and the job will be placed into the 'error' state.</p> <p>The user can, at some later date, select the study for manual export. EnVisor will give the user the option of storing all the images or just those that did not make it in the earlier job.</p> <p>The 5-minute timeout and the number of retries are configurable by the user from the DICOM Setup screens. The 5-minute timeout is mapped to the "Retry Interval" input control on the DICOM Setup screen and the number of retries is mapped to 'Maximum Retries' on the DICOM Setup Screen.</p>
Warning	D000, B000, B006, B007 0111	<p>If the Storage SCP issues a warning on a particular image (perhaps it had to use coercion, EnVisor logs the warning to the DICOM log file as an informational event and continues on as if the image was successfully stored to the PACS (see row below).</p>

Condition (After C-Store)	Status Codes (C-Store-RSP)	Response
Success	0000	When an image is successful store to the Storage SCP (PACS), EnVisor will keep a record of the successful storage. If all the images in the job are successfully stored, EnVisor will notify the user (through an icon on the list of studies). And the job will be removed from the job manager. The successful storage will be recorded in the DICOM log file as a service level event.

**Table 3: Responses to Image Storage Error Conditions**

Note that in “Send As You Go” mode, one association is established for one study, and closed when all images of the study currently existing in the EnVisor system are stored. If more images of the same study are presented to the EnVisor system, additional associations will be initiated to transfer the remaining images using the same Study and Series Instance UIDs.

### 2.1.2.1.2 Proposed Presentation Context

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS

#### 1. Proposed Presentation Context

Each time the Network Storage service initiates an association in response to the store request, it requests services summarized in Table 4.

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Ultrasound Multi-frame Image Storage SOP Class	1.2.840.100 08.5.1.4.1.1. 3.1	JPEG baseline (Proposed if photometric interpretation is YBR_FULL_422)	1.2.840.100 08.1.2.4.50	SCU	None
		RLE Lossless (Proposed for Palette Color and RGB)	1.2.840.100 08.1.2.5		
		DICOM Implicit VR Little Endian (Used only if JPEG or RLE lossless is declined)	1.2.840.100 08.1.2		

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Ultrasound Image Storage SOP Class	1.2.840.10008.5.1.4.1.1.6.1	JPEG baseline (Proposed if photometric interpretation is YBR_FULL_422)	1.2.840.10008.1.2.4.50	SCU	None
		RLE Lossless (Proposed for Palette Color and RGB)	1.2.840.10008.1.2.5		
		DICOM Implicit VR Little Endian (Used only if JPEG or RLE lossless is declined)	1.2.840.10008.1.2		

**Table 4: Transfer Syntaxes**

The values of certain image attributes used in the transfer of each image depend on the photometric interpretation (as configured by the user) and the type of the image.

Table 5 describes the relationships among these parameters.

		Resultant Attribute Values				
Image Format (configured by user)	Image Type	Photometric Interpretation (0028,0004)	Samples Per Pixel (0028,0002)	Bits allocated (0028,0100)	Rows (0028,0010)	Columns (0028,0011)
Palette Color	2D B&W Image <sup>1</sup>	PALETTE COLOR	1	8	564	800
Palette Color	2D color Image	PALETTE COLOR	1	16	564	800
Palette Color	2D B&W loop	PALETTE COLOR	1	8	564	800
Palette Color	2D color loop	PALETTE COLOR	1	16	564	800
Palette Color	3D single frame	RGB	3	8	528	720
Palette Color	3D multi frame	RGB	3	8	528	720
Palette Color	Panview	RGB	3	8	528	720
Palette Color	Report	PALETTE COLOR	1	16	564	800
RGB	2D B&W Image	RGB	3	8	564	800
RGB	2D color Image	RGB	3	8	564	800

<sup>1</sup> 2D B&W Image include "Colorized" images, which map a sample to a color instead of a gray scale value.

RGB	2D B&W loop	RGB	3	8	564	800
RGB	2D color loop	RGB	3	8	564	800
RGB	3D single frame	RGB	3	8	528	720
RGB	3D multi frame	RGB	3	8	528	720
RGB	Panview	RGB	3	8	528	720
RGB	Report	RGB	3	8	564	800
YBR	2D B&W Image	YBR_FUL L_422	3	8	564	800
YBR	2D color Image	YBR_FUL L_422	3	8	564	800
YBR	2D B&W loop	YBR_FUL L_422	3	8	564	800
YBR	2D color loop	YBR_FUL L_422	3	8	564	800
YBR	3D single frame	RGB	3	8	528	720
YBR	3D multi frame	RGB	3	8	528	720
YBR	Panview	RGB	3	8	528	720
YBR	Report	YBR_FUL L_422	3	8	564	800

**Table 5: Image Attributes based upon Photometric Interpretation and Image Type**

Notes:

1. The bits allocated (0028,0100) and the bits stored (0028,0101) are always the same.
2. The high bit (0028,0102) is always one less than the bits allocated.
3. The pixel representation (0028,0103) is always zero
4. Ultrasound data present (0028,0014) is always 1 (true).
5. 3D and Panview images always employ an RGB photometric interpretation irrespective of the 'image format' configured by the user. Also 3D and Panview images are slightly smaller (528 rows by 720 columns) than 2D images.

### 2.1.2.1.2.1 SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class

The EnVisor AE uses the Ultrasound Image IOD Modules for both Ultrasound Image (1.2.840.10008.5.1.4.1.1.6.1) and Ultrasound Multi-frame Image (1.2.840.10008.5.1.4.1.1.3.1) IODs as follows:

### 2.1.2.1.2.1.1 Ultrasound Image & Ultrasound multi-frame image Storage Modules Used

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - US Image & US multi-frame image Storage Modules Used

For each SOP class, DICOM defines what modules must be supported. A module simply defines a set of DICOM tags that must be present in the DICOM file.

Per the DICOM standard (PS3.3-2001 A.1.3), modules may be mandatory, optional or conditionally mandatory:

- **Mandatory** modules shall be supported per the definitions, semantics and requirements defined in PS3.3-2001, Annex C.
- **User Option** Modules may or may not be supported. If an optional Module is supported, the Level 1 (and Level 2) Attribute Types specified in the Modules shall be supported.
- **Conditional** Modules are Mandatory Modules if specific conditions are met. If the specified conditions are not met, this Module shall not be supported; that is, no information defined in that Module shall be sent.

The following table defines the modules that are supported by EnVisor for ultrasound images when they are sent to a Storage SCP (PACS).

Module	US Image (Still)		US Multi-frame Image (Loop)	
	DICOM Standard	Supported in EnVisor	DICOM Standard	Supported in EnVisor
Patient	Mandatory	✓	Mandatory	✓
General Study	Mandatory	✓	Mandatory	✓
Patient Study	User Option	✓	User Option	✓
General Series	Mandatory	✓	Mandatory	✓
Frame of Reference	User Option		User Option	
Synchronization	User Option		User Option	
General Equipment	Mandatory	✓	Mandatory	✓
General Image	Mandatory	✓	Mandatory	✓
Image Pixel	Mandatory	✓	Mandatory	✓
Palette Color Lookup Table	Conditional	✓	Conditional	✓
Contrast/Bolus	Conditional		Conditional	
Cine	Unused		Mandatory	✓
Multi-Frame	Unused		Mandatory	✓
Ultrasound Image	Mandatory	✓	Mandatory	✓

Module	US Image (Still)		US Multi-frame Image (Loop)	
	DICOM Standard	Supported in EnVisor	DICOM Standard	Supported in EnVisor
Overlay Plane	Unused		User Option	
Curve Identification	Mandatory	Not used since Curve & Curve Id is mutually exclusive with Image Pixel	Mandatory	Not used since Curve & Curve Id is mutually exclusive with Image Pixel
Curve	Mandatory		Mandatory	
Audio	User Option		User Option	
VOI LUT	User Option		User Option	
SOP Common	Mandatory	✓	Mandatory	✓

For each module that must be present in an ultrasound image that is going to be sent to a storage SCP, a subsequent sub-section defines the tags in that module that are supported by EnVisor.

Note: *Unused type "3" tags are **not** listed.*

### 2.1.2.1.2.1.2 Patient Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - Patient Module

The Patient Module (PS3.3-2001, Table C.7.1.1) defines attributes that provide information about the Patient who is the subject of a diagnostic Study. This module is mandatory for storage of ultrasound single-frame or multi-frame images.

\*Note: If MWL is enabled, the user-generated values in the table below are provided by the modality worklist.

Attribute Name	Tag	Type VR		Description	Generated by		Value
					Usr	Sys	
Patient's Name	0010, 0010	2	PN	Patient's full name.	✓		Entered by user from the Patient ID screen.
Patient ID	0010, 0020	2	LO	Primary hospital identification number or code for the patient.	✓		Entered by user from the Patient ID screen. This maps to the MRN field of the Patient ID screen and can be up to a maximum of 18 characters. If the user does not enter a value, the system will automatically generate one.
Patient's Birth Date	0010, 0030	2	DA	Birth date of the patient.	✓		Entered by user from the Patient ID screen. If the user does not enter a value, the system includes this tag as the empty string.
Patient's Sex	0010, 0040	2	CS	Sex of the named patient. Enumerated Values: M = male F = female O = other	✓		Selected from a drop-down list, by the user, from the Patient ID screen. If the user selects 'Unknown', this attribute is the empty string.
Other Patient IDs	0010, 1000	3	LO	Other identification numbers or codes used to identify the patient.	✓		Entered by user from the Patient ID screen. This maps to the Alternate ID Number of the Patient ID screen. If the user does not enter a value, the tag is not sent.

### 2.1.2.1.2.1.3 General Study Module


Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - General Study Module

The General Study Module (PS3.3-2001, Table C.7.2.1) defines Attributes that provide information about the Study that was performed. This module is mandatory for storage of ultrasound single-frame or multi-frame images.

\*Note: If MWL is enabled, the user-generated values in the table below are provided by the modality worklist.

Attribute Name	Tag	Type VR		Description	Generated by		Value
					Usr	Sys	
Study Instance UID	0020, 000D	1	UI	A unique identifier for the Study.		✓	A system generated Unique Identifier of the form: 1.2.840.113543.6.6.3.2.nnnnnnnn nnnnnnnnnnnnnnnnnnnnnnnnnnnnnnnnnnnn (64 characters) The first part is for EnVisor. The right-most digits (nnnnnn) are unique based on timestamp and machine characteristics.
Study Date	0008,	2	DA	Date the Study		✓	The system computes this value as the date the study was

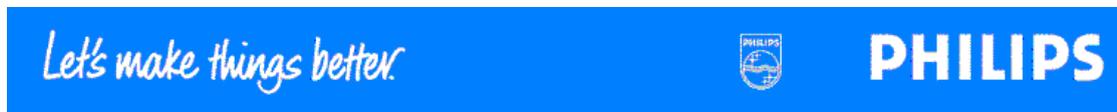

Attribute Name	Tag	Type VR		Description	Generated by		Value
					Usr	Sys	
	0020			started. The format is yyyymmdd			created. Every image (with the same Study Instance UID) will have the same Study date.
Study Time	0008, 0030	2	TM	Time the Study started. The format is hhhmmss		✓	The system computes this value as the time the study was created. Every image (with the same Study Instance UID) will have the same Study time.
Referring Physician Name	0008, 0090	2	PN	Physician(s) who are responsible for overall patient care at time of Study	✓		Entered by user from the Patient ID screen. If the user does not enter a value, the system fills includes this tag as the empty string.
Study ID	0020, 0010	2	SH	User or equipment generated Study identifier.		✓	A system generated Study identifier that is unique only within the EnVisor system that generated the study. The Study Identifier starts at 1 and is incremented by one for each new study created on that system. Study Identifiers will not be unique across multiple EnVisor systems.
Accession Number	0008, 0050	2	SH	A RIS generated number, which identifies the order for the Study.	✓		Entered by user from the Patient ID screen. If the user enters a value for this field, then it must be unique. If the user does not enter a value, the system includes this tag as the empty string.
Study Description	0008, 1030	3	LO	Institution-generated description or		✓	Constant string. In English systems the string is "Ultrasound Study". In non-

Attribute Name	Tag	Type		Description	Generated by		Value
					Usr	Sys	
				classification of the Study (component) performed.			English EnVisor systems, this field will be <b>translated</b> to the semantically equivalent string in the language of the locality.

#### 2.1.2.1.2.1.4 Patient Study Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - Patient Study Module

The Patient Study Module (PS3.3-2001, Table C.7.2.2) defines Attributes that provide information about the Patient at the time the Study was performed. This module is optional for storage of ultrasound single-frame or multi-frame images.

\*Note: If MWL is enabled, the user-generated values in the table below are provided by the modality worklist.

Attribute Name	Tag	Type		Description	Generated by		Value
					Usr	Sys	
Patient's Size	0010, 1020	3	DS	Length or size of the Patient, in meters.	✓		Entered by user from the Patient ID screen. If the user does not enter a value, this tag is not sent.
Patient's Weight	0010, 1030	3	DS	Weight of the Patient, in kilograms.	✓		Entered by user from the Patient ID screen. If the user does not enter a value, this tag is not sent.
Additional Patient's History	0010, 21B0	3	LT	Additional information about the Patient's medical history.	✓		Entered by user from the Patient ID screen. If the user does not enter a value, this tag is not sent.

### 2.1.2.1.2.1.5 General Series Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - General Series Module

The General Series Module (PS3.3-2001, Sec C.7.3.1, Table C.7-5) defines Attributes that identify and describe general information about a Series within a Study. . This module is mandatory for storage of ultrasound single-frame or multi-frame images. . Each EnVisor Study has exactly one Series.

\*Note: If MWL is enabled, the user-generated values in the table below are provided by the modality worklist.

Attribute Name	Tag	Type		Description	Generated by		Value
					Usr	Sys	
Modality	0008, 0060	1	CS	Type of equipment that originally acquired the data used to create the images in this Series.		✓	Always "US" for ultrasound
Series Instance UID	0020, 000E	1	UI	Unique identifier of the Series.		✓	Same as the Study Instance UID, tag 0020, 000D. See General Study Module section 2.1.2.1.2.1.3
Series	0020,	2	IS	Number of the		✓	Always "0".

Attribute Name	Tag	Type VR		Description	Generated by		Value
					Usr	Sys	
Number	0011			series			EnVisor studies have exactly one series.
Performing Physician's Name	0008, 1050	3	PN	Name of the physicians administering the Series.	✓		Entered by user from the Patient ID screen. This maps to the 'Performed by' field of the Patient ID screen. If the user does not enter a value, this tag is not sent.
Operator's Name	0008, 1070	3	PN	Name of the operator (or technician) using the system.		✓	Entered by the system as the same text as 'Performing Physician's Name', (tag 0008,1050).
Patient Position	0018, 5100	2C	CS	Required for CT and MR images. See C.7.3.1.1.2 of the DICOM standard for Defined Terms and further explanation.			Not used as not required for Ultrasound (US).
Requested Procedure ID	0040, 1001	1C	SH	Identifier which identifies the Requested Procedure in the Imaging Service Request. Required if Sequence Item is present.			From the modality worklist if MWL is enabled, blank otherwise.
Scheduled Procedure Step ID	0040, 0009	1C	SH	Identifier which identifies the Scheduled Procedure Step. Required if			From the modality worklist if MWL is enabled, blank otherwise.

Attribute Name	Tag	Type		Description	Generated by		Value
					Usr	Sys	
				Sequence Item is present.			

### 2.1.2.1.2.1.6 General Equipment Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - General Equipment Module

The General Equipment Module (PS3.3-2001, Sec C.7.5.1, Table C.7-8) defines attributes that identify and describe the piece of equipment that produced a Series of Images. This module is mandatory for storage of ultrasound single-frame or multi-frame images.

Attribute Name	Tag	Type VR		Description	Generated by		Value
					Usr	Sys	
Manufacturer	0008, 0070	2	LO	Manufacturer of the equipment that produced the digital images.		✓	Constant string.  In English systems the string is "Philips Medical Systems". In non-English EnVisor systems, this field will be <b>translated</b> to the semantically equivalent string in the language of the locality.

### 2.1.2.1.2.1.7 General Image Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - General Image Module

The General Image Module (PS3.3-2001, Sec C.7.6.1, Table C.7-9) defines Attributes that describe an image within a particular series. This module is optional for storage of ultrasound single-frame or multi-frame images.

All attributes are system generated.

Attribute Name	Tag	Type		Description	Value
			VR		
Instance Number	0020, 0013	2	IS	A number that identifies this image.  Note: This Attribute was named Image Number in earlier versions of this Standard.	The system computes this value as a unique number for each image in a study. The value ascends as each image is acquired but the value does not start at "1" for each study. Gaps may be present if images were deleted before DICOM store was requested.
Patient Orientation	0020, 0020	2C	CS	Patient direction of the rows and columns of the image.	The system computes this value as Zero length for 2D images, not used in

Attribute Name	Tag	Type	VR	Description	Value
					3D/PanView images.
Content Date	0008, 0023	2C	DA	The date the image pixel data creation started. Required if image is part of a series in which the images are temporally related. <i>Note: This Attribute was formerly known as Image Date.</i>	The system computes this value as the date that image was acquired. . The format is yyyyymmdd.
Content Time	0008, 0033	2C	TM	The time the image pixel data creation started. Required if image is part of a series in which the images are temporally related. <i>Note: This Attribute was formerly known as Image Time.</i>	The system computes this value as the time that image was acquired. The format is hhmmss
Image Type	0008, 0008	2	CS	Image identification characteristics.	The system computes this value as the multi-value attribute ORIGINAL/PRIMARY/cccc cc/nnnn ORIGINAL/PRIMARY denotes original source-data based on primary examination. The third field is automatically generated based on the type of the exam. It is mapped to the most appropriate value from the DICOM standard (Ex: "ABDOMINAL").  Note: The third field is <b>not</b>

Attribute Name	Tag	Type	VR	Description	Value
					present in stills and loops from 3D/PanView.  The fourth field is always blank.
Acquisition Date	0008,0022	3	DA	The date the acquisition of data that resulted in this image started	The system uses the same value as the Content Date, tag 0008,0023.
Acquisition Time	0008,0032	3	TM	The time the acquisition of data that resulted in this image started	The system uses the same value as the Content time, tag 0008,0033.
Acquisition Datetime	0008,002A	3	DT	The date and time that the acquisition of data that resulted in this image started.	The system generates this as a combination of Acquisition Date and Acquisition Time. . The format is <code>yyyymmddhhmmss</code>
Lossy Image Compression	0028,2110	3	CS	Specifies whether an Image has undergone lossy image compression. Enumerated Values:  00 = Image has NOT been subjected to lossy image compression.  01 = Image has been subjected to lossy image compression.	00 - for uncompressed images or RLE compressed images.  01 – for JPEG compressed images.

### 2.1.2.1.2.1.8 Image Pixel Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - Image Pixel Module

The Image Pixel Module (PS3.3-2001, Sec C.7.6.3, Table C.7-11) defines Attributes that describe the pixel data of an image. This module is mandatory for storage of ultrasound single-frame or multi-frame images.

Attribute Name	Tag	Type	VR	Description	Value
Samples per Pixel	0028, 0002	1	US	Number of samples (planes) in this image.	Based on the 'Image Format' that is set by the user in DICOM Setup.  Palette Color Mode: 2D B/W and Color stills/loops: 1 Reports: 1  RGB Mode: 2D B/W and Color stills/loops: 3 Reports: 3 3D & Panview: 3

Attribute Name	Tag	Type	VR	Description	Value
					YBR Mode: 2D B/W and Color stills/loops: 3 Reports: 3
Photometric Interpretation	0028,0004	1	CS	Specifies the intended interpretation of the pixel data.	2D Images & Reports: Based on the 'Image Format' that is set by the user in DICOM Setup. Can be either: PALETTE COLOR, RGB, or YBR_FULL_422 3D & Panview Images: Always RGB
Rows	0028,0010	1	US	Number of rows in the image.	2D B/W & Color stills/loops: 564 2D B/W & Color stills/loops from stress: 245 Reports: 564 3D & PanView: 528
Columns	0028,0011	1	US	Number of columns in the image	2D B/W & Color stills/loops: 800 2D B/W & Color stills/loops from stress: 320 Reports: 800 3D & PanView: 720
Bits Allocated	0028,0100	1	US	Number of bits allocated for each pixel sample.	Based on the 'Image Format' that is set by the user in DICOM Setup.  Palette Color Mode: 2D B&W: 8 bits 2D Color & Reports: 16 bits  RGB Mode:

Attribute Name	Tag	Type	VR	Description	Value
					2D B&W, 3D & PanView: 8 bits 2D Color & Reports: 8 bits  YBR Mode: 2D B&W: 8 bits 2D Color & Reports: 8 bits
Bits Stored	0028, 0101	1	US	Number of bits stored for each pixel sample.	Based on the 'Image Format' that is set by the user in DICOM Setup. The number of Bits Stored and Bits Allocated are always the same.  Palette Color Mode: 2D B&W: 8 bits 2D Color & Reports: 16 bits  RGB Mode: 2D B&W, 3D & PanView: 8 bits 2D Color & Reports: 8 bits  YBR Mode: 2D B&W: 8 bits 2D Color & Reports: 8 bits
High Bit	0028, 0102	1	US	Most significant bit for pixel sample data.	Based on the 'Image Format' that is set by the user in DICOM Setup. The High Bit is always one less than the Bits Allocated.  Palette Color Mode: 2D B&W: 7 bits

Attribute Name	Tag	Type	VR	Description	Value
					2D Color & Reports: 15 bits  RGB Mode: 2D B&W, 3D & PanView: 7 bits 2D Color & Reports: 7 bits  YBR Mode: 2D B&W: 7 bits 2D Color & Reports: 7 bits
Pixel Representation	0028, 0103	1	US	Data representation of the pixel samples. Each sample shall have the same pixel representation. Enumerated Values: 0000H = unsigned integer. 0001H = 2's complement	000H (Unsigned integers)
Pixel Data	7FE0, 0010	1	OB	A data stream of the pixel samples which comprise the Image.	The pixel data of the DICOM image.
Planar Configuration	0026, 0006	1 C	US		Always zero for 3D/PanView (RGB) images, otherwise this tag is omitted
Pixel Aspect Ratio	0028, 0034	1 C	IS	Ratio of the vertical size and horizontal size of the pixels in the image specified by a pair of integer values where the first	Always 1/1.

Attribute Name	Tag	Type VR		Description	Value
				value is the vertical pixel size, and the second value is the horizontal pixel size.	
Red Palette Color Lookup Table Descriptor	0028, 1101	1 C	US	Specifies the format of the Red Palette Color Lookup Table Data	See 'Palette Color Lookup Table Module', section 2.1.2.1.2.1.9.
Green Palette Color Lookup Table Descriptor	0028, 1102	1 C	US	Specifies the format of the Green Palette Color Lookup Table Data	See 'Palette Color Lookup Table Module', section 2.1.2.1.2.1.9.
Blue Palette Color Lookup Table Descriptor	0028, 1103	1 C	US	Specifies the format of the Blue Palette Color Lookup Table Data	See 'Palette Color Lookup Table Module', section 2.1.2.1.2.1.9..
Red Palette Color Lookup Table Data	0028, 1201	1 C	OW	Red Palette Color Lookup Table Data.	See 'Palette Color Lookup Table Module', section 2.1.2.1.2.1.9.
Green Palette Color Lookup Table Data	0028, 1202	1 C	OW	Green Palette Color Lookup Table Data.	See 'Palette Color Lookup Table Module', section 2.1.2.1.2.1.9.
Blue Palette Color Lookup Table Data	0028, 1203	1 C	OW	Blue Palette Color Lookup Table Data.	See 'Palette Color Lookup Table Module', section 2.1.2.1.2.1.9.

### 2.1.2.1.2.1.9 Palette Color Lookup Table Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - Palette Color Lookup Table Module

The Palette Color Lookup Module (PS3.3-2001, Sec C.7.9, Table C.7-22) defines Attributes that describe the Lookup table data for images with Palette Color photometric interpretation. This module is present for EnVisor 2D B/W and Color stills/loops but is not present in (RGB) files created by EnVisor's 3D/Panview application.

All attributes are system generated.

Attribute Name	Tag	Type	VR	Description	Value
Red Palette Color Lookup Table Descriptor	0028, 1101	1C	US	Specifies the format of the Red Palette Color Lookup Table Data	Used only for 2D Greyscale: 256, 0, 16 Color: 0, 0, 16  <b><i>This tag and other tags related to Palette Color are not present in 3D/PanView since these files are RGB.</i></b>
Green Palette Color Lookup Table	0028, 1102	1C	US	Specifies the format of the Green Palette Color Lookup Table Data	Used only for 2D Greyscale: 256, 0, 16 Color: 0, 0, 16

Attribute Name	Tag	Type	VR	Description	Value
Descriptor					
Blue Palette Color Lookup Table Descriptor	0028, 1103	1C	US	Specifies the format of the Blue Palette Color Lookup Table Data	Used only for 2D Greyscale: 256, 0, 16 Color: 0, 0, 16
Red Palette Color Lookup Table Data	0028, 1201	1C	OW	Red Palette Color Lookup Table Data.	Used only for 2D.
Green Palette Color Lookup Table Data	0028, 1202	1C	OW	Green Palette Color Lookup Table Data.	Used only for 2D.
Blue Palette Color Lookup Table Data	0028, 1203	1C	OW	Blue Palette Color Lookup Table Data.	Used only for 2D.
Segmented Red Palette Color Lookup Table Data	0028, 1221	1C	OW	Segmented Red Palette Color Lookup Table Data.	Not used
Segmented Green Palette Color Lookup Table Data	0028, 1222	1C	OW	Segmented Green Palette Color Lookup Table Data.	Not used
Segmented Blue Palette Color Lookup	0028, 1223	1C	OW	Segmented Blue Palette Color Lookup Table Data.	Not used

Attribute Name	Tag	Type	VR	Description	Value
Table Data					



### 2.1.2.1.2.1.10 Cine Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - Cine Module

The Cine Module (PS3.3-2001, Sec C.7.6.5, Table C.7-13) defines Attributes of a Multi-frame Cine image. This module is mandatory for Multi-frame images but is not used for Single-frame images.

All attributes are system generated.

Attribute Name	Tag	Type		Value
			VR	
Recommended Display Frame Rate	0008, 2144	3	IS	Used for Multiframe
Cine Rate	0018, 0040	3	IS	Used for Multiframe
Effective Series Duration	0018, 0072	3	DS	Used for Multiframe
Frame Time Vector	0018, 1065	1C	DS	An array that contains the real time increments (in msec) between frames for a Multi-frame image. Required since Frame Increment Pointer (0028,0009) points to Frame Time Vector

### 2.1.2.1.2.1.11 Multi-Frame Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - Multi-Frame Module

The Multi-Frame Module (PS3.3-2001, Sec C.7.6.6, Table C.7-14) defines Attributes of a Multi-frame pixel data image. This module is mandatory for Multi-frame images but is not used for Single-frame images. All attributes are system

generated.

Attribute Name	Tag	Type	VR	Value
Number of Frames	0028, 0008	1	IS	Used
Frame Increment Pointer	0028, 0009	1	AT	(0018,1065) Frame Time Vector

### 2.1.2.1.2.1.12 US Region Calibration Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - US Region Calibration Module

The US Region Calibration Module (PS3.3-2001, Sec C.8.5.5.1, Table C.8-17) defines Attributes that describe an ultrasound region calibration. This module is optional for ultrasound.

All attributes are system generated.

Attribute Name	Tag	Type	VR	Value
Sequence of Ultrasound Regions	0018, 6011	1	SQ	Used
Region Spatial Format	0018, 6012	1	US	Used
Region Data Type	0018, 6014	1	US	Used
Region Flags	0018, 6016	1	UL	Used
Region Location Min X0	0018, 6018	1	UL	Used
Region Location Min Y0	0018, 601A	1	UL	Used

Attribute Name	Tag	Type	VR	Value
Region Location Max X1	0018, 601C	1	UL	Used
Region Location Max Y1	0018, 601E	1	UL	Used
Reference Pixel X0	0018, 6020	1	SL	Used
Reference Pixel Y0	0018, 6022	1	SL	Used
Physical Units X Direction	0018, 6024	1	US	Used
Physical Units Y Direction	0018, 6026	1	US	Used
Ref Pixel Physical Value X	0018, 6028	1	FD	Used
Ref Pixel Physical Value Y	0018, 602A	1	FD	Used
Physical Delta X	0018, 602C	1	FD	Used
Physical Delta Y	0018, 602E	1	FD	Used

### 2.1.2.1.2.1.13 US Image Module

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Storage of DICOM studies to a PACS
        - Proposed Presentation Context
          - SOP Specific Conformance Statement for Ultrasound Image Storage SOP Class
            - US Image Module

The US Image Module (PS3.3-2001, Sec C.8.5.6, Table C.8-18) defines attributes that describe ultrasound images. This module is mandatory for storage of ultrasound single-frame or multi-frame images.

Attribute Name	Tag	Type		Value
			VR	
Samples per Pixel	0028, 0002	1	US	See 'Image Pixel Module' section 2.1.2.1.2.1.8.
Photometric Interpretation	0028, 0004	1	CS	See 'Image Pixel Module' section 2.1.2.1.2.1.8.
Bits Allocated	0028, 0100	1	US	See 'Image Pixel Module' section 2.1.2.1.2.1.8.
Bits Stored	0028, 0101	1	US	See 'Image Pixel Module' section 2.1.2.1.2.1.8.
High Bit	0028, 0102	1	US	See 'Image Pixel Module' section 2.1.2.1.2.1.8.
Pixel Representation	0028, 0103	1	US	Always zero
Image Type	0008,	2	CS	See 'General Image Module' 2.1.2.1.2.1.7.

Attribute Name	Tag	Type		Value
			VR	
	0008			
Lossy Image Compression	0028, 2110	1C	CS	See 'General Image Module' 2.1.2.1.2.1.7.
Ultrasound Color Data Present	0028, 0014	3	US	"1" for B/W and Color 2D and 3D/PanView stills/loops.
Number of Stages	0008, 2124	2C	IS	Number of stages in a protocol. <i>For stress Multiframe images only, or else this tag is not used.</i>
Number of Views in Stage	0008, 212A	2C	IS	Number of views in a stage. <i>For stress Multiframe images only</i>
Stage Name	0008, 2120	3	SH	Used in protocols <i>For stress Multiframe images only</i>
Stage Number	0008, 2122	3	IS	Which stage in a protocol. <i>For stress Multiframe images only</i>
View Number	0008, 2128	3	IS	Number of event timers used during acquisition <i>For stress Multiframe images only.</i> 1, 2, or 3 Null if timers not used
Heart Rate	0018, 1088	3	IS	Beats per minute. <i>For stress Multiframe images only</i>
View Name	0008, 2127	3	SH	The name of the view. <i>For stress Multiframe images only</i>



### 2.1.2.2 Issuing of Storage Commitment requests to an image manager

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Issuing of Storage Commitment requests to an image manager

EnVisor provides Standard Conformance to the following DICOM V3.0 **Storage Commitment** SOP Class as an SCU.

SOP Class Name	SOP Class UID	Role
Storage Commitment Push Model	1.2.840.10008.1.20.1.1	SCU

**Table 6: SOP Class Supported by Storage Commitment service**

EnVisor sends images to the storage server for permanent storage. The request for Storage Commitment may then be transmitted from EnVisor together with a list of references to one or more SOP instances. This action is invoked through the DIMSE N-ACTION primitive.

### 2.1.2.2.1 Associated Real-World Activity

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Issuing of Storage Commitment requests to an image manager
        - Associated Real-World Activity

Storage Commitment is initiated when a study is saved by the user, or, in Send As You Go mode, after each image acquisition. The storage SCP and commit SCP can be different AE's.

Storage Commitment Association Behavior (By Save Study or Send As You Go)

User Action	DICOM Activity – Storage Commitment Device Association	Association Status
Save Study (Or Image acquisition in Send As You Go)	Each Save Study operation will initiate an association with the SC server, and send an N-Action Request, containing a list of all images that need to be committed. Then Association Release Request.  In Send As You Go mode, each image acquisition initiates the same DICOM activity as Save Study.	Association closed.
Reverse Role Negotiation	The system will remain available as long as it is connected to the network to receive Storage Commitment responses from the SC server. The SCP will send an N-Event Report with status. Then the association is released.	Association closed.

### 2.1.2.2.2 Proposed Presentation Contexts

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Issuing of Storage Commitment requests to an image manager
        - Proposed Presentation Contexts

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Storage Commitment Push Model	1.2.840.10008.1.20.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None

**Table 7: Storage Commitment - Presentation Context**

### 2.1.2.2.1 SOP Specific Conformance Statement for Storage Commitment SOP Class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Issuing of Storage Commitment requests to an image manager
        - Proposed Presentation Contexts
          - SOP Specific Conformance Statement for Storage Commitment SOP Class

EnVisor provides standard conformance to the DICOM Storage Commitment Service Class.

EnVisor supports the following elements for this SOP class as an SCU. The Transaction UID Attribute (0008,1195) value generated by EnVisor uniquely identifies each Storage Commitment Request.

Action Type Name	Action Type ID	Attribute Name	Tag
Request Storage Commitment	1	Transaction UID	(0008,1195)
		Referenced SOP Sequence	(0008,1199)
		>Referenced SOP Class UID	(0008,1150)
		>Referenced SOP Instance UID	(0008,1155)

**Table 8 – Storage Commitment Request – Attributes**

Subsequently, EnVisor expects N-EVENT-REPORT's from the storage commit server although EnVisor does not assume that the event will arrive at any particular time. EnVisor does not wait but will process the event whenever it arrives.

EnVisor might be either powered down or disconnected from the network and used in portable mode, it is possible for the N-EVENT-REPORT to arrive from the Storage Commitment SCP while EnVisor cannot receive it. If an outstanding N-EVENT-REPORT does not arrive within 96 hours, then EnVisor will reissue the same Storage Commitment request. When the event arrives, EnVisor returns an N-EVENT-REPORT response primitive with one of the following status codes.

Service Status	Further Meaning	Protocol Codes	Related Fields	Description
Success	Success	0000		N-EVENT-REPORT message understood.
Error	Failed	0110		N-EVENT-REPORT message was not processed successfully.

**Table 9 - Storage Commitment status codes**

### 2.1.2.3 Verification of the existence of DICOM server on the hospitals network

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Verification of the existence of DICOM server on the hospitals network

EnVisor provides standard conformance to the DICOM V3.0 SOP Class as shown in Table 10.

SOP Class Name	SOP Class UID	Role
Verification SOP Class	1.2.840.10008.1.1	SCU

**Table 10: SOP Class Supported by Verification Service**

### 2.1.2.3.1 Associated Real-World Activity

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Verification of the existence of DICOM server on the hospitals network
      - Associated Real-World Activity

The user can verify the existence of a DICOM server on the hospitals network, through a button in the 'DICOM Setup' screen. When the user presses this button, EnVisor will initiate the association.

### 2.1.2.3.2 Proposed Presentation Contexts

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Verification of the existence of DICOM server on the hospitals network
      - Proposed Presentation Contexts

Only one association is established for each verification attempt. When the association is opened, the presentation contexts noted in Table 11 are proposed.

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
		Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None

**Table 11: Proposed Presentation Contexts**

### 2.1.2.3.2.1 SOP Specific Conformance Statement for the Verification SOP class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Verification of the existence of DICOM server on the hospitals network
        - Proposed Presentation Contexts
          - SOP Specific Conformance Statement for the Verification SOP class

The C-ECHO request primitive is sent to the Verification SCP. The Verification SCP with a status indicator of success returns the C-ECHO response primitive. The absence of a C-ECHO response within a specific timeout period is an indication that the server cannot be located through the Verification service.

### 2.1.2.4 Printing DICOM studies to a B&W or color printer

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer

EnVisor provides standard conformance to the following DICOM V3.0 SOP Class as an SCU.

SOP Class Name	SOP Class UID	Role
Basic Film Session	1.2.840.10008.5.1.1.1	SCU
Basic Film Box	1.2.840.10008.5.1.1.2	SCU
Basic Grayscale Image Box	1.2.840.10008.5.1.1.4	SCU
Basic Grayscale Print Management Meta	1.2.840.10008.5.1.1.9	SCU
Referenced Grayscale Print Management Meta	1.2.840.10008.5.1.1.9.1	SCU
Basic Color Image Box	1.2.840.10008.5.1.1.4.1	SCU
Basic Color Print Management Meta	1.2.840.10008.5.1.1.18	SCU
Referenced Color Print Management Meta	1.2.840.10008.5.1.1.18.1	SCU
Basic Annotation Box	1.2.840.10008.5.1.1.15	SCU
Printer	1.2.840.10008.5.1.1.16	SCU
Print Job	1.2.840.10008.5.1.1.14	SCU
Presentation LUT	1.2.840.10008.5.1.1.23	SCU

**Table 12: SOP Classes Supported by Print Service**

### 2.1.2.4.1 Associated Real World Activity

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Associated Real World Activity

EnVisor issues Print Management requests to an SCP supporting the DICOM V3.0 Print services, in order to produce hard copy representations of DICOM images, based on user requests.

Print association Negotiation - Association Status (After Each Image)

User Action	DICOM Activity - Print Batch Mode	DICOM Activity – Print Send As You Go Mode
First image acquired from system	None.	If a full page is ready to print: Association

Print Association Negotiation - Association Status  
(Save Study in Batch Print Mode)

User Action	DICOM Activity – Print
Save Study	Association Negotiation + N_Create Film Session and N_Create Film Box, N_GET Status then N_Sets for each image and N_Action for each page, then Association Release Request

### 2.1.2.4.2 Proposed Presentation Contexts

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Proposed Presentation Contexts

Print AE supports the following Presentation Contexts for **Print**.

Abstract Syntax		Transfer Syntax			Extended Negotiation
Name	UID	Name List	UID List	Role	
Basic Film Session	1.2.840.10008.5.1.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None
Basic Film Box	1.2.840.10008.5.1.1.2	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None
Basic Grayscale Image Box	1.2.840.10008.5.1.1.4	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None
Basic Grayscale Print Management Meta	1.2.840.10008.5.1.1.9	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None
Basic Color Image Box	1.2.840.10008.5.1.1.4.1	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None

Abstract Syntax		Transfer Syntax			Extended
Name	UID	Name List	UID List	Role	Negotiation
Basic Color Print Management Meta	1.2.840.10008.5.1.1.18	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None
Basic Annotation Box	1.2.840.10008.5.1.1.15	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None
Printer	1.2.840.10008.5.1.1.16	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None

**Table 13: Print Presentation Contexts**

EnVisor provides standard conformance to the DICOM Print Service Classes by supporting a number of distinct Service classes described below.

### 2.1.2.4.2.1 SOP Specific Conformance to Basic Film Session SOP Class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Proposed Presentation Contexts
          - SOP Specific Conformance to Basic Film Session SOP Class

EnVisor requests the following DIMSE-N commands for the Basic Film Session SOP Class: N-CREATE

1 = Generated By

Attribute Name	Tag	U s a g e	Attribute Description	DICOM Notes	1		Value	
					U s r	S y s	Options	Default
Number of Copies	(2000, 0010)	U	Number of duplicate copies to print	DICOM supports an integer number.	✓		1 to 99	1
Print Priority	(2000, 0020)	U	Print priority sets the 'importance' of your print job relative to other jobs received by the printer.	DICOM supports: LOW, MEDIUM, HIGH Print priorities		✓	MEDIUM	
Medium Type	(2000, 0030)	U	The type of media the printer prints on.	DICOM Supports PAPER, CLEAR FILM & BLUE FILM s as well as 'Printer Specific' options	✓		PAPER CLEAR FILM BLUE FILM	PAPER

Attribute Name	Tag	Usage	Attribute Description	DICOM Notes	1		Value	
					U	S	Options	Default
Film Destination	(2000, 0040)	U	The processed film will be stored in a film magazine or processor.	DICOM supports PROCESSOR and MAGAZINE as well as 'Printer Specific' options	✓		PROCESSOR MAGAZINE	PROCESSOR

**Table 14 Basic Film Session Attributes**

### 2.1.2.4.2.2 SOP Specific Conformance to Basic Film Box SOP Class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Proposed Presentation Contexts
          - SOP Specific Conformance to Basic Film Box SOP Class

EnVisor requests the following DIMSE-N commands for the Basic Film Box SOP Class: N-CREATE

1 = Generated By

Attribute Name	Tag	Usage	Attribute Description	DICOM Notes	1		Value	
					U	S	Options	Default
Image Display Format	(2010, 0010)	M	Images are arranged on the film in a rectangular grid. The columns and rows control the layout.	DICOM Supports STANDARD\cols,rows as well as SLIDE, ROW/COL symmetric and printer specific options.	✓		STANDARD\cols,rows <i>Cols:1..99, Rows:1..99</i>	STANDARD\2,3
Film Orientation	(2010, 0040)	U	The orientation of the printed film or paper.	DICOM Supports: PORTRAIT & LANDSCAPE	✓		PORTRAIT LANDSCAPE	PORTRAIT
Film Size ID	(2010, 0050)	U	The overall size of the film or paper.	DICOM Supports all	✓		8INX10IN, 8_5INX11IN,	8INX10IN

Attribute Name	Tag	Usage	Attribute Description	DICOM Notes	1		Value	
					U	S	Options	Default
				the user options as well as 'Printer Specific' options			10INX12IN, 10INX14IN, 11INX14IN, 11INX17IN, 14INX14IN, 14INX17IN, 24CMX24CM, 24CMX30CM, A4, A3	
Magnification Type	(2010, 0060)	U	The algorithm used to scale images to fit on the media.	DICOM Supports: NONE, CUBIC, BILINEAR, REPLICATE as well as printer specific options		✓	CUBIC	
Border Density	(2010, 0100)	U	Border Density is the optical density of the film in the areas surrounding and between images.	DICOM Supports: BLACK & WHITE as well as a number (that is entered in 100's of OD) as well as Printer Specific options		✓	BLACK	

Attribute Name	Tag	U s a g e	Attribute Description	DICOM Notes	1		Value	
					U s r	S y s	Options	Default
Empty Image Density	(2010, 0110)	U	Empty Image Density	DICOM Supports: BLACK & WHITE as well as a number (that is entered in 100's of OD) as well as Printer Specific options		✓	BLACK	
Trim	(2010, 0140)	U	Draw frame box around each image	DICOM Supports: YES or NO		✓	YES	
Min Density	(2010, 0120)	U	Min and Max Density give the range of allowable image densities for an image on the film. If the density is outside the printer's range, the printer's limit will be used.	Entered in hundredths of Optical Density (OD)		✓	0	
Max Density	(2010, 0130)	U	Min and Max Density give the range of allowable image densities for an image on the film. If the density is outside the printer's range, the printer's limit will be used.	Entered in hundredths of Optical Density (OD)	✓		0 - 999	280

Attribute Name	Tag	Usage	Attribute Description	DICOM Notes	1		Value	
					U	S	Options	Default
Configuration Information	(2010, 0150)	U	Printer-specific configuration Information	DICOM supports a config ID # or a config string	✓		Config ID # Or Config string	BLANK
Annotation display format ID	(2010, 0030)	U	A printer specific Annotation format ID string.	Refer to the printers DCS for a list of supported annotation modes.		✓	BLANK	
Smoothing Type	(2010, 0080)	U	Smoothing type used when CUBIC Magnification is used to scale images.	Values are printer specific and defined in mfg's DCS.		✓	BLANK	
Referenced Film Session Sequence	(2010, 0500)	M	Referenced Film Session Sequence	N/A		✓	Always set	
>Reference d SOP Class UID	(0008, 1150)	M	>Referenced SOP Class UID	N/A		✓	Always set	
>Reference d SOP Instance UID	(0008, 1155)	M	>Referenced SOP Instance UID	N/A		✓	Always set	

**Table 15 Basic Film Box Attributes**

## N-ACTION

EnVisor provides all possible printer settings. For a specific printer, the user must check the manufacturer's documentation to determine the subset of available settings that the printer actually supports. For example, if the user configures the B&W printer to use a film-size of 14Inx17IN but the maximum film size supported by the printer is 8\_5INX11IN, then the printer may reject the images.

### 2.1.2.4.2.3 SOP Specific Conformance to Basic Grayscale Image Box SOP Class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Proposed Presentation Contexts
          - SOP Specific Conformance to Basic Grayscale Image Box SOP Class

Print AE issues the following DIMSE-N commands for the Basic Grayscale Image Box SOP Class:

N-SET

Attribute Name	Tag	Usage	Description	Generated By		Value	
				Usr	Sys	Options	Default
Image Position	(2020, 0010)	M	The position of the image on the media		✓	Always set	
Polarity	(2020, 0020)	U	Polarity of image on media (NORMAL or REVERSE)		✓	NORMAL	
Magnification Type	(2010, 0060)	U	Same as 2.1.2.4.2.2, SOP Specific Conformance to Basic Film Box SOP Class				
Smoothing Type	(2010, 0080)	U	Same as 2.1.2.4.2.2, SOP Specific Conformance to Basic Film Box SOP Class				

Attribute Name	Tag	Usage	Description	Generated By		Value	
				Usr	Sys	Options	Default
Basic Grayscale Image Sequence Type	(2020, 0110)	M	The image data attributes		✓	Always set (but only for B&W Images.)	
>Samples Per Pixel	(0028, 0002)	M	The number of data samples per pixel		✓	Always set (1)	
>Photometric Interpretation	(0028, 0004)	M	Interpretation of pixel data (MONOCHROME1/2, PALETTE COLOR, RGB, etc.)		✓	Always set (Monochrome2)	
>Rows	(0028, 0010)	M	The number of rows in the image		✓	Always set	
>Columns	(0028, 0011)	M	The number of columns in the image		✓	Always set	
>Pixel Aspect Ratio	(0028, 0034)	M	Ratio of hor & vert pixel size		✓	Always set	
>Bits Allocated	(0028, 0100)	M	Number of bits per pixel allocated		✓	Always set (8)	
>Bits Stored	(0028, 0101)	M	Number of bits per pixel actually stored		✓	Always set (8)	
>High Bit	(0028, 0102)	M	The most-significant-bit in the pixel		✓	Always set (7)	
>Pixel Representation	(0028, 0103)	M	Pixel representation (unsigned – 0 or signed – 1)		✓	Always set (0)	

Attribute Name	Tag	Usage	Description	Generated By		Value	
				Usr	Sys	Options	Default
>Pixel Data	(7FE0, 0010)	M	The pixel data		✓	Always set	

**Table 16 Basic Grayscale Image Box Attributes**

#### 2.1.2.4.2.4 SOP Specific Conformance to Basic Color Image Box SOP Class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Proposed Presentation Contexts
          - SOP Specific Conformance to Basic Color Image Box SOP Class

Print AE issues the following DIMSE-N commands for the Basic Color Image Box SOP Class:  
N-SET.

Attribute Name	Tag	Usage	Description	Generated By		Value		
				Usr	Sys	Options	Default	
Image Position	(2020, 0010)	M	Same as 2.1.2.4.2.3, SOP Specific Conformance to Basic Grayscale Image Box SOP Class					
Polarity	(2020, 0020)	U	Same as 2.1.2.4.2.3, SOP Specific Conformance to Basic Grayscale Image Box SOP Class					
Magnification Type	(2010, 0060)	U	Same as 2.1.2.4.2.2, SOP Specific Conformance to Basic Film Box SOP Class					
Smoothing Type	(2010, 0080)	U	Same as 2.1.2.4.2.2, SOP Specific Conformance to Basic Film Box SOP Class					
Basic Color Image Sequence	(2020, 0111)	M	The image data attributes		✓	Always set (but only for Color Images.)		

Attribute Name	Tag	Usage	Description	Generated By		Value	
				Usr	Sys	Options	Default
>Samples Per Pixel	(0028, 0002)	M	The number of data samples per pixel		✓	Always set (3)	
>Photometric Interpretation	(0028, 0004)	M	Interpretation of pixel data (MONOCHROME1/2, PALETTE COLOR, RGB, etc.)		✓	Always set (RGB)	
>Planar Configuration	(0028, 0006)	M	Planar configuration (color-by-plane or color-by-pixel)		✓	Always set (0)	
>Rows	(0028, 0010)	M	The number of rows in the image		✓	Always set	
>Columns	(0028, 0011)	M	The number of columns in the image		✓	Always set	
>Pixel Aspect Ratio	(0028, 0034)	M	Ratio of hor & vert pixel size		✓	Always set	
>Bits Allocated	(0028, 0100)	M	Number of bits per sample allocated		✓	Always set (8)	
>Bits Stored	(0028, 0101)	M	Number of bits per sample actually stored		✓	Always set (8)	
>High Bit	(0028, 0102)	M	The most-significant-bit in the sample		✓	Always set (7)	
>Pixel Representation	(0028, 0103)	M	Pixel representation (unsigned – 0 or signed – 1)		✓	Always set (0)	
>Pixel Data	(7FE0, 0010)	M	The pixel data		✓	Always set	

## Table 17 Basic Color Image Box Attributes

*Let's make things better.*



**PHILIPS**

#### **2.1.2.4.2.5 SOP Specific Conformance to Basic Annotation Box SOP Class**

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Proposed Presentation Contexts
          - SOP Specific Conformance to Basic Annotation Box SOP Class

The Basic Annotation Box is not used in EnVisor.

#### 2.1.2.4.2.6 SOP Specific Conformance to Printer SOP Class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Proposed Presentation Contexts
          - SOP Specific Conformance to Printer SOP Class

EnVisor issues the following DIMSE-N commands for the Printer SOP Class:

N-GET.

Attribute Name	Tag	Usage SCU/SCP
Printer Status	(2110,0010)	U/M
Printer Status Info	(2110,0020)	U/M
Printer Name	(2110,0030)	U/U
Manufacturer	(0008,0070)	U/U
Manufacturer's Model Name	(0008,1090)	U/U
Device Serial Number	(0018,1000)	U/U
Software Version	(0018,1020)	U/U

**Table 18 Printer Attributes**

Note: These printer commands are issued for internal use only. The printer status is never reported back to the user.

#### **2.1.2.4.2.7 SOP Specific Conformance to Basic Grayscale Print Management Meta SOP Class**

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Proposed Presentation Contexts
          - SOP Specific Conformance to Basic Grayscale Print Management Meta SOP Class

The Meta SOP class is requested at negotiation, but is then ignored. EnVisor uses the individual SOP classes defined by the DICOM specification.

#### **2.1.2.4.2.8 SOP Specific Conformance to Basic Color Print Management Meta SOP Class**

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Printing DICOM studies to a B&W or color printer
        - Proposed Presentation Contexts
          - SOP Specific Conformance to Basic Color Print Management Meta SOP Class

The Meta SOP class is requested at negotiation, but is then ignored. EnVisor uses the individual SOP classes defined by the DICOM specification.

### 2.1.2.5 Obtaining a list of scheduled work from the HIS via Modality Worklists

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Obtaining a list of scheduled work from the HIS via Modality Worklists

EnVisor provides Standard Conformance to the following DICOM V3.0 **Worklist Management** SOP Class as an SCU.

SOP Class Name	SOP Class UID	Role
Modality Worklist Info Model - FIND	1.2.840.10008.5.1.4.31	SCU

**Table 19: SOP Class Supported by Worklist Management Service**

EnVisor requests the transfer of worklists with the DIMSE C-FIND command.

### 2.1.2.5.1 Associated Real-World Activity

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Obtaining a list of scheduled work from the HIS via Modality Worklists
        - Associated Real-World Activity

EnVisor obtains scheduled worklists from the Modality Worklist Server in two ways; the user can manually request a fresh copy of the desired worklist from the Modality Worklist Server by pressing the Refresh button on the Patient Selection screen, also requests can be made on a polled basis in the background, with a polling interval configured by the user.

If EnVisor is not connected to the network, the 'Refresh' button is grayed-out. If EnVisor is connected to the network, the 'Refresh' button is enabled; the user can then press it to perform a manual refresh of EnVisor's worklist: EnVisor will first negotiate a C-ECHO with the server to verify that the MWL server is live and if live, perform the C-Find.

Note that if EnVisor is not connected to the Modality Worklist Server network, the worklist cached in EnVisor may be out-of-date with the worklist maintained by the Modality Worklist Server. Nevertheless, the cached worklist is available for use on portable exams.

When EnVisor is reconnected to the network a fresh copy of the current list is requested.

### 2.1.2.5.2 Proposed Presentation Contexts

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Obtaining a list of scheduled work from the HIS via Modality Worklists
        - Proposed Presentation Contexts

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Modality Worklist Info Model – FIND	1.2.840.1000 8.5.1.4.31	Implicit VR Little Endian	1.2.840.10008. 1.2	SCU	None

**Table 20: Worklist Management - Presentation Context**

### 2.1.2.5.2.1 SOP Specific Conformance Statement for the Modality Worklist SOP Class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Obtaining a list of scheduled work from the HIS via Modality Worklists
      - Proposed Presentation Contexts
        - SOP Specific Conformance Statement for the Modality Worklist SOP Class

EnVisor provides standard conformance to the DICOM Worklist Management Service Class.

Table 21 describes the use of attributes as both Matching Key values in the C-FIND request message, and as Return Keys in the set of C-FIND-RSP messages. The Matching Key Usage follows the DICOM Standard for attribute matching, including Single Value matching and Range matching. For those Matching Keys that are used by EnVisor, the Attribute Type as defined by DICOM is indicated: Required or Optional. These values indicate the degree to which the MWL SCP must support the attribute as a Matching Key.

Similarly, the Attribute Type of values used as Return Keys is given as defined by DICOM: Type 1 (required), Type 1C (conditionally required), Type 2 (required but may be NULL), Type 2C (conditionally required but may be NULL), or Type 3 (optional).

An empty value in the Matching Key column means that this value is not used as a matching key. An empty value in the Return Key column means that EnVisor ignores this value. If an attribute that is non-mandatory to the SCU is not used by EnVisor as a matching key and its value as a return key is ignored, the attribute is omitted from the list of attributes.

Attribute Name	Tag	Matching Key Usage	Return Key Usage
<b>SOP Common</b>			
Specific Character Set	(0008,0005)		Ignored by EnVisor.
Scheduled Procedure Step			
Scheduled Procedure Step Sequence	(0040,0100)	Required	Type 1
> Scheduled Station AE Title	(0040,0001)	Required	Type 1 Set in MPPS.
> Scheduled Procedure Step Start Date	(0040,0002)	Required	Type 1 Used in Patient Selection screen. Set in MPPS.
> Scheduled Procedure Step Start Time	(0040,0003)	Required	Type 1 Used in Patient Selection screen. Set in MPPS.
> Modality	(0008,0060)	Required	Type 1 Set in MPPS.
> Scheduled Performing Physician's Name	(0040,0006)		Type 2 Sets "Performed by" in the Patient ID screen, and "Performing Physician's Name" in the MPPS.
> Scheduled Procedure Step Description	(0040,0007)		Type 1C

Attribute Name	Tag	Matching Key Usage	Return Key Usage
			Set in MPPS and images. May be used to set "Description" field on the Patient Selection screen, and "Study Description" in images: 2nd choice, configurable
> Scheduled Procedure Step Location	(0040,0011)		Type 2  Sets "Location" field on the Patient Selection screen.
> Scheduled Protocol Code Sequence	(0040,0008)		Type 1C  Set as "Scheduled Action Item Code Sequence" and "Performed Action Item Code Sequence" in MPPS, and as "Scheduled Protocol Code Sequence" in images.
>> Code Value	(0008,0100)		Type 1  Set in MPPS and images.
>> Coding Scheme Designator	(0008,0102)		Type 1  Set in MPPS and images.
>> Coding Scheme Version	(0008,0103)		Type 3

Attribute Name	Tag	Matching Key Usage	Return Key Usage
			If present, set in MPPS and images.
>> Code Meaning	(0008,0104)		Type 3 If present, set in MPPS and images.  May also be used to set "Description" field on the Patient Selection screen, and "Study Description" in images: 3rd choice, configurable
> Scheduled Procedure Step ID	(0040,0009)		Type 1  Set in MPPS and images.
Requested Procedure			
Requested Procedure ID	(0040,1001)		Type 1  Set in MPPS and images.
Requested Procedure Description	(0032,1060)		Type 1C  Set in MPPS.  May also be used to set "Description" field on the Patient Selection screen, and "Study Description" in images: 1st choice, configurable

Attribute Name	Tag	Matching Key Usage	Return Key Usage
Requested Procedure Code Sequence	(0008,1064)		Type 1C If present, set as "Procedure Code Sequence" in MPPS.
> Code Value	(0008,0100)		Type 1C Set in MPPS.
> Coding Scheme Designator	(0008,0102)		Type 1C Set in MPPS.
> Coding Scheme Version	(0008,0103)		Type 3 If present, set in MPPS.
> Code Meaning	(0008,0104)		Type 3 If present, set in MPPS.
Study Instance UID	(0020,000D)		Type 1 Set in MPPS and images.
Referenced Study Sequence	(0008,1110)		Type 2 Set in MPPS and images.
> Referenced SOP Class UID	(0008,1150)		Type 1C Set in MPPS and images.
> Referenced SOP Instance UID	(0008,1155)		Type 1C

Attribute Name	Tag	Matching Key Usage	Return Key Usage
			Set in MPPS and images.
Reason for the Requested Procedure	(0040,1002)		Type 3  May be used to set "Indication" field on the Patient Selection screen:  1st choice, configurable
Imaging Service Request			
Accession Number	(0008,0050)		Type 2  Displayed on Patient ID screen. Set in MPPS and images.
Referring Physician's Name	(0008,0090)		Type 2  Sets "Referring Physician" in the Patient ID screen and the 'Patient Selection' screen.
Reason for Imaging Service Request	(0040,2001)		Type 3  May be used to set "Indication" field on the Patient Selection screen:  2nd choice,

Attribute Name	Tag	Matching Key Usage	Return Key Usage
			configurable
Visit Relationship			
Referenced Patient Sequence	(0008,1120)		Type 2 Set in MPPS.
> Referenced SOP Class UID	(0008,1150)		Type 2 Set in MPPS.
> Referenced SOP Instance UID	(0008,1155)		Type 2 Set in MPPS.
Patient Identification			
Patient Name	(0010,0010)		Type 1 Displayed on 'Patient ID' screen and 'Patient Selection' screen. Set in MPPS and used as a tag in images.
Patient ID	(0010,0020)		Type 1 Displayed in "MRN" field of 'Patient ID' screen and 'Patient Selection' screen. Set in MPPS and used as a tag in images.
Patient Demographic			

Attribute Name	Tag	Matching Key Usage	Return Key Usage
Patient's Birth Date	(0010,0030)		Type 2 Sets the "DOB" field on the Patient ID and Patient Selection screen. Set in MPPS.
Patient Sex	(0010,0040)		Type 2 Sets the "Gender" field on the 'Patient ID' screen and 'Patient Selection' screen. Set in MPPS.
Patient's Weight	(0010,1030)		Type 2 Sets the "Weight" field on the 'Patient ID' and 'Patient Selection' screens.
Patient's Size	(0010,1020)		Type 3 Sets the "Height" field on the 'Patient ID' and 'Patient Selection' screens.
<b>Table 21: Modality Worklist Usage in the Worklist Management service</b>			

### 2.1.2.6 Updating the status of a scheduled procedure

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Updating the status of a scheduled procedure

EnVisor provides Standard Conformance to the following DICOM V3.0 Modality Performed Procedure Step (MPPS) SOP Class as an SCU.

SOP Class Name	SOP Class UID	Role
Modality Performed Procedure Step SOP Class	1.2.840.10008.3.1.2.3.3	SCU

**Table 22: SOP Class Supported by MPPS service**

The system uses N-CREATE and N-SET commands to notify the MPPS Server whenever the status of a patient's study has changed.

### 2.1.2.6.1 Associated Real-World Activity

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Updating the status of a scheduled procedure
        - Associated Real-World Activity

The opening of a study marks the beginning of a new Modality Performed Procedure Step (MPPS). At this time, a MPPS record is created on the MPPS SCP through the use of the N-CREATE service. If the MPPS SCP is unavailable at the time the first image is stored, the request is queued and will be sent when the MPPS SCP is available.

When the user ends the scheduled procedure by closing the study and saving any changes, the MPPS status is “Completed”. Alternatively, the user may choose to cancel acquisition, the study is saved in local storage and the MPPS status becomes “Discontinued”. At this time, the Study Management AE attempts to modify the MPPS on the MPPS SCP through the use of the N-SET service. If the MPPS SCP is unavailable, the request is queued and will be sent when the MPPS SCP is available..

### 2.1.2.6.2 Proposed Presentation Contexts

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Updating the status of a scheduled procedure
        - Associated Real-World Activity
          - Proposed Presentation Contexts

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Modality Performed Procedure Step	1.2.840.10008.3.1.2.3.3	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None

**Table 20: MPPS - Presentation Context**

### 2.1.2.6.2.1 SOP Specific Conformance Statement for the MPPS SOP Class

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Initiation by Real-World Activity
      - Updating the status of a scheduled procedure
        - Associated Real-World Activity
          - Proposed Presentation Contexts
            - SOP Specific Conformance Statement for the MPPS SOP Class

EnVisor provides standard conformance to the DICOM MPPS Service Class.

The updated attributes are shown in Table 21. The “N\_CREATE Usage” column shows the attributes transmitted when the status of the study changes to “IN\_PROGRESS”. The “N\_SET Usage” column shows the attributes transmitted when the status of the study changes to “COMPLETED” or “DISCONTINUED”.

Note: The following fields are copied from the selected MWL entry to the Patient ID screen:

Accession Number,  
Patient’s Name,  
Patient’s ID,  
Patient’s Birth Date  
Patient’s Sex  
Performing Physician’s Name

Usually, the performing physician will accept the information in the Patient ID Screen, as is, however the physician has the option of editing the information before starting the study. If the physician edits this information then the MPPS N-CREATE command that is sent to the MPPS server on study start will use the edited information and not the original MWL information.

Attribute Name	Tag	N-CREATE Usage	N-SET Usage
----------------	-----	----------------	-------------

Attribute Name	Tag	N-CREATE Usage	N-SET Usage
Specific Character Set	(0008,0005)	Not used, even though some attributes may contain characters from the Latin 1 character set	Not used, even though some attributes may contain characters from the Latin 1 character set
<b>Performed Procedure Step Relationship</b>			
Scheduled Step Attribute Sequence	(0040,0270)	Present	Not allowed
> Study Instance UID	(0020,000D)	If available from the MWL; else synthesized by the host imaging system	Not allowed
> Referenced Study Sequence	(0008,1110)	If present in MWL else NULL	Not allowed
>> Referenced SOP Class UID	(0008,1150)	Detached Study Mgmt SOP Class: 1.2.840.10008.3.1.2.3.1	Not allowed
>> Referenced SOP Instance UID	(0008,1155)	From the MWL, if present; else the SOP Instance UID of this study.	Not allowed
> Accession Number	(0008,0050)	From the "Accession number" field of the Patient ID screen	Not allowed
> Requested Procedure ID	(0040,1001)	If available from the MWL; else NULL	Not allowed
> Requested Procedure Description	(0032,1060)	If available from the MWL; else NULL	Not allowed
> Scheduled Procedure Step ID	(0040,0009)	If available from the MWL; else NULL	Not allowed
> Scheduled Procedure Step Description	(0040,0007)	If available from the MWL; else NULL	Not allowed

Attribute Name	Tag	N-CREATE Usage	N-SET Usage
> Scheduled Protocol Code Sequence	(0040,0008)	If available from the MWL; else NULL	Not allowed
>> Code Value	(0008,0100)	From the MWL	Not allowed
>> Coding Scheme Designator	(0008,0102)	From the MWL	Not allowed
>> Coding Scheme Version	(0008,0103)	From the MWL	Not allowed
>> Code Meaning	(0008,0104)	From the MWL	Not allowed
Patient Name	(0010,0010)	Generated from the "Name" fields of the Patient ID screen	Not allowed
Patient ID	(0010,0020)	From the "MRN" field of the Patient ID screen	Not allowed
Patient's Birth Date	(0010,0030)	From the "Birth Date" field of the Patient ID screen	Not allowed
Patient Sex	(0010,0040)	From the "Gender" field of the Patient ID screen	Not allowed
Referenced Patient Sequence	(0008,1120)	If available from the MWL; else NULL	Not allowed
> Referenced SOP Class UID	(0008,1150)	Detached Patient Mgmt SOP Class UID 1.2.840.10008.3.1.2.1.1	Not allowed
> Referenced SOP Instance UID	(0008,1155)	If available from the MWL; else NULL	Not allowed
<b>Performed Procedure Step Information</b>			
Performed Procedure Step ID	(0040,0253)	If available from the MWL, else generated by EnVisor	Not allowed
Performed Station AE Title	(0040,0241)	AE Title of EnVisor	Not allowed

Attribute Name	Tag	N-CREATE Usage	N-SET Usage
Performed Station Name	(0040,0242)	If available from the MWL, else NULL	Not allowed
Performed Location	(0040,0243)	If available from the MWL, else NULL	Not allowed
Performed Procedure Step Start Date	(0040,0244)	Date of the acquisition of the first image in the study	Not allowed
Performed Procedure Step Start Time	(0040,0245)	Time of the acquisition of the first image in the study	Not allowed
Performed Procedure Step Status	(0040,0252)	"IN PROGRESS"	"COMPLETED" or "DISCONTINUED"
Performed Procedure Step Description	(0040,0254)	If "Scheduled Procedure Description" available from MWL, else "Indication" field from Patient ID screen	Not used
Performed Procedure Type Description	(0040,0255)	If "Scheduled Procedure Description" available from MWL, else "Indication" field from Patient ID screen	Not used
Procedure Code Sequence	(0008,1032)	If "Requested Procedure Code Sequence" available from the MWL; else generated by EnVisor	If "Requested Procedure Code Sequence" available from the MWL; else NULL
> Code Value	(0008,0100)	From the MWL	From the MWL
> Coding Scheme Designator	(0008,0102)	From the MWL	From the MWL

Attribute Name	Tag	N-CREATE Usage	N-SET Usage
> Coding Scheme Version	(0008,0103)	If available from the MWL; else omitted	If available from the MWL; else omitted
> Code Meaning	(0008,0104)	If available from the MWL; else omitted	If available from the MWL; else omitted
Performed Procedure Step End Date	(0040,0250)	Blank string. (Required)	Date "End Study" is pressed.
Performed Procedure Step End Time	(0040,0251)	Blank string. (Required)	Time "End Study" is pressed.
<b>Image Acquisition Results</b>			
Modality	(0008,0060)	"US"	Not allowed
Study ID	(0020,0010)	If MWL is used, set to "Requested Procedure ID" if available from the MWL; else the empty string. This "DICOM Study ID" differs from the value of "Study ID" in DICOM images.	Not allowed
Performed Protocol Code Sequence	(0040,0260)	If "Scheduled Protocol Code Sequence" available from the MWL; else NULL	If "Scheduled Protocol Code Sequence" available from the MWL; else NULL
> Code Value	(0008,0100)	From the MWL	From the MWL
> Coding Scheme Designator	(0008,0102)	From the MWL	From the MWL
> Coding Scheme Version	(0008,0103)	If available from the MWL; else omitted	If available from the MWL; else omitted
> Code Meaning	(0008,0104)	If available from the MWL; else omitted	If available from the MWL; else omitted
Performed Series Sequence	(0040,0340)	One item representing the series used for this MPPS	One item representing the series used for this MPPS

Attribute Name	Tag	N-CREATE Usage	N-SET Usage
> Performing Physician's Name	(0008,1050)	From the "Performed By" field of the Patient ID screen	From the "Performed By" field of the Patient ID screen
> Protocol Name	(0018,1030)	"CLR Standard" for all exams.	"CLR Standard" for all exams.
> Operator's Name	(0008,1070)	From the "Performed By" field of the Patient ID screen; else NULL	From the "Performed By" field of the Patient ID screen; else NULL
> Series Instance UID	(0020,000E)	Synthesized by EnVisor	Synthesized by EnVisor
> Series Description	(0008,103E)	NULL	NULL
> Retrieve AE Title	(0008,0054)	From the "System" tab of DICOM Setup	NULL
> Referenced Image Sequence	(0008,1140)	NULL	One item for each image in the MPPS
>> Referenced SOP Class UID	(0008,1150)	No items in this SQ	SOP Class UID of the image
>> Referenced Instance UID	(0008,1155)		SOP Instance UID of the image
> Referenced Non-image Composite SOP Instance Sequence	(0040,0220)	NULL	NULL
>> Referenced SOP Class UID	(0008,1150)	No items in this SQ	No items in this SQ
>> Referenced Instance UID	(0008,1155)		

**Table 21: Modality Performed Procedure Step N-CREATE and N-SET Attributes**

## 2.1.3 Association Acceptance Policy

### 2.1.3.1 Responding to a verification request from a remote DICOM server

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Acceptance Policy
      - Responding to a verification request from a remote DICOM server

EnVisor provides standard conformance to the DICOM V3.0 SOP Class as shown in the Table 22.

SOP Class Name	SOP Class UID	Role
Verification SOP Class	1.2.840.10008.1.1	SCP

**Table 22: SOP Class Supported by Verification service**

### 2.1.3.1.1 Associated Real-World Activity

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Acceptance Policy
      - Responding to a verification request from a remote DICOM server
        - Associated Real-World Activity

EnVisor will respond to *external Verification* requests. It will respond to a verification request from any DICOM server, whether or not that server is 'known' to the EnVisor system.

### 2.1.3.1.2 Accepted Presentation Contexts

Context: Expansion of Headings and sub-headings

- Application Entity Specifications
  - EnVisor AE Specification
    - Association Acceptance Policy
      - Responding to a verification request from a remote DICOM server
        - Accepted Presentation Contexts

Only one association is established for each verification attempt. When the association is opened, the presentation contexts noted in Table 23 are accepted.

Abstract Syntax		Transfer Syntax		Role	Extended Negotiation
Name	UID	Name List	UID List		
Verification SOP Class	1.2.840.10008.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	SCP	None

**Table 23: Accepted Presentation Contexts**

### 3. EnVisor as a Media Storage Application

The implementation model, application data flow diagram, functional definition of the EnVisor AE, sequencing of real world activities are the same as in section 2.

#### 3.1 File Meta Information for the EnVisor AE

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - File Meta Information for the EnVisor AE

Element	Implementation Value
Implementation Class UID	1.2.840.113543.6.6.3.2
Implementation Version Name	EnVisor_B.0

**Table 24: Implementation Identifying Information**

## 3.2 Real-World Activities

### 3.2.1 Saving a DICOM Study to removable media

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Saving a DICOM Study to removable media

The EnVisor AE conforms to the Application Profile for Ultrasound Media Storage applications. For all SOP Classes described in the Application Profile, this AE performs in the role of File Set Creator (FSC) and File Set Updater (FSU). The particular physical media available is 3.5" floppy diskette, 3.5" MOD, or CD-R. For previously imported studies, EnVisor will export the IODs using the transfer syntax and tags that were used when EnVisor originally imported the study.

Supported Application Profile	Real-World Activity	Roles	Service Class Option
STD-US-SC-SF&MF	Export Study	FSC and FSU	Interchange
STD-US-ID-SF&MF	Export Study	FSC and FSU	Interchange

**Table 25: Export study to DICOM media**

The Export DICOM Objects Application Entity acts as FSC and FSU using the Interchange Option.

When saving a DICOM study to removable media, the user can specify the photometric interpretation and transfer syntax used to create DICOM files on the media. The options available to users are listed in Table 26.

<b>Transfer Syntax and Photometric Interpretation options for removable media</b>
Uncompressed (DICOM Implicit VR Little Endian) Palette Color
Uncompressed (DICOM Implicit VR Little Endian) RGB
RLE (lossless) Compression Palette Color
RLE (lossless) Compression RGB
JPEG (lossy) Compression YBR

**Table 26: Photometric Interpretation and Transfer Syntax Options for Saving to Removable Media**

EnVisor creates special 3D and PanView internal files called 'dataset' files that are never exported to a PACS but may be optionally exported to media. These DICOM files are not exported in network storage since they are only of use to EnVisor's 3D/PanView application and EnVisor does not support networked Query/Retrieve. However, these files may optionally be exported to media with the rest of the study for archival purposes. The study could later be imported into an EnVisor system and the user would be able to click on the dataset file to enter into the 3D/PanView application

### 3.2.2 Reading a DICOM study from removable media

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media

When requested to read the media directory, the Import DICOM Objects Application Entity acts as FSR using the Interchange Option,

The user choosing the Import operation from a menu initiates importing images. See the system user manuals for a description of the specific user interface capabilities.

### 3.2.2.1 Import DICOM Object Specifications

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media
      - Import DICOM Object Specifications

The EnVisor AE conforms to the Application Profile for Ultrasound Media Storage applications. For all SOP Classes described in the Application Profile, this AE performs in the role of File Set Reader (FSR). The particular physical media available is 3.5" floppy diskette, 3.5" MOD, or CD-R. Image Display and Spatial Calibration of Single and Multi-Frame image objects on any media in the Ultrasound Application Profile is therefore supported.

Supported Application Profile	Real-World Activity	Roles	Service Class Option
STD-US-SC-SF&MF-FLOP	Import Studies	FSR	Interchange
STD-US-SC-SF&MF-MOD128	Import Studies	FSR	Interchange
STD-US-SC-SF&MF-MOD230	Import Studies	FSR	Interchange
STD-US-SC-SF&MF-MOD540	Import Studies	FSR	Interchange
STD-US-SC-SF&MF-MOD650	Import Studies	FSR	Interchange
STD-US-SC-SF&MF-MOD12	Import Studies	FSR	Interchange
STD-US-SC-SF&MF-MOD23	Import Studies	FSR	Interchange
STD-US-SC-SF&MF-CDR	Import Studies	FSR	Interchange

### Table 27: DICOM Objects Application Profiles

EnVisor's DICOM Study Import feature is designed for importing studies that *were originally exported from EnVisor*. The system will not allow the user to import ultrasound studies created by another manufacturer's system.

### 3.2.2.2 Media Storage Application Profiles

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media
      - Media Storage Application Profiles

The supported Application Profiles are listed in Table 27 DICOM Objects Application Profiles.

### 3.2.2.3 SOP Specific Conformance For “DICOM Directory” SOP Class

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media
      - SOP Specific Conformance For “DICOM Directory”

Type 1, 1C, 2, and 2C data elements present in the Basic Directory Object are supported as required in DICOM 3.0, Parts 3 and 10. They are used for properly navigating through the directory data structures, recognizing and conforming to the character set being used, and the Import Study user interface to aid in the selection of objects to import. Data elements that elicit behavior that is specific to the Application Entity are described in the sections below. If Type 2 data elements are null or if Type 3 data elements are absent, the data elements are ignored by the system and the corresponding display fields in the user interface screen(s) are left blank.

### 3.2.2.4 File-Set Identification Module

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media
      - File-Set Identification Module

Contents of the File-set Identification Module are not displayed or otherwise used in this version of EnVisor.

### 3.2.2.5 Directory Information Module

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media
      - Directory Information Module

All data elements are used as described in DICOM 3.0 Part 3 for Basic Directory Object Definitions. As stated in the Ultrasound Application Profile, “The (DICOMDIR) Directory shall include Directory Records of PATIENT, STUDY, SERIES, and IMAGE corresponding to the information object files in the File-set”. Given this requirement, EnVisor uses these directory records to identify the study to import. If there are DICOM image files on the import media that do not appear in the DICOMDIR Directory Information Module (either because references to these files were omitted or because the Directory Information Module, optional in DICOM but required in the Ultrasound Application Profile, does not exist), these files are not recognized by the system.

EnVisor ignores directory Record Types other than those above.

EnVisor also ignores the “File-set consistency Flag” (0004, 1212).

### 3.2.2.5.1 Patient Directory Record

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media
      - Directory Information Module
        - Patient Directory Record

Attribute Name	Tag	Type	Usage
Specific Character Set	(0008, 0005)	1C	The default DICOM character set and optional set ISO-IR 100 (Latin 1) are supported. See Section A.7 for details.
Patient Name	(0010, 0010)	2	Displayed to help the user identify the patient folder in which to place the studies for this patient.
Patient ID	(0010, 0020)	1	Displayed to help the user identify the patient folder in which to place the studies for this patient.

**Table 28: Specific Usage of Patient Directory Record Information**

### 3.2.2.5.2 Study Directory Record

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media
      - Directory Information Module
        - Study Directory Record

Attribute Name	Tag	Type	Usage
Specific Character Set	(0008, 0005)	1C	The Default DICOM character set and optional set ISO-IR 100 (Latin 1) are supported. See Section A.7 for details.
Study Date	(0008, 0020)	1	Used in displaying list of studies to user
Study Time	(0008, 0030)	1	Used in displaying list of studies to user
Accession Number	(0008, 0050)	2	Stored in the system database
Study Description	(0008, 1030)	2	Generated
Study Instance UID	(0020, 000D)	1C	Stored in the system database
Study ID	(0020, 0010)	1	Stored in the system database

**Table 29: Specific Usage of Study Directory Record Information**

### 3.2.2.5.2.1 Series Directory Record

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media
      - Directory Information Module
        - Series Directory Record

Attribute Name	Tag	Type	Usage
Specific Character Set	(0008, 0005)	1C	The default DICOM character set and optional set ISO-IR 100 (Latin 1) are supported. See Section A.7 for details.
Modality	(0008, 0060)	1	Only US is supported. Other modalities are ignored.
Series Description	(0008, 103E)	3	Stored
Series Number	(0020, 0011)	1	Stored

**Table 30: Specific Usage of Series Directory Record Information**

### 3.2.2.5.2.2 Image Directory Record

Context: Expansion of Headings and sub-headings

- EnVisor as a Media Storage Application
  - Real-World Activities
    - Reading a DICOM study from removable media
      - Directory Information Module
        - Image Directory Record

Attribute Name	Tag	Type	Usage
Specific Character Set	(0008, 0005)	1C	The default DICOM character set and optional set ISO-IR 100 (Latin 1) are supported. See Section A.7 for details.
Referenced File ID	(0004, 1500)	1C	Used
Referenced SOP Class UID in File	(0004, 1510)	1C	Used
Referenced SOP UID in File	(0004, 1511)	1C	Used
Referenced Transfer Syntax UID in File	(0004, 1512)	1C	Used
Image Date	(0008, 0023)	3	Used for ordering the thumbnail display. On Export, comes from the image.
Image Time	(0008, 0033)	3	Used for ordering the thumbnail display. On Export, comes from the image.

**Table 31: Specific Usage of Image Directory Record Information**

#### 4. Communications Profiles

EnVisor provides DICOM V3.0 TCP/IP Network Communication Support as defined in Part 8 of the DICOM Standard.

## 5. Extensions/Specializations/Privatizations

### 5.1 2D

Context: Expansion of Headings and sub-headings

- Extensions/Specializations/Privatizations
  - 2D

There are no extensions, specializations or privatizations in EnVisor's 2D application.

### 5.2 3D Stills and Loops, and PanView

Context: Expansion of Headings and sub-headings

- Extensions/Specializations/Privatizations
  - 3D Stills and Loops, and PanView

3D and PanView image files contain the following private tags for use by EnVisor's 3D/PanView application:

Attribute Name	Tag	Type	VR	Description	Value
Private Creator	7777, 0010		LO	3D/PanView component type	"Philips EnVisor"
	7777, 1001		DA	Date 3D/PanView internal 'dataset' file was acquired	
	7777, 1002		CS	Type of 3D or PanView	3D Still: "3D STILL" 3D Loop: "3D MOVIE" PanView still: "PANVIEW STILL"
	7777,		LO	Internal 3D/PanView	"1.0"

Attribute Name	Tag	Type	VR	Description	Value
Private Creator	7777, 0010		LO	3D/PanView component type	"Philips EnVisor"
	1003			software version number	
	7777, 1014		LT	Private string	

## 6. Configuration

The DICOM setup screen allows the user to configure a significant number of options including:

- For the EnVisor system, it's AE Title and Port number.
- For DICOM servers, their AE Title, port number, IP address.
- For Storage SCP's, the photometric interpretation of image pixel data: Palette Color, RGB, or YBR.
- For media storage, the photometric interpretation of pixel data and the compression format: Palette Color uncompressed, Palette Color RLE compressed, RGB uncompressed, RGB RLE compressed or YBR JPEG compressed.
- For DICOM Printers, all DICOM configuration settings
- For a MWL server, the query parameters: scheduled procedure start range, modality, AE Title.

## 7. Support for Extended Character Sets

EnVisor will offer support for Japanese, Chinese, and Russian. This includes translating system text into these languages and allowing the user to input Japanese, Chinese, and Cyrillic characters into the system. One important aspect of this is that the user will be able to enter these special characters into the Patient ID screen.

The present DICOM standard allows Code Extension Techniques for multi-byte characters. Therefore, as well as the default character set (ISO-IR 6), EnVisor supports the following extended character sets:

- ISO-IR 100 Latin Alphabet No. 1
- ISO-IR 87 Japanese Kanji (ideographic), Hiragana (phonetic) and Katakana (phonetic)
- ISO-IR 13 Japanese Katakana (phonetic)
- ISO-IR 159 Supplementary Kanji (ideographic)
- ISO-IR 144 Russian Cyrillic

### Important Note:

When an Application Entity which does not support Code Extension Techniques receives a Data Set which includes multi-byte characters from an EnVisor system, misrepresentation of characters may occur.

The DICOM standard states that it is the responsibility of the Application Entity which receives the Data Sets to take whatever action is considered necessary to minimize the effect of misrepresented characters. It is not the responsibility of the EnVisor system to take such action.

## 7.1 Support for Russian and Japanese Markets

EnVisor uses "Code-extension techniques" to encode Japanese stroke based characters and Russian Cyrillic characters in DICOM tags with value representations of SH, LO, ST, LT, UT, and PN.

The technique requires two things in a DICOM file that contains these characters:

1. Add the Optional Specific Character Set TAG (0008,0005) and set the value to the list of identifiers for all the non-standard character sets that will appear in any string in the file separated by backslashes. For example:

For Japanese systems:

(0008,0005) = "ISO 2022 IR 13\ISO 2022 IR 87\ISO 2022 IR 159\ISO 2022 IR 100"

For Russian systems:

(0008,0005) = "ISO 2022 IR 144\ISO 2022 IR 100"

For English systems:

(0008,0005) = "ISO 2022 IR 100"

2. Embed escape sequences in the strings that contain Asian or Cyrillic characters to cause the DICOM interpreting code to switch from one character set to another.

The escape sequences to be used are defined as:

"<ESC>\$B" ISO - IR 87 Japanese Kanji (ideographic), Hiragana (phonetic), Katakana (phonetic)

"<ESC>(B" ISO - IR 6 ASCII - DICOM default character set

"<ESC>\$(D" ISO - IR 159 Supplementary Kanji (ideographic)

"<ESC>(J" ISO - IR 144 Russian Cyrillic

## 7.2 Additional Support for Japanese Markets

Japanese markets will have additional fields to the Patient ID screen so that the user can enter the Roman, Ideographic, and Phonetic representations of a patient's name.

The DICOM patient name field, tag (0010,0010) of type PN, is a single string field that contains up to five components (last, first, middle, title, honorific) in up to three language variants (Roman, Idiographic, and Phonetic.) The format of the patient name field is:

```
“Roman-last^Roman-first^Roman-middle^Roman-prefix^Roman-suffix=  
Ideographic-last^Ideographic-first^Ideographic-middle^Ideographic-prefix^Ideographic-suffix=  
Phonetic-last^Phonetic-first^Phonetic-middle^Phonetic-prefix^Phonetic-suffix”
```

In the above string the five components are separated with the ‘^’ Ascii character and the three language variants are separated by the ‘=’ Ascii character. The only required component is the Roman Last name. All other components are optional. Trailing ‘^’ and ‘=’ characters can be excluded.

When this string is encoded in a DICOM image file or DICOMDIR directory file, the escape sequences appropriate for the character sets used are inserted into the string for storage as a single-byte string. On media import the escape sequences are removed.

### 7.3 Support for Chinese Markets

The current DICOM standard as of this release of EnVisor does not support Chinese character sets. EnVisor however provides support for Chinese customers so that they can enter text using Chinese characters.

If the system is set up for Chinese, then (unlike for Japanese markets) the user can enter just one version of the patient name. This would make Chinese systems work in the same way as Russian, English, French, Italian, and Spanish systems. The Chinese user will be able to enter the patient name using a combination of Chinese and Roman characters – all of the characters will appear wherever the system displays the patient name (image, report, Search for Study window, etc.).

Since the DICOM Standard does not offer support for Chinese characters, all Chinese characters entered into the Patient ID screen will be lost if a user exports or backs up a study to media. This will be noticed when the study is imported back into the system; upon import, each Chinese character will be replaced with a question mark (“?”) character. The question marks will make it obvious to the user that the characters were lost.

If the user enters a patient name that consists entirely of Chinese characters, then the name will come back as “??????”. In this case, the user will have to identify the study in the “Import Study” and “Search for Study” windows by the MRN. If the user enters a patient name that consists of a combination of Roman and Chinese characters, then Roman characters will be preserved, and the name will come back as something like “Lee ??????????”. This will give users who like to back up their studies the flexibility of

entering a patient name with a combination of Roman and Chinese characters, and have at least part of the name come back during import.

Note that the original Chinese name will be “burned into” study images that are exported to media. These Chinese characters will remain on the images when the studies are imported back into the system.