PHILIPS

ECG Safety Cable Lead Sets

**M1601A, M1603A, M1605A, M1609A,
M1611A, M1613A, M1615A, M1619A,
M1621A, M1623A, M1625A, M1629A,
M1631A, M1633A, M1635A, M1639A**

Cleaning, Disinfecting and Sterilizing

For more information, please call your local Philips sales office listed in your telephone directory or a Philips regional office listed below for the location of your nearest sales office.

United States:

Philips Medical Systems
Cardiac and Monitoring Systems
Headquarters
3000 Minuteman Road
Andover, MA 01810, USA
(800) 934-7372

Canada:

Philips Medical Systems Canada
281 Hillmount Road
Markham, ON
L6C 2S3
(800) 291-6743

Latin America Headquarters:

Philips Medical Systems
1550 Sawgrass Corporate Parkway #300
Sunrise, FL 33323
Tel: (954) 835-2600
Fax: (954) 835-2626

Europe, Middle East and Africa:

Philips Medizin Systeme Böblingen GmbH
Cardiac and Monitoring Systems
Hewlett-Packard Str. 2
71034 Böblingen
Germany
Fax: (+49) 7031 463 1552

Asia Pacific Headquarters:

Philips Medical Systems
30/F Hopewell Centre
17 Kennedy Road
Wanchai
Hong Kong
Tel: (852) 2821 5888
Fax: (852) 2527 6727

<http://shop.medical.philips.com>

Manufacturing Address:

Philips Medizin Systeme
Böblingen GmbH
Hewlett-Packard-Strasse 2
71034 Böblingen
Germany

Subject to modification
© 2004 Philips Medizin Systeme
Böblingen GmbH
Printed in Germany 09/2004
M1601-9001A
4512 610 04871

English

Cleaning

To keep your lead set free of dust and dirt clean it with a lint free cloth, moistened with either warm water (40°C/104°F maximum) and soap, a diluted non-caustic detergent or one of the approved cleaning agents listed below. Never immerse or soak the lead sets. Do not allow the cleaning agent to remain on the lead set – remove it immediately by wiping with a cloth dampened with water.

If you see signs of deterioration or damage, replace the lead set. Do not use it for further patient monitoring.

Recommended Cleaning Agents and Brands

Soaps	mild soaps
Tensides	dishwasher detergents: Edisonite Schnellreiniger®, Alconox®
Ammonias	Dilution of Ammonia <3%, Windowcleaner
Alcohol based ^a	Ethanol 70%, Isopropanol 70%, Windowcleaner

a. Do not use alcohol with non-shielded (purple) lead sets M1609A, M1619A, M1629A, M1639A

Disinfecting

We recommend that you disinfect the lead set only when necessary as determined by your hospital's policy, to avoid long term damage to the lead set. Never immerse or soak the lead sets. Do not allow the disinfectant to remain on the lead set – remove it immediately by wiping with a cloth dampened with water.

We also recommend that the lead set be cleaned first as described under "Cleaning".

Recommended Disinfecting Substances

Alcohol based ^a	Ethanol 70%, Isopropanol 70%
Aldehyde based	Cidex®

a. Do not use alcohol with non-shielded (purple) lead sets M1609A, M1619A, M1629A, M1639A

Sterilizing

We recommend that you sterilize only when necessary as determined by your hospitals policy, to avoid long term damage to the lead set. We also recommend that the lead set be cleaned first as described in "Cleaning".

The lead set has been tested to withstand Ethylene Oxide (EtO) gas sterilization. Be sure that all safety precautions regarding aeration after EtO exposure are followed.

Caution Do not autoclave the lead set or use bleaches containing sodium hypochlorite (for example, Clorox™).

Ordering Information

All lead sets listed below are for use with Philips monitoring equipment only.

3-Electrode Lead Sets

AAMI	OR* Grabber, shielded	1.0m	M1601A
	ICU Grabber, shielded	1.0m	M1603A
	ICU Snap, shielded	1.0m	M1605A
	ICU Clip, non-shielded	0.7m	M1609A
IEC	OR* Grabber, shielded	1.0m	M1611A
	ICU Grabber, shielded	1.0m	M1613A
	ICU Snap, shielded	1.0m	M1615A
	ICU Clip, non-shielded	0.7m	M1619A

5-Electrode Lead Sets

AAMI	OR* Grabber, shielded	1.0m/1.6m	M1621A
	ICU Grabber, shielded	1.0m/1.6m	M1623A
	ICU Snap, shielded	1.0m/1.6m	M1625A
	ICU Clip, non-shielded	0.7m/1.3m	M1629A
IEC	OR* Grabber, shielded	1.0m/1.6m	M1631A
	ICU Grabber, shielded	1.0m/1.6m	M1633A
	ICU Snap, shielded	1.0m/1.6m	M1635A
	ICU Clip, non-shielded	0.7m/1.3m	M1639A

* You cannot measure respiration if you are using an OR ECG cable.

Federal Law (U.S.) restricts this device to sale by or on the order of a physician.

These products are for use only in combination with Philips Patient Monitors when listed as accessories in the monitor's User Documentation.

The ECG Patient Lead Sets comply with the requirements of the Council Directive 93/42/EEC of 14 June 1993 (Medical Device Directive).

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