PHILIPS

ECG Safety Trunk Cables

**M1500A, M1510A, M1520A,
M1530A, M1540C, M1550C,
M1560C, M1570C, M1590A**

Cleaning, Disinfecting and Sterilizing

For more information, please call your local Philips sales office listed in your telephone directory or a Philips regional office listed below for the location of your nearest sales office.

United States:
Philips Medical Systems
Cardiac and Monitoring Systems
Headquarters
3000 Minuteman Road
Andover, MA 01810, USA
(800) 934-7372

Canada:
Philips Medical Systems Canada
281 Hillmount Road
Markham, ON
L6C 2S3
(800) 291-6743

Latin America Headquarters:
Philips Medical Systems
1550 Sawgrass Corporate Parkway #300
Sunrise, FL 33323
Tel: (954) 835-2600
Fax: (954) 835-2626

Europe, Middle East and Africa:
Philips Medizin Systeme Böblingen GmbH
Cardiac and Monitoring Systems
Hewlett-Packard Str. 2
71034 Böblingen
Germany
Fax: (+49) 7031 463 1552

Asia Pacific Headquarters:
Philips Medical Systems
30/F Hopewell Centre
17 Kennedy Road
Wanchai
Hong Kong
Tel: (852) 2821 5888
Fax: (852) 2527 6727

<http://shop.medical.philips.com>

Manufacturing Address:
Philips Medizin Systeme
Böblingen GmbH
Hewlett-Packard-Strasse 2
71034 Böblingen
Germany

Subject to modification
© 2004 Philips Medizin Systeme
Böblingen GmbH
Printed in Germany 09/2004
M1500-9001A
4512 610 04841

English

ECG Safety Trunk Cables

Cleaning

To keep your cable free of dust and dirt clean it with a lint free cloth, moistened with either warm water (40°C/104°F maximum) and soap, a diluted non-caustic detergent or one of the approved cleaning agents listed below. Never immerse or soak the cables. Do not allow the cleaning agent to remain on the cable – remove it immediately by wiping with a cloth dampened with water.

If you see signs of deterioration or damage, replace the cable. Do not use it for further patient monitoring.

Recommended Cleaning Agents and Brands

Soaps	mild soaps
Tensides	dishwasher detergents: Edisonite Schnellreiniger®, Alconox®
Ammonias	Dilution of Ammonia <3%, Windowcleaner

Disinfecting

We recommend that you disinfect the cable only when necessary as determined by your hospital's policy, to avoid long term damage to the cable. Never immerse or soak the cables. Do not allow the disinfectant to remain on the cable – remove it immediately by wiping with a cloth dampened with water.

We also recommend that the cable be cleaned first as described under "Cleaning".

Recommended Disinfecting Substances

Aldehyde based	Cidex®
----------------	--------

Sterilizing

We recommend that you sterilize only when necessary as determined by your hospitals policy, to avoid long term damage to the cable. We also recommend that the cable be cleaned first as described in "Cleaning".

The cable has been tested to withstand Ethylene Oxide (EtO) gas sterilization. Be sure that all safety precautions regarding aeration after EtO exposure are followed.

Caution Do not autoclave the cable or use bleaches containing sodium hypochlorite (for example, Clorox™).

Ordering Information

All trunk cables listed below are for use with Philips monitoring equipment only.

3-Lead Trunk Cables

AAMI	2.7m	M1500A
	0.9m	M1540C
IEC (8-pin instrument plug)	2.7m	M1510A
	2.7m	M1590A
	0.9m	M1550C

5-Lead Trunk Cables

AAMI	2.7m	M1520A	EASI Label*
	0.9m	M1560C	5967-2069
IEC	2.7m	M1530A	5967-2070
	0.9m	M1570C	5967-2070

* If you are using EASI lead-placement, replace the standard lead-placement label on the cable with the appropriate EASI label.
EASI is a trademark of Zymed Inc.

This symbol indicates that the cables and accessories are designed to have special protection against electric shocks (particularly regarding allowable leakage currents), and are defibrillator proof.

Federal Law (U.S.) restricts this device to sale by or on the order of a physician.

These products are for use only in combination with Philips Patient Monitors when listed as accessories in the monitor's User Documentation.

The ECG Patient Cables comply with the requirements of the Council Directive 93/42/EEC of 14 June 1993 (Medical Device Directive).

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