SpO₂ Adapter and Extension Cables

M1900B

M1940A

M1941A

M1943A

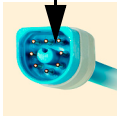
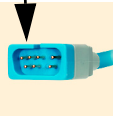
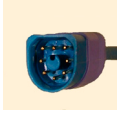
M1943AL

M1943NL

PHILIPS

Contents

English Instructions for Use	2	Български Указания за употреба	27
Manuel d'utilisation – Français	3	Eestikeelne kasutusjuhend	29
Gebrauchsanweisung – Deutsch	5	Naudojimo instrukcija lietuvių kalba	30
Nederlandse gebruiksaanwijzing	6	Lietošanas instrukcijas latviešu valodā	32
Instrucciones de uso - Español	8	Инструкция по эксплуатации на русском языке	33
Português - Instruções de Uso	9	Ελληνικά Οδηγίες Χρήσης	35
Istruzioni d'uso - Italiano	11	Türkçe Kullanım Talimatları	36
Norsk brukerhåndbok	12	ユーザーズ・ガイド（日本語）	38
Svensk bruksanvisning	14	繁體中文使用說明	39
Suomenkieliset käyttöohjeet	15	简体中文使用说明	41
Dansk brugerhåndbog	17		
Magyar nyelvű használati útmutató	18		
Instrukcja obsługi w języku polskim	20		
Český návod k obsluze	21		
Návod na používanie v slovenčine	23		
Navodila za uporabo v slovenščini	24		
Instrucțiuni de utilizare în limba română	26		

				
1	M1943A M1943AL			
2	M1940A			
3	M1900B			
4	M1941A			
5	M1943NL			

English Instructions for Use

These cables connect a validated SpO₂ sensor to HP/Agilent/Philips patient monitors if the cable is listed as an accessory in the monitor's Instructions for Use.

Before use, verify that combination of instrument/adaptor cable/extension/sensor is specified in the HP/Agilent/Philips Instructions for Use. Unspecified combinations may result in patient injury.

Key to illustration on page 1

1	M1940A (2m adapter)	12-pin cable to 8-pin
2	M1900B (3m adapter)	12-pin to 9-pin D-sub
3	M1941A (2m extension)	8-pin to 8-pin
4	M1943A (1.1m adapter) M1943AL (3m adapter)	8-pin to 9-pin D-sub 8-pin to 9-pin D-sub
5	M1943NL (3m adapter) (with lavender-blue plug jacket)	8-pin to 9-pin D-sub (OXIMAX)

OXIMAX™ is a registered trademark of Nellcor Puritan Bennett Inc.

Cleaning and Disinfecting

Use only the approved agents listed. Others may damage the cable or its connecting wires, shorten the product lifetime, or cause safety hazards.

Never immerse or soak the cables or connectors in liquid. Do not allow the cleaning agent or disinfectant to remain on the cable; wipe it off immediately using a cloth dampened with water.

If you see signs of physical or electrical deterioration or damage, replace the cable. Do not use it for further patient monitoring. Disinfect, decontaminate, and dispose of it according to local laws for hospital waste disposal.

Cleaning

Use a lint-free cloth moistened with warm water (40°C/104°F maximum) and soap, or a diluted non-caustic detergent, or a 1% salt solution.

Disinfecting

Use a lint-free cloth moistened with one of these low level disinfectants:

Alcohol based	Ethanol 70%, Isopropanol 70%, Microzid Liquid®
Aldehyde based	Cidex® Opa Solution

Clean the cable before disinfecting, and disinfect only when necessary, as determined by your hospital's policy, to avoid long-term damage to the cable.

Do not autoclave the adapter cable or use bleaches containing sodium hypochlorite such as Clorox.

Do not sterilize or expose the adapter cable to any radiation, such as X-Ray.

 M1900B M1940A	Always use separate collection for waste electrical and electronic equipment (WEEE).  0°C 32°F 40°C 104°F Operating temperature
 For further information about these cables, including specifications, warnings and cautions, consult the User's Guide for your monitor.	

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© Copyright 2007 Koninklijke Philips Electronics N.V.
All rights reserved.
Printed in China 10/2007

The SpO₂ adapter cables comply with the requirements of the Council Directive 93/42/EEC of 14 June 1993 (Medical Device Directive).
All devices described herein are latex-free.

4512 610 24671
M1900-9001C

For more information, please call your local Philips sales office listed in your telephone directory or a Philips regional office listed below for the location of your nearest sales office.

United States:

Philips Medical Systems
North America Corporation
3000 Minuteman Road
Andover, MA 01810-1099 USA
(800) 225-0230

Canada:

Philips Medical Systems Canada
281 Hillmount Road
Markham, ON Canada
L6C 2S3
(800) 291-6743

Latin America Headquarters:

Philips Medical Systems
1550 Sawgrass Corporate Parkway
#300
Sunrise, FL 33323 USA
Tel: (954) 835-2600
Fax: (954) 835-2626

Europe, Middle East and Africa:

Philips Medizin Systeme
Böblingen GmbH
Cardiac and Monitoring Systems
Hewlett-Packard Str. 2
71034 Böblingen, Germany
Fax: (+49) 7031 463 1552

Asia Pacific Headquarters:

Philips Medical Systems
30/F Hopewell Centre
17 Kennedy Road
Wanchai
Hong Kong
Tel: (852) 2821 5888
Fax: (852) 2527 6727

R_x only