

PRODUCT RECYCLING PASSPORT

<b>Product name:</b>	<b>Essential MRI Patient Monitor</b>	
<b>Identification code(s):</b>	865353	
<b>Total Weight (in kg):</b>	<b>2.1 lb (1.0 kg)</b>	
<b>Producer / Manufacturer:</b>	Name company:	Invivo Corporation
	Address:	12151 Research Parkway, Suite 200, Orlando, FL
	Zip code:	32826
	Country:	USA
	Electronic info:	A-865353-90004

<b>Recycle info</b>	<b>Item</b>	<b>Location</b>
<b>Special Attention</b> 	None	Not applicable
<b>Fluids / Gases</b> 	None	Not applicable
<b>Batteries</b>  <b>To be removed</b>	<b>Substances</b> Lithium-ion, Quantity 1 (used with the wireless module) Lithium-polymer, Quantity 1 (used with the display) Lithium coin (Lithium / Manganese Dioxide (Li/MnO <sub>2</sub> )), Quantity 1 (within the display)	Figure 1 Figure 2 Figure 3
<b>Hazardous</b>  <b>To be removed</b>	None	Not applicable

To remove the module battery, press both locking tabs and then slide the battery out of the module.




**Figure 1: Battery Location, Wireless Module**

To remove the display battery, move the locking tab on the battery to the right and lift the battery.



**Figure 2: Battery Location, Display (external)**

The lithium coin battery is located on the innermost side of the printed circuit board inside the device.

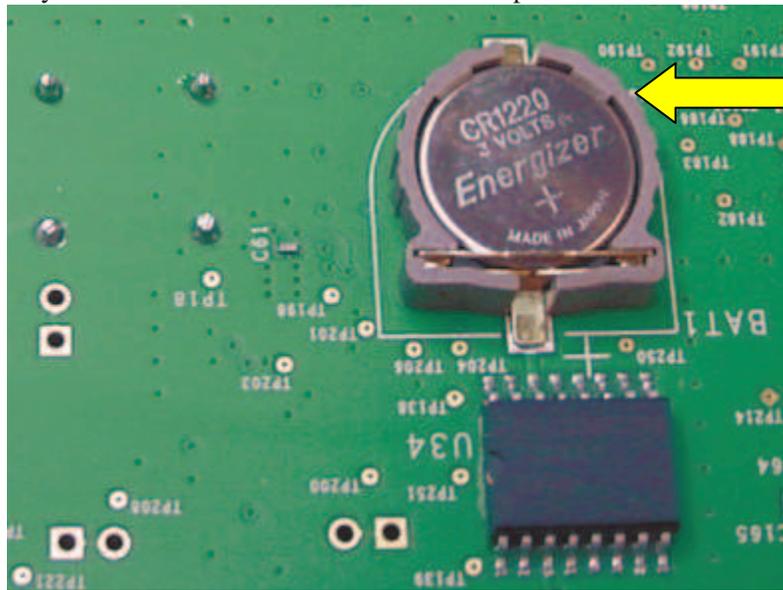

**Figure 3: Battery Location, Display (internal)**