


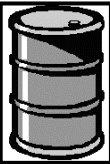



Product name:	HeartStart XL Defibrillator	
Identification code	M4735A	
Total weight (in Kg)	Up to 6.5 kgs	
Producer /Manufacturer	Name company:	Philips Medical Systems
	Address:	3000 Minuteman Road, Andover, MA
	Zip code:	01810
	Country:	USA
	Electronic info:	www.philips.pms.recycleinfo.com

Recycle info	Substances:	Location
Special attention	Item	Location
	<p>WARNING: Remove all power sources (AC, external battery). See Figure 1. Wait at least 2 hours for any charge on the high voltage defibrillation capacitor and/or power supply capacitors to discharge before beginning disassembly. After waiting 2 hours, fully discharge the high voltage capacitor prior to touching inside the device. Dangerous voltages may be present on components and connections. Using an insulated probe, short between the red and white wire attachment points on the pc board. See Figure 2. Failure to do so could result in serious injury or death.</p>	Figure 1 & 2
Batteries  To be removed	Type:	Location
	External Lead (Pb) Acid	Figure 1
	Internal Lithium	Figure 3
Hazardous  To be removed	LCD display backlight tubes contain mercury (Hg)	Figure 4
Fluids / Gases 	Item	Location
	None	

Locations as mentioned in the Passport (Pictures information)



AC Cord



External Lead (Pb)
Acid Battery



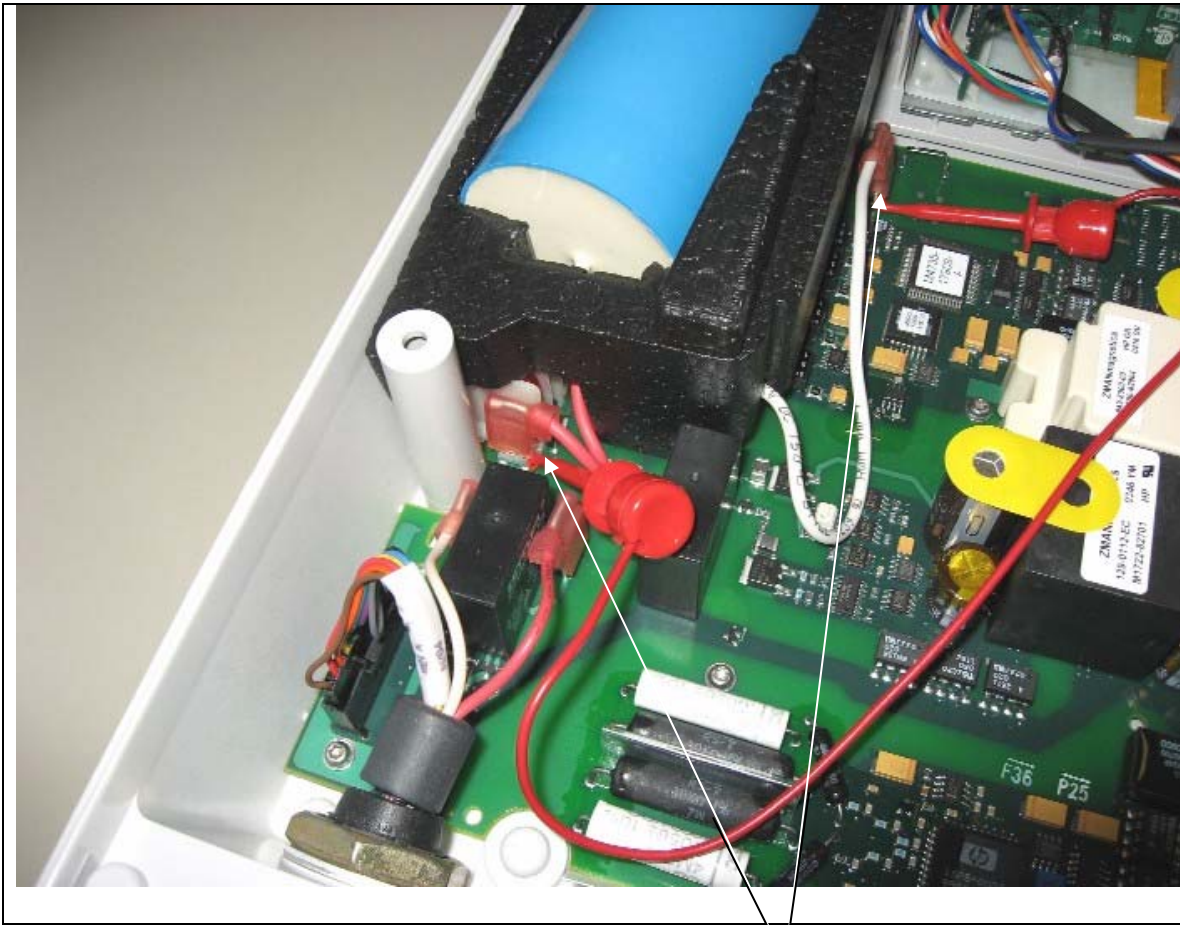


Figure 1



Discharge High voltage
Capacitor




Figure 2

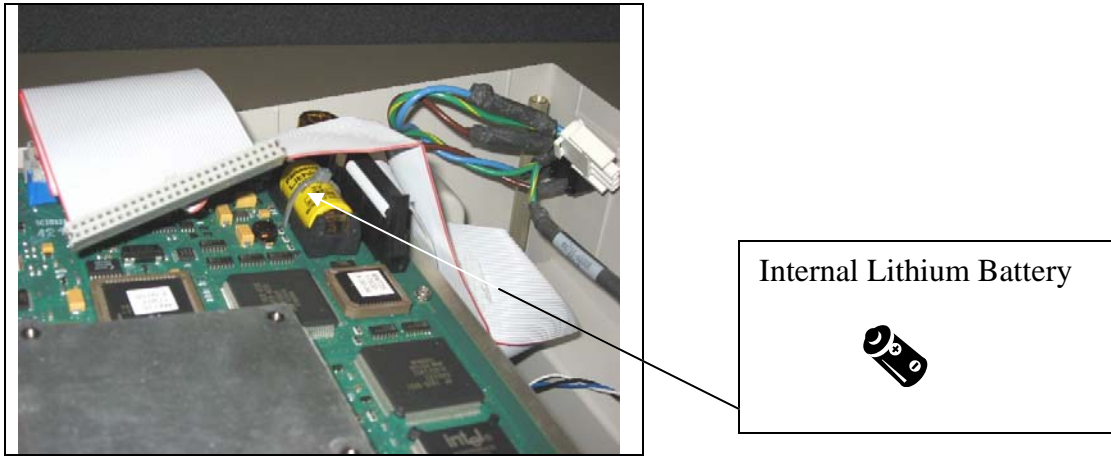


Figure 3

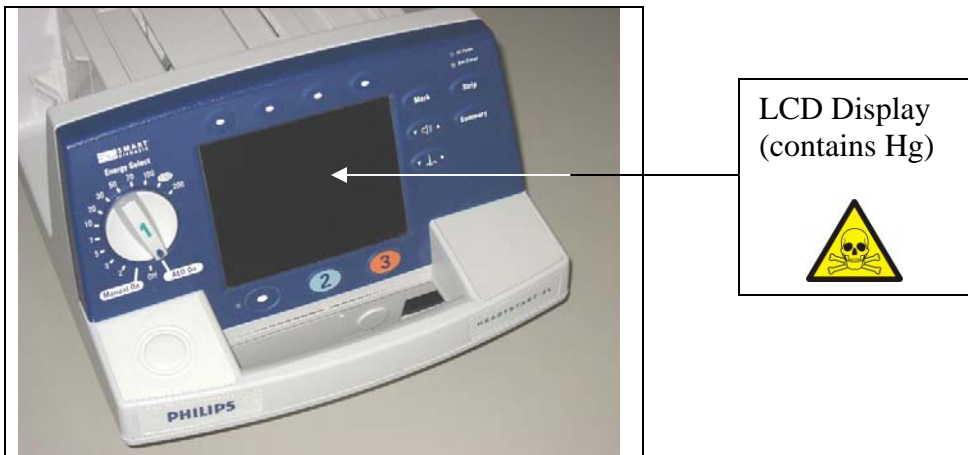


Figure 4