

Mask settings for System One Resistance Control

System One Resistance Control is a unique technology incorporated into the Philips Respironics Sleep Therapy System that knows how to enhance the device's pressure delivery

for the Philips Respironics masks listed below. The table outlines the System One Resistance Control settings for Philips Respironics' nasal, minimal contact, and full-face masks.

Nasal masks	Size	System One setting
TrueBlue	All sizes	X1
EasyLife	All sizes	X1
ComfortGel Blue	All sizes	X1
ComfortFusion	All sizes	X1
ComfortSelect	All sizes	X1
ComfortClassic	All sizes	X1
Profile Lite	All sizes	X1

Full-face masks	Size	System One setting
Amara	All sizes	X1
FitLife	All sizes	X1
FullLife	All sizes	X1
ComfortGel Full	All sizes	X1
ComfortFull 2	All sizes	X1

Minimal contact masks	Size	System One setting
GoLife	Petite	X4
	Small	X3
	Medium	X3
	Large	X2
OptiLife	Pillows – petite	X3
	Pillows – small	X4
	Pillows – medium	X2
	Pillows – large	X1
	CradleCushion – all sizes	X1
ComfortLite 2	Direct Seal – 1	X2
	Direct Seal – 2	X2
	Direct Seal – 3	X2
	Direct Seal – 4	X1
	Direct Seal – 5	X2
	Direct Seal – 6	X2
	Pillows – petite	X4
	Pillows – small	X4
	Pillows – medium	X2
	Pillows – large	X1
	Simple Cushion – all sizes	X1

Respironics, Amara, ComfortClassic, ComfortFull, ComfortFusion, ComfortGel, ComfortLite, ComfortSelect, EasyLife, FitLife, FullLife, GoLife, OptiLife, Profile Lite, and TrueBlue are trademarks of Koninklijke Philips Electronics N.V. and its affiliates. All rights reserved.

©2012 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

CAUTION: US federal law restricts these devices to sale by or on the order of a physician.
www.philips.com/respironics +1 724 387 4000 +1 800 345 6443

edoc JJ 06/18/12 MCI 4104974

PHILIPS

RESPIRONICS

sense and simplicity